

# OC-Light™ S FIT



## Specifications

- Easy one-step operation
- High haemoglobin sensitivity  
50ng hHb/mL (10µg hHb/g of stool)
- Immunochemical method
- Result in 5 minutes
- Hygienic and easy sampling operation

## Correlation

		OC-Light™ 'EIKEN'		
		Positive	Negative	Total
OC-LIGHT™S FIT	Positive	84	0	84
	Negative	1	62	63
	Total	85	62	147

Positive agreement rate  
98.8%(84/85)  
Negative agreement rate  
100.0%(62/62)  
Overall agreement rate  
99.3%(146/147)

( in house data )

# Procedure

## Specimen Collection

STEP1


- Open green cap by turning to the left and pulling upwards.

STEP2

- Collect faecal sample by scraping surface of faecal widely with the sample probe.

STEP3

- Close sampling bottle by inserting the sample probe and screwing the cap tightly to the right. Shake the sampling bottle vigorously. **DO NOT REOPEN.**

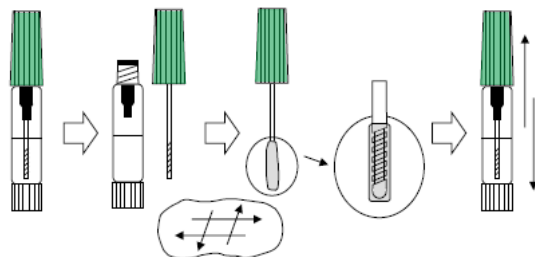

## Test Procedure

STEP4


- Remove the white cap on the sampling bottle .

STEP5

- Drop the sample end of the test strip into the sampling bottle.

STEP6

- Read results at 5 minutes.

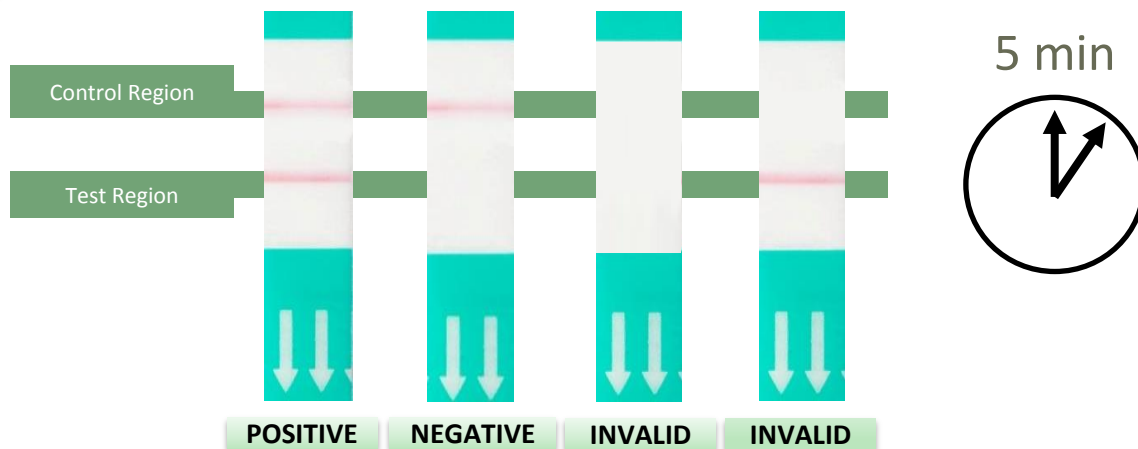

Product code	Product name	Contents	Storage
V-PC52	OC-Light™ FIT Test Strips	10 × 50 strips	2-30°C
V-PH82	OC-Light™ FIT Sampling Bottles	20 × 50 bottles	2-30°C

Read the Instructions for Use carefully before Use.



**EIKEN CHEMICAL CO., LTD.**  
4-19-9, Taito, Taito-ku, Tokyo 110-8408, Japan  
<http://www.eiken.co.jp/en>

6172 AMK

August 2015