Instructions for Use

380097-C

English

REF V-PH52

OC-Calibrator 2 kit

(For OC-SENSOR PLEDIA, OC-SENSOR io)

INTENDED USE

This product is intended for use as a calibration agent (hemoglobin concentration: 1000 ng/mL) when using automated immunochemical analyzers [OC-SENSOR PLEDIA, OC-SENSOR io] to measure faecal hemoglobin.

REAGENTS

Reagents are stable before and after opening until the date printed on the label when stored at 2-8°C.

OC-Calibrator 2 1 × 3 mL, Ready to use.
OC-Calibrator Diluent 1 × 45 mL, Ready to use.

WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use only.
- Do not freeze products.
- Do not use expired products.
- Air bubbles on the surface of the product after being poured into sample cups can result in error measurement, and should be removed.
- OC-Calibrator 2 is manufactured from human blood material. Only blood testing negative for HBs antigens, HIV (HIV-1 and HIV-2) antibodies and HCV antibodies is used, but the presence of pathogens and infection risk cannot be completely ruled out. Therefore, the product should be handled in the same manner as patient samples to ensure safety.
- The Hemoglobin concentration is displayed on the label.
- Do not use for either OC-SENSOR DIANA or OC-SENSOR Micro.

PROCEDURE

Create the calibration curve in the manner described by the automatic analyzer manual or briefly described below.

[OC-SENSOR PLEDIA, OC-SENSOR io]

Pour 600 µL (12 drops or more) of OC-Calibrator 2 into a sample cup. Pour 2mL (or more) of OC-Calibrator Diluent into another sample cup. Set these sample cups on a rack according to the user manual and create calibration curve by the analyzer.

Volume of 1 drop from the vial of OC-Calibrator 2 is approximately 50 µL.

Creating a new calibration curve is required under the following conditions:

- When the lot of latex reagent is changed.
- When the value of OC-Control (LV1/LV2/LV3) is not within the indicated range.
- When opened latex reagent vials are re-used after over 14 days storage at 2-10°C.
- When opened latex reagent vials are left onboard for more than 7 days.
- After maintenance support procedures.

PRODUCT CODE, PRODUCT NAME & STORAGE

Product code	Product name	Contents	Storage
V-PH52	OC-Calibrator 2 kit	Calibrator 2 : 1 × 3 mL Diluent : 1 × 45 mL	2-8°C

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Advena Ltd.
EIKEN CHEMICAL CO., LTD.