# Instructions for Use

380098-D

English

<b>OC-Control LV1</b>	<b>REF</b> V-PH53
<b>OC-Control LV2</b>	<b>REF</b> V-PH54
<b>OC-Control LV3</b>	<b>REF</b> V-PH59

## INTENDED USE

OC-Control LV1/LV2/LV3 is intended for use as a liquid internal quality control in clinical laboratories making use of OC-SENSOR automated immunochemical analyzers [OC-SENSOR DIANA, OC-SENSOR PLEDIA, OC-SENSOR Micro, OC-SENSOR io] to measure faecal hemoglobin.

## REAGENTS

Reagents are stable before and after opening until the date printed on the label when stored at 2-8°C.

OC-Control LV1	2 × 5 mL,	Ready to use.
OC-Control LV2	2 × 5 mL,	Ready to use.
OC-Control LV3	2 × 5 mL,	Ready to use.

## WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use only.
- Do not freeze product.
- Do not use expired product.
- Air bubbles on the surface of the product after being poured into sample cups can result in error measurement, and should be removed.
- OC-Control is manufactured from human blood material. Only blood testing negative for HBs antigens, HIV (HIV-1 and HIV-2) antibodies, and HCV antibodies is used, but the presence of pathogens and infection risk cannot be completely ruled out. Therefore, the product should be handled in the same manner as patient samples to ensure safety.
- The hemoglobin concentration is displayed on the label.
- The product should be used with OC-Calibrator 1 kit (REF V-PH51) or OC-Calibrator 2 kit (REF V-PH52).

## PROCEDURE

Dispense an appropriate amount of the product into a sample cup and then operate the automatic analyzer according to its instructions. Minimum requirement is to analyze the OC-Control LV1 & LV2 before sample analysis on a daily basis. Analyze OC-Control LV3 as required.

[OC-SENSOR DIANA, OC-SENSOR PLEDIA, OC-SENSOR Micro, OC-SENSOR io]

Add OC-Control LV1, LV2 & LV3 into sample cups, then test OC-Control LV1, LV2 & LV3 by the analyzer. For one measurement, total 150 µL (3 drops) of OC-Control LV1/LV2/LV3 is required, where dead volume of sample cup is 100 µL. For duplication test, the amount of OC-Control LV1/LV2/LV3 is 200 µL (4 drops). One drop from the OC-Control LV1/LV2/LV3 vial is approximately 50 µL in volume.

## PRODUCT CODE, PRODUCT NAME & STORAGE

Product code	Product name	Contents	Storage
V-PH53	OC-Control LV1	2×5mL	2-8°C
V-PH54	OC-Control LV2	2×5mL	2-8°C
V-PH59	OC-Control LV3	2×5mL	2-8°C

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Advena Ltd.
EIKEN CHEMICAL CO., LTD.