



# Instructions for Use

380292-C

English

For Veterinary Research Use Only

## VET-SAA-QC-Low

**REF** V-SZ91  
**REF** G-SZ94

### INTENDED USE

The product is intended for use as an internal quality control fluid in laboratories making use of automated analyzers to measure Serum Amyloid A (SAA) for animal.

### REAGENTS

Reagents are stable until the date printed on the label assuming the container remains unopened at a storage temperature of 2-8 °C.

### WARNINGS AND PRECAUTIONS

- For veterinary research use only.
- Do not freeze product.
- Do not use expired product.
- Do not dissolve with other than the dedicated Reconstitution buffer.
- Air bubbles on the surface of the product after being poured into sample cups can result in error measurement, and should be removed.
- No human-derived ingredients are contained in this product, however, the risk of infection and the possible existence of other pathogens cannot be ruled out completely. Handle it as carefully as an animal sample.
- The product should be used with VET-SAA Calibrator Set (**REF** V-SZ90 **REF** G-SZ93).
- Use the reagents as quickly as possible after reconstitution.
- Dispose of used reagents, contents and containers as infectious waste in accordance with local regulations.
- The product contains Guanidinium chloride (CAS No.50-01-1, EC No.200-002-3) and ProClin300 (Mixture of 5-Chloro-2-methyl-4-isothiazolin-3-one and 2-Methyl-2H-isothiazol-3-one: CAS No.55965-84-9, EC No.911-418-6). For details, see the box label.

### PROCEDURE

- Gently remove the rubber stopper from the vial. Add 1.0 mL of Reconstitution buffer that brought back to a room temperature with a pipette.
- Seal the vial with the rubber stopper, and mix by gently inverting the vial several times. Reconstitute the control at least 15 minutes before use.
- Dispense an appropriate quantity of the reconstituted control into a sample cup and then operate the automatic analyzer by following to its instructions.

### \* PRODUCT CODE, PRODUCT NAME & STORAGE

The product is stable until the expiration date on the label.

Product code	Product name	Contents	Storage
V-SZ91, G-SZ94	VET-SAA -QC-Low	Control : 5 x 1 mL Reconstitution buffer : 1 x 12 mL	2-8 °C

### CONCENTRATION

Range of quality control is stated on the vial label.

### \* EXPLANATION OF SYMBOLS

<b>LOT</b>	Batch code		Manufacturer
	Use by date		Temperature limitation
<b>REF</b>	Catalog number		Consult instructions for use

**EIKEN CHEMICAL CO.,LTD.**  
 \*4-6 Kanda Surugadai, Chiyoda-ku, Tokyo,  
 101-0062 JAPAN  
<https://www.eiken.co.jp/en/>

EIKEN CHEMICAL CO., LTD. Tel +81-280-56-2822

\* Date of revision: February 01, 2026  
Date of issue: February 01, 2024