Instructions for Use

380317-A

English

OC-FCa Calibrator

REF V-PH12

INTENDED USE

This product is intended for use as a calibration agent when using automated immunochemical analysers to measure faecal calprotectin.

REAGENTS

Reagents are stable before and after opening until the date printed on the label when stored at 2-8°C.

OC-FCa Calibrator 6 × 1 mL, Ready to use.

WARNINGS AND PRECAUTIONS

1. For *in vitro* diagnostic use only.
2. Do not freeze products.
3. Do not use expired products.
4. Start analyses immediately after set the calibrators on racks to avoid evaporation.
5. Air bubbles on the surface of the product after being poured into sample cups can result in error measurement, and should be removed.
6. No human-derived ingredients are contained in this product, however, the risk of infection and the possible existence of other pathogens cannot be ruled out completely. Handle it as carefully as a patient's specimen.
7. If reagents get into eye or mouth, rinse it out with large volume of running water, and perform other required first aid. If necessary, seek medical attention.
8. Use the reagents as quickly as possible after they are opened. If they are to be stored, be sure to close the caps and store them using the prescribed method.
9. Disposal of used reagents and containers should be treated as medical waste in accordance with applicable regulations.

PROCEDURE

Create the calibration curve in the manner described by the automatic analyser manual or briefly described below.

Prepare 6 sample cups for OC-FCa Calibrators of 6 different concentration.

Pour 200µL (4 drops or more) of OC-FCa Calibrators into each sample cup. Set these sample cups on a rack according to the user manual and create calibration curve by the analyser.

Volume of 1 drop from each vial of OC-FCa Calibrator is approximately 50µL.

Creating a new calibration curve is required under the following conditions:

- When the lot of OC-FCa Reagent is changed.
- When the value of OC-FCa Control (LV1/LV2) is not within the indicated range.
- When opened OC-FCa Reagent vials are re-used after over 14 days storage at 2-10°C.
- When opened OC-FCa Reagent vials are left onboard for more than 7 days.
- After maintenance support procedures.

PRODUCT CODE, PRODUCT NAME & STORAGE

Product code	Product name	Contents	Storage
V-PH12	OC-FCa Calibrator	6 × 1 mL	2-8°C

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Advena Ltd.
EIKEN CHEMICAL CO., LTD.