# Instructions for Use

380318-A

English

**OC-FCa Control LV1**

**REF V-PH13**

**OC-FCa Control LV2**

**REF V-PH14**

## INTENDED USE

OC-FCa Control LV1/LV2 is intended for use as a liquid internal quality control in clinical laboratories making use of OC-SENSOR automated immunochemical analyser to measure faecal calprotectin.

## REAGENTS

Reagents are stable before and after opening until the date printed on the label when stored at 2-8°C.

OC-FCa Control LV1	2 × 5 mL,	Ready to use.
OC-FCa Control LV2	2 × 5 mL,	Ready to use.

## WARNINGS AND PRECAUTIONS

1. For *in vitro* diagnostic use only.
2. Do not freeze product.
3. Do not use expired product.
4. Start analyses immediately after set the controls on racks to avoid evaporation.
5. Air bubbles on the surface of the product after being poured into sample cups can result in error measurement, and should be removed.
6. No human-derived ingredients are contained in this product, however, the risk of infection and the possible existence of other pathogens cannot be ruled out completely. Handle it as carefully as a patient's specimen.
7. If reagents get into eye or mouth, rinse it out with large volume of running water, and perform other required first aid. If necessary, seek medical attention.
8. Use the reagents as quickly as possible after they are opened. If they are to be stored, be sure to close the caps and store them using the prescribed method.
9. Disposal of used reagents and containers should be treated as medical waste in accordance with applicable regulations.
10. The calprotectin concentration is displayed on the label.

## PROCEDURE

Dispense an appropriate amount of the product into a sample cup and then operate the automatic analyser according to its instructions. Minimum requirement is to analyse the OC-FCa Control LV1 and LV2 before sample analysis on a daily basis.

Add OC-FCa Control LV1 and LV2 into sample cups, then test OC-FCa Control LV1 and LV2 by the analyser. For one measurement, total 150 µL (3 drops) of OC-FCa Control LV1/LV2 is required, where dead volume of sample cup is 100 µL. For duplication test, the amount of OC-FCa Control LV1/LV2 is 200 µL (4 drops). One drop from the OC-FCa Control LV1/LV2 vial is approximately 50 µL in volume.

## PRODUCT CODE, PRODUCT NAME & STORAGE

Product code	Product name	Contents	Storage
V-PH13	OC-FCa Control LV1	2×5mL	2-8°C
V-PH14	OC-FCa Control LV2	2×5mL	2-8°C

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**Advena Ltd.**
**EIKEN CHEMICAL CO., LTD.**