
Ceres

Operation Manual

NN1-1741

REF MV5K00

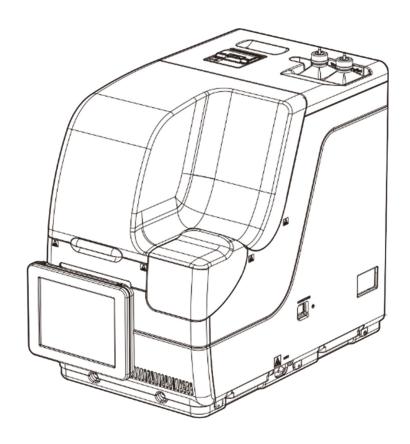

Be sure to read this Operation Manual and all the Safety Precautions carefully for safe usage of this product.

Store this Operation Manual in a place so that it can be referred to when necessary.

Manufacturer: EIKEN CHEMICAL CO., LTD.

4-19-9 TAITO, TAITO-KU, TOKYO 110-8408 JAPAN

Authorized Representative : Advena Ltd.

Tower Business Centre 2F Tower Street, Swatar BKR 4013 Malta

Important

- Copyright for this Operation Manual belongs to Eiken Chemical Co., Ltd. and Hitachi, Ltd. Use, reprinting, duplication, and modification of its contents in part or in whole without permission is strictly prohibited.
- The content of this Operation Manual and system specifications may be changed without notice in order to make improvements.
- We are not responsible for any damages or the like if the system is not used in accordance with this Operation Manual.
- In vitro Diagnostic Medical Device Regulation 2017/746

• EN 61010-1 : Safety requirements for electrical equipment for measurement,

control, and laboratory use Part 1: General requirements.

• IEC 61010-2-101 : Safety requirements for electrical equipment for measurement,

control, and laboratory use - Part 2-101: Particular requirements for in

vitro diagnostic (IVD) medical equipment.

• EN 61326-1 : Electrical equipment for measurement, control, and laboratory use.

EMC requirements. General requirements.

• EN 61326-2-6 : Electrical equipment for measurement, control, and laboratory use.

EMC requirements. Particular requirements. In vitro diagnostic (IVD)

medical equipment.

Introduction

Prior to reading this Operation Manual

OC-SENSOR Ceres is a mobile discrete method fully automated fecal occult blood analyzer (hereinafter referred to as the "system").

The system and its Operation Manual are meant for doctors, clinical laboratory technicians, and those who have received specialized education or training in test procedures that use diagnosis systems outside the body.

Read this Operation Manual carefully prior to use, in order to use the system properly.

Note that using the system in a manner not described in this Operation Manual or under conditions outside the system specifications could negatively impact the system's safety and performance.

Be sure to use the system according to the instructions in this Operation Manual.

All serious accidents occurred in connection with the system need to be reported to regulating authorities of the countries in which the manufacturer, users, and/or patients live.

Operation Manual organization

This Operation Manual consists of the following chapters:

Introduction : Describes the organization and notation of this Operation

Manual, as well as Safety Precautions for using the system.

Chapter 1 Overview : Gives an overview of the system, including measurement

principles and analysis flow, as well as the names and

functions of each part.

Chapter 2 Basic Operation : Gives an overview of initial settings prior to use and basic

operation of the system.

Chapter 3 Applied : Describes advanced operation of the system, such as

Operations searching, recalculation, output, deletion, and accuracy

control of the test data.

Chapter 4 Prep Functions : Describes preparatory functions such as priming.

Chapter 5 Maintenance : Describes inspections and maintenance procedures users

should follow in order to safely use the system, maintain performance, and discover malfunctions and the like as early

as possible.

Chapter 6 Settings : Gives details on initial settings for operating the system.

Chapter 7 Error Handling : Explains how to read the error screen.

Appendix : Explains data processing, analysis operations, printing

examples, and errors.


Index/Glossary

Introduction

Operation Manual Notation

This Operation Manual explains matters that must be followed in order to safely use the system, prevent danger to the user and others, and prevent damage to property.

"Warnings," "Cautions," and "Requests"

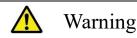

Indicates the possibility that improper use of the system will cause death or severe personal injury.



Caution

Indicates the possibility that improper use of the system will cause personal injury or damage to property.



Request

Indicates something the user should do to prevent damage or degradation to the product and to use the system efficiently.

Severe injuries:

Injuries that result in permanent damage or which require long-term hospitalization or outpatient treatment. Examples include loss of eyesight, injury, burns, electric shock, fractures, poisoning, and infection.

Slight injuries:

Injuries that do not require long-term hospitalization or outpatient treatment.

■ Symbols

Caution symbols: Indicate that users should use caution.









Attention

Biohazard

Electrical shock Pinching hazard

Prohibition symbols: Indicate prohibited actions.







Disassembly prohibited

Instructional symbol: Indicate instructions users must follow.



Required

■ Other notations

Note : Describe related supplementary notes.

}: Indicate something that can be operated, such as a tab or a button on the screen.

[]: Indicate the name of a screen, or settings that require selection or input.

: Indicate reference locations.

Safety Precautions

Be sure to read this section before using the system.

Installation precautions

Power supplies and cords



Warning



Prohibited

- Do not connect the power cord to an extension cord or an adaptor plug.
- Do not connect or disconnect the power plug with wet hands.
- Do not damage or alter the power cord.
- Do not apply excessive force to the power cord.
- Do not fasten the power cord using metal fixtures or the like.
- Do not use power cords other than that provided.

Failure to observe this precaution may lead to electric shock or fire.



Warning



• Make sure that the system is grounded.

Failure to observe this precaution may lead to electric shock.





Required

• Connect to the appropriate power supply.

Power supply voltage: $230~\mathrm{V}~\mathrm{AC}$

Frequency: 50/60 Hz

Power consumption: 630 VA or lower

Socket: The power plug uses a protective earth terminal.

Use a fixed power socket (electrical outlet for medical use) that is correctly

grounded.

Failure to observe this precaution may lead to electric shock or fire.

Safety Precautions

Installation conditions



Caution



Required

- Use indoors.
- Install and store at locations not exposed to water.
- Avoid harmful effects that can result from barometric pressure, atmospheric temperature, humidity, poor ventilation, sunlight, dust, saline matter, or air including sulfur.
- Install in a location that is flat and free of vibration or impact.
- Install in a location other than a chemical storage area or in a location free from gas emissions.
- The installation location must be horizontal.



Caution



Required

• Do not move the system.

This system should only be installed or moved by a maintenance person who has been certified by Eiken Chemical.


• Use a platform that can support the weight of the system. Make sure that the platform is stable.

If the system falls, it could cause unexpected injury.

• Install the system in a location with plenty of space, so as to not hinder operation or maintenance.

(The maximum height when the safety guard is opened is approx. 90 cm. Therefore, the height from the floor to the ceiling must be 100 cm or higher.)

(See the following drawing for the installation spacing.)

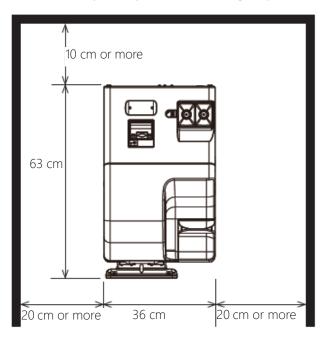

• Do not place the system near the exhaust ports of heating and cooling equipment.

Failure to observe this precaution may lead to erroneous analysis result.

Safety Precautions

Environmental conditions



Caution



Required

• Adhere to the following environmental conditions:

Usage conditions Temperature: 15 - 30°C

(Within ± 2 °C change during testing) Altitude: 20 - 80% (no condensation)

Height of 2,000 m or less *Pollution degree 2* Over-voltage category II*

Storage conditions Temperature: -10 - 50 °C

Humidity: 10 - 90 % (no condensation)

Transport conditions Temperature: -10 - 50 °C

Humidity: 10 - 90 % (no condensation)

Note

Items marked with "*" are standard conditions based on EN 61010-1

Security



Caution



Required

Observe the following to prevent leakage or falsification of data.

- To avoid unauthorized access, restrict the operation of the system to authorized personnel only.
- Make sure that only verified computers are connected to the system network.
- Conduct cybersecurity assessments regularly to ensure an appropriate level of data security.
- Do not use personally identifiable information for sample IDs and patient IDs. Instead, use anonymized IDs for management.
- For network security, address the security of IT infrastructure in your facility to ensure that IT security is not compromised by malicious software or hacker attacks.
 - Protect all devices and services used in your facility from malicious software and unauthorized access.
 - Connect the system only to the internal network of your facility that is not connected to the external network.
- Regularly protect your data by storing it on external media. There is a possibility of data corruption due to cyber attacks, disasters, etc.
- Please review your users and passwords regularly.
- Check the USB flash drive in advance for viruses and make sure that no viruses have been detected.

Failure to observe this precaution may lead to infection by computer viruses.

Failure to observe this precaution may lead to leakage or falsification of information.

Usage precautions

Work and operation precautions



Warning



- Wear protective gear when handling the samples, reagents, washing solution, and test cells.
- After using the system, wash hands thoroughly.

Biohazard Failure to observe this precaution may lead to infection from samples.



Biohazard

- Dispose of drainage after using the system.
- Make sure that drainage does not leak into the area surrounding the device when detaching the drain tank hose.
- Make sure the drain tank is empty prior to using the system.

Failure to observe this precaution may lead to infection from samples.



Biohazard

- Be careful of sample splattering when removing sampling bottles from a rack
- Be careful of sample splattering when disposing of measurement cells.

Failure to observe this precaution may lead to infection from sample.



Disassembly prohibited

- Do not disassemble the system.
- Do not remove any of the system's exterior parts.

Failure to observe this precaution may lead to electric shock.



Electrical shock



Prohibited

• Do not spill any samples or reagents inside the system.

• Do not mix the washing solution with acid washing solution.

• Do not touch the system with wet hands.

Failure to observe this precaution may lead to electric shock.



Failure to observe this precaution may damage the operator's health.

Prohibited

Safety Precautions

Caution



Required

- Correctly connect the purified water/washing solution bottle, drain tank, and each hose.
- Periodically monitor the accuracy control function.

Failure to observe this precaution may lead to false diagnoses.



Caution



Required

• Do not connect to any device other than a USB stick.

For details of recommended items, contact the manufacturer.

• Manage external media appropriately. Check for computer viruses periodically.

Failure to observe this precaution may lead to infection from a computer virus.



Caution



• Use only reagents that have not expired.

Failure to observe this precaution may lead to false diagnoses.

Required



Caution



• If the reagent refrigerator becomes abnormally hot, dispose of the stored reagents.

Failure to observe this precaution may lead to false diagnoses.





Caution



- Do not turn off the power while testing.
- Do not turn off the primary power switch until the system shut downs completely.

Required

Failure to observe this precaution may lead to a damaged hard disk or loss of data.



Caution



- Use the designated sample cup.
- Do not reuse measurement cells.

Failure to observe this precaution may lead to false diagnoses.

Required



Caution



• Do not touch the touch panel with wet hands. Failure to observe this precaution may lead to system breakdown.

Required

Safety Precautions

Maintenance and inspections



Warning



Biohazard

- Wear protective gear when performing maintenance and inspections.
- The tip of the nozzle is sharp. Use caution when handling it.
- The tip of the puncture needle is sharp. Use caution when handling.

Failure to observe these precautions may lead to infection from the samples.



Request

- Inspect the system before starting operations each time.
 - Check that there is no water leakage.
 - No devices other than the designated ones should be connected.
 - The environmental conditions are satisfied.



Request

• If the system has not been used for some time, check that the system operates properly prior to use.



Request

• If system failure is suspected, do not touch the system, and do not attach a failure notice or other notification. Immediately contact the manufacturer or a legal representative.

Disposing of drainage and waste



🔼 Warning



Biohazard

• Dispose of drainage and waste (reagent containers, reaction containers, sampling bottles, sample cups, and measurement cells) appropriately by following the facility safety management procedures and the instructions of the person in charge of infectious medical waste management.

(Some examples of infectious medical waste include reagent containers, reaction containers, sampling bottles, sample cups, and measurement cells.)


- Refer to the usage instructions attached to the reagent for information on how to dispose the reagent containers and drainage.
- Contact the manufacturer or your legal representative when disposing the system.
- Wear protective tools when disposing.

Failure to observe this precaution may lead to environmental pollution.


Caution labels and where they are affixed

The following caution labels are affixed to the system.

Understand where the caution labels are and what they mean before using the system.

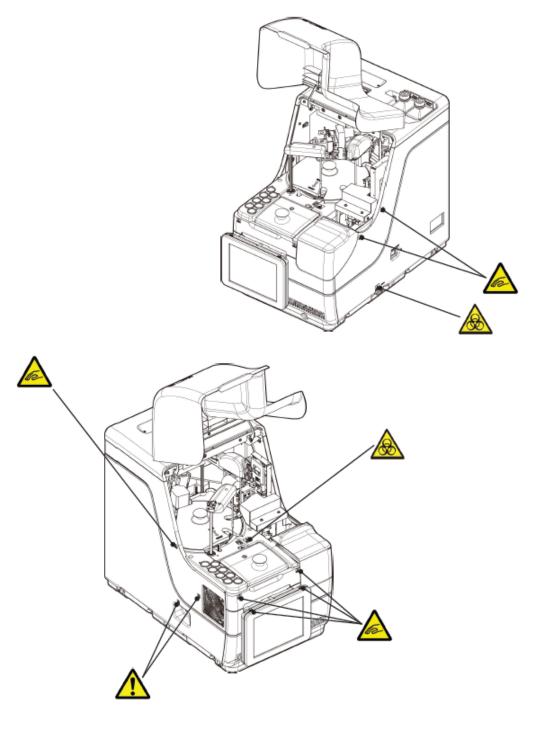


Safety Precautions

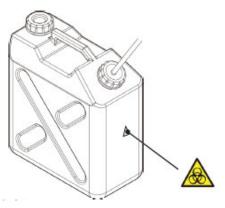

Drainage tank

Caution Label	Meaning
	Keep out of operational range during system operation. Failure to observe this precaution may lead to injury.
	1
	Do not handle drainage with bare hands.
	Failure to observe this precaution may lead to infection and pollution.
	(Fan)
^	Do not touch the fan while it is operating.
	Failure to observe this precaution may lead to injury.
	(Connecting port)
	Connect only permitted device.
	When any device other than those permitted is connected, the system may experience
	erroneous operation.

Primary power switch and system switch

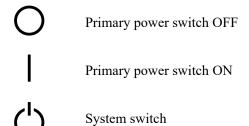

Identification plate

Identification Label (DRAFT)

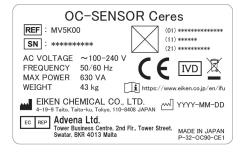


Table of Contents

ntroduction	
Prior to reading this Operation Manual ·····	
Operation Manual organization · · · · · · · · · · · · · · · · · · ·	
Operation Manual Notation	·····ii
afety Precautions	
Installation precautions	iv
Power supplies and cords · · · · · · · · · · · · · · · · · · ·	iv
Installation conditions ·····	7
Environmental conditions	v
Security	v
Usage precautions	····· vi
Work and operation precautions ·····	····· vi
Maintenance and inspections	ix
Disposing of drainage and waste····	ix
Caution labels and where they are affixed	у
Table of Contents	
hapter 1 Overview	
1.1 Features of OC-SENSOR Ceres ····	2
1.2 Application	2
1.3 Measurement Principles · · · · · · · · · · · · · · · · · · ·	2
1.4 System Configuration	
1.5 Test Flow·	5
1.6 System Specification	······································
1.7 Reagents Used by the System····	{
1.8 System Dimensions	
1.9 Names and Functions of Parts ····	10
1.9.1 Exterior Parts·····	10
1.9.2 Internal Parts · · · · · · · · · · · · · · · · · · ·	12
1.9.3 Accessories and Others ·····	12

Chapter 2 Basic Operation	
2.1 Basic Screen Operation · · · · · · · · · · · · · · · · · · ·	16
2.1.1 [MENU] screen and functions ·····	16
2.1.2 Screen Configuration and Button Operations ·····	17
2.1.3 [Analyzing] Screen and Functions ·····	23
2.1.4 [Set reagent] Screen and Functions ·····	25
2.2 Starting the System · · · · · · · · · · · · · · · · · · ·	27
2.3 Logging In · · · · · · · · · · · · · · · · · ·	28
2.4 Initial Settings ·····	31
2.4.1 System Settings ·····	31
2.4.2 Protocol Settings · · · · · · · · · · · · · · · · · · ·	33
2.5 Daily Operation · · · · · · · · · · · · · · · · · · ·	35
2.5.1 Flow of Daily Operations · · · · · · · · · · · · · · · · · · ·	35
2.5.2 Setting the Reagent	37
2.5.3 Preparing Purified Water ·····	41
2.5.4 Preparing Washing Solution ·····	42
2.5.5 Washing Solution and Purified Water Settings ·····	43
2.5.6 Setting Printer Sheets · · · · · · · · · · · · · · · · · · ·	44
2.5.7 Checking the Drain Tank·····	46
2.5.8 Creating CC ····	47
2.5.9 Setting Samples · · · · · · · · · · · · · · · · · · ·	58
2.5.10 Starting Analysis (Initial Test) · · · · · · · · · · · · · · · · · · ·	60
2.5.11 Start Analyzing (Retest)·····	62
2.5.12 Starting Analysis (QC Sample) ·····	65
2.5.13 Additionally Setting the Sample ·····	70
2.5.14 Checking the Analysis Information · · · · · · · · · · · · · · · · · · ·	72
2.5.15 Ending the Analysis · · · · · · · · · · · · · · · · · ·	75
2.6 Inspections and Cleaning After Use·····	77
2.6.1 Removing the Racks ·····	77
2.6.2 Replacing Cells · · · · · · · · · · · · · · · · · ·	··· 78
2.6.3 Cleaning Various Sections ·····	84
2.6.4 Opening Parts Checklist · · · · · · · · · · · · · · · · · · ·	86
2.6.5 Opening the Error Log·····	88
2.7 Shutting Down the System (Shut Down Mode)·····	89
2.8 Processing the Drain Tank·····	91

Chapter 3 Applied Operations

3.1 Processing Test Data · · · · · · · · · · · · · · · · · ·	
3.1.1 Opening the [Process Data] Screen ·····	
3.1.2 Opening the [Test Data] Screen ·····	
3.1.3 Specifying a {Date} for the Test Data·····	
3.1.4 Selecting {Select-s} for Test Data ·····	
3.1.5 Reading "Test Data" Not Displayed in the List (Re-reading)	104
3.1.6 Reading "Test Data" in External Media (Switching the External Media)	105
3.1.7 Searching the Test Data····	106
3.1.8 Outputting Test Data · · · · · · · · · · · · · · · · · ·	
3.2 Editing Sample IDs····	
3.3 Replication····	114
3.3.1 Displaying the [Sample Replication] Screen (Sample) ······	114
3.3.2 Displaying [Replicate (STD)] Screen ·····	
3.3.3 Saving Replication Data (STD) to External Media·····	120
3.3.4 Displaying the [Replicate (QC)] Screen ·····	
3.3.5 Specifying {Select-s} for Replication Data·····	123
3.3.6 Specifying Replication Data by {Select Item] ·····	
3.3.7 Searching Replication Data · · · · · · · · · · · · · · · · · ·	
3.3.8 Editing and Recalculating CC ·····	
3.3.9 Displaying and Printing a Time Course (Sample) · · · · · · · · · · · · · · · · · · ·	
3.3.10 Displaying and Printing a Time Course (STD) · · · · · · · · · · · · · · · · · · ·	
3.3.11 Changing the Time Course Range (Sample) · · · · · · · · · · · · · · · · · · ·	133
3.3.12 Changing the Time Course Range (STD) · · · · · · · · · · · · · · · · · · ·	135
3.4 Accuracy Control · · · · · · · · · · · · · · · · · · ·	137
3.4.1 Displaying the QC Lot List ([QC lot select] screen) ······	137
3.4.2 Selecting a QC Lot ·····	140
3.4.3 Deleting a QC Lot ·····	141
3.4.4 Opening the [Intra-day/Inter-day] Screen ·····	142
3.4.5 Editing (Recalculating) Intra-Day and Inter-Day Data	145
3.4.6 Deleting Intra-Day and Inter-Day Data · · · · · · · · · · · · · · · · · ·	147
3.4.7 Selecting Intra-Day and Inter-Day Data with {Select-s} ······	149
3.4.8 Displaying X-R Control ······	151
3.4.9 Changing the Range of \bar{X} -R Control · · · · · · · · · · · · · · · · · · ·	155
3.5 LOG IN/LOG OUT ·····	157
3.5.1 LOGIN · · · · · · · · · · · · · · · · · · ·	157
3.5.2 LOGGING OUT ·····	158
3.5.3 Changing a Password ·····	159

1 Overview
2 Basic Operation
3 Applied Operations
4 Prep Functions
5 Maintenance
6 Settings

7 Error Handling

Chapter 4 Prep Functions	
4.1 Initialization ·····	162
4.2 Priming ·····	164
4.3 Washing ·····	166
Chapter 5 Maintenance	
5.1 Inspection and Maintenance	170
5.1.1 Opening the [Maintenance] Screen ·····	170
5.1.2 Cleaning Touch Panel (Daily) ·····	171
5.1.3 Cleaning the Rack Lane (Daily)·····	172
5.1.4 Cleaning Washing Solution and Purified Water Compartments (Daily) · · · · · · · · · · · · · · · · · · ·	173
5.1.5 Cleaning the Reagent Cap Tray (Daily) ·····	174
5.1.6 Cleaning the Nozzles (Weekly) ·····	175
5.1.7 Cleaning the Racks (Weekly)·····	176
5.1.8 Cleaning Bottles/Tanks (Monthly)·····	177
5.2 List of Parts to Check and Replace	178
5.2.1 Opening the [Parts checklist] Screen ·····	178
5.2.2 Replacing Parts ·····	179
Chapter 6 Settings	
6.1 System Settings ·····	
6.1.1 Sample Barcode Settings (Common) ·····	182
6.1.2 Sample Barcode Settings (Detailed Settings for Each Barcode Type) · · · · · · · · · · · · · · · · · · ·	184
6.1.3 Rack Setting·····	188
6.1.4 Config. · · · · · · · · · · · · · · · · · · ·	190
6.1.5 Data Output - [Selecting Data Output Destination] · · · · · · · · · · · · · · · · · · ·	192
6.1.6 Data output - [Online Settings]·····	195
6.1.7 Data Output - [Test Setting]·····	198
6.1.8 Output Format Settings (Basic Format) ······	200
6.1.9 Output Format Setting (External Media)·····	203
6.1.10 User Account Settings ·····	206
6.1.11 Language Settings ······	210
6.1.12 Operator Mode ·····	212
6.1.13 Printing the System Settings ·····	214
6.1.14 Backing Up System Settings and Protocol Settings ·····	215
6.1.15 Restoring System Settings and Protocol Settings ······	216

Table of Contents

6.2 Protocol Settings ·····	217
6.2.1 Sample/QC Protocol Settings·····	217
6.2.2 CC Protocol Settings·····	224
Chapter7 Error Handling	
7.1 How to Read the [FUNCTION ERROR] Screen ·····	228
7.2 Error Handling Buttons ·····	229
7.3 Cancelling Errors · · · · · · · · · · · · · · · · · · ·	230

Appendix

1 Calculation Processes	
1.1 STD/QC Sample Measured Data Check ·····	
1.2 DA Value Calculation · · · · · · · · · · · · · · · · · · ·	
1.3 Measured Data (Concentration) Calculation and Qualitative Assessment······	237
1.4 Prozone Check · · · · · · · · · · · · · · · · · · ·	
1.5 Reagent Blank Check · · · · · · · · · · · · · · · · · · ·	240
1.6 Calculate Cell Blank · · · · · · · · · · · · · · · · · · ·	240
2 Printing Example	
2.1 Positive Sample Printing · · · · · · · · · · · · · · · · · · ·	
2.2 Final Result Printing at Retesting ·····	
2.3 Printing when STD Sample and QC Sample are Measured · · · · · · · · · · · · · · · · · · ·	
2.4 Printing Error Messages ·····	247
3 Error List	
3.1 ERR# 0-1001 to 0-3005 (Main) · · · · · · · · · · · · · · · · · · ·	
3.2 ERR# 1-001 - 1-200(SS1)·····	
3.3 ERR# 2-001 - 2-200(SS2) ·····	264
4 Saving to External Media	
4.1 Sample Measured Data Information · · · · · · · · · · · · · · · · · · ·	
4.2 Sample Replication Information	
4.3 QC Measured Data Information · · · · · · · · · · · · · · · · · · ·	275
4.4 QC Replication Information	277
4.5 STD Measured Data Information	279
4.6 STD Replication Information	280
4.7 Time Course Data Information · · · · · · · · · · · · · · · · · · ·	281
4.8 List of Error Codes for External Media Output ·····	282
5 Management USB Stick Setting	
5.1 How to Set a Management USB Stick·····	283
Index/Glossary	
Index ····	285
Glossary ·····	292

1 Overview

2 Basic Operation

3 Applied Operations

4
Prep
Functions

5 Maintenance

 $\underset{\text{Settings}}{6}$

Error Handling

Chapter 1 Overview

- 1.1 Features of OC-SENSOR Ceres
- 1.2 Application
- 1.3 Measurement Principles
- 1.4 System Configuration
- 1.5 Test Flow
- 1.6 System Specification
- 1.7 Reagents Used by the System
- 1.8 System Dimensions
- 1.9 Names and Functions of Parts



Chapter 1 Overview

This section describes the general outline and the configuration of the system, "Mobile discrete method Fully automated fecal occult blood analyzer (fecal occult blood test) OC-SENSOR Ceres"

1.1 Features of OC-SENSOR Ceres

- The OC-SENSOR Ceres (hereinafter called "the system") features automatic sample dilution, a wide test range, countermeasures against carryover, prozone judgment, and more.
- The system is always cooling the reagents. Therefore, samples can be tested anytime.

1.2 Application

The system measures the objective material in the sample or hemoglobin in feces by detecting changes in transmitted light by latex agglutination reaction.

1.3 Measurement Principles

Latex agglutination turbidimetry

An antigen-antibody reaction is a specific reaction that occurs between an antigenic determinant and the active group of an antibody. The amount of bonding depends on the concentrations of the antigen and antibody.

A latex agglutination reaction is the clumping of antigen- or antibody-sensitized polystyrene latex particles caused by an antigen-antibody reaction. Light is passed through the reaction liquid to measure changes in the intensity of the transmitted light beam. This method, used by this system, is called "latex turbidimetry."

1.4 System Configuration

Before using the system, check the configuration.

Name		Q'ty	Remarks
Main body	Mobile discrete method Fully automated fecal occult blood analyzer (OC-SENSOR Ceres)	1 set	
Accessories	① Software package		
	· Software program	1 set	Installed on the hard disk
	② Racks		
	· Sample rack	1 box	2 pc/box
	· STD and QC rack	1 box	1 pc/box
	③ Tanks and bottles		
	· Purified water bottle (Purified Water)	1	500 mL bottle
	 Washing solution bottle (Wash Solution) 	1	500 mL bottle
	· Drain tank	1	5L tank
	· Beaker	1	
	· Replacement bottle for purified water	1	500 mL bottle
	④ Others		
	· Power supply cable	1	
	· Barcode labels for racks	3 pc	"01-10," "11-20," "91-100"
	· Bottle and tank labels	1 set	Already attached to the bottles and
			tanks
	· Sample cup	1 bag	
	· Thermal roll paper	1 roll	W: 58mm / D: 40mm
	Binding band	5 pc	
Documents	· Operation Manual	1	

1.4 System Configuration

1 Overview

■ Accessories (sold separately)


Name	Product code	Quantity	Remarks
DISPO-11	M-5K12	1 box	55 x 20 pc

Option

Name	Quantity	Remarks
Handy barcode reader	1 pc/box	Used to read the calibrator and QC barcodes.
2D code reader replacement kit	1 pc/box	
340 nm detection kit	1 pc/box	Used when developing a new item
600 nm detection kit	1 pc/box	Used when developing a new item
800 nm detection kit	1 pc/box	Used when developing a new item

1.5 Test Flow

The test flow of this system is as follows

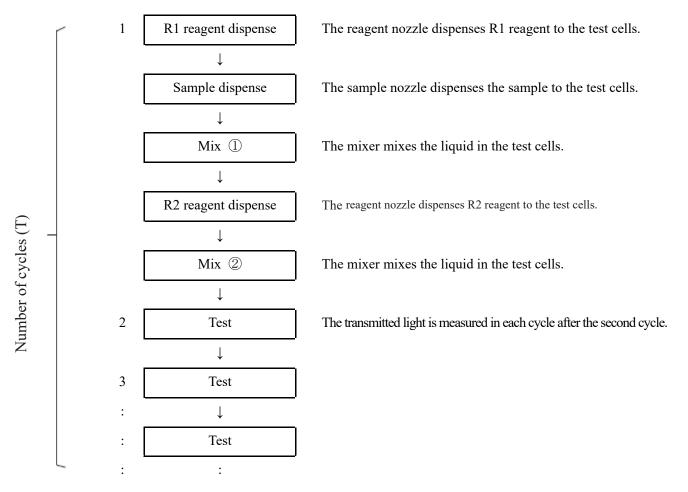

Note) The required time for one cycle is 40 seconds.

1.6 System Specification

■ Basic specifications

Name	Specification	
Measurement principles	Latex turbidimetry method	
Method	Discrete method, random access method (max. 3 items)	
Test mode	1-step rate	
Samples	Feces	
Processing capacity	Up to 90 tests/hour (40 seconds per cycle)	
Sample setting	20 samples:10 samples x 2 racks	
Sample container	OC-Auto Sampling Bottle 3 (Eiken Chemical proprietary bottle)	
	Sample cup (A20)	
Calibration curves	Automatic generation of calibration curves	
Retest	Automatic re-test function	
	Dilution re-test function (x10, x20, x100, x200, x400)	
Reaction cuvette	Five disposable cuvettes (5 x 11 reaction cells)	
Sample dispensing	Sampling nozzle with liquid level detection sensor and nozzle cleaning	
	function	
Sample dispensing range	0, 3-21μL (0.1μL unit)	
	* For protocol settings, please set dispensing volume specified by the manufacturer.	
Reagent dispensing	Sampling nozzle with liquid level detection sensor and nozzle cleaning	
	function	
Reagent dispensing range	0, 30-210μL (1μL unit)	
	* For protocol settings, please set dispensing volume specified by the manufacturer.	
Mixing	Mixer rotating system (with cleaning function)	
Barcodes	Rack barcodes/Sample barcodes and Reagent barcodes	
Prozone check	PRC method, RBC method, OR method	
Check reagent blank	Detection using A1 value check, DA1 value check	
Reagent loading capacity	3reagent sets (R1, R2) and 2 sample diluent buffers	
	Hemoglobin, Calprotectin	
Cooling system	Peltier cooling system surrounding reagent storage area (working for	
	24 hours)	
Thermal insulation system	Reaction table, Silicone rubber heating System	
Light source	LED (wavelengths: 660 nm, 340 nm,* 600 nm,* 800 nm*)	
	*: Optional wavelengths.	
Detector	Photodiode	
Operation control/data	Multiple CPU configuration with internal network	
processing		
External connections	RS-232C, Ethernet	
Sample dispensing accuracy	CV0.5% or less at 10 μL	
Reagent dispensing accuracy	CV0.5% or less at 30 µL	
Security	Whitelist	

Name	Specification
Inputs	Color LCD (8.4 inch) touch panel, barcode reader
Outputs	Thermal printer (58 mm wide thermal paper)
	Integrated hard disk
	External media USB connection
Dimensions	Approx. 360 mm x 625 mm x 545 mm (W x D x H)
Weight	Approx. 43 kg
Power	~100-240 V 50/60 Hz 630VA

Note The dimensional tolerance is +/- 10%, and weight tolerance is +/- 10%.

■ Sample barcode specification

Туре	Number of digits	Check digit	Remarks
NW-7	5- 17 (including start and stop characters)	Modulus 10/3 weight Modulus 16 Modulus 11 Modulus 10/2 weight 7 check DR Weighing modulus 11 Loons	
ITF	6- 15	Modulus 10/3 weight	
IND 2 of 5	6- 15	None	
COOP 2 of 5	6- 15	None	
CODE39	5- 15 (Including start and stop characters)	Modulus 43	
JAN	5- 15	Modulus 10/3 weight	
CODE128	5- 15	None	
QR	6- 50	None	
Data Matrix	6- 50	None	2D code reader
PDF417	6- 50	None	(optional)
GS1 DataBar Omni-directional	6- 50	None	

Usable life

Eight years (self-certified [based on manufacturer data]) from start of use (installation)

^{*} Assuming periodic inspection/maintenance, parts replacement, and repairs/overhaul (as required after inspection) are conducted as described in this Operation Manual.

1.7 Reagents Used by the System

Refer to the attached documents for reagents used in this system.

Reagents for hemoglobin testing

Product name	Product code	Package
OC-SENSOR FIT (for OC-SENSOR Ceres)	V-PH01	$2 \times 6 \text{ mL}$ $2 \times 20 \text{ mL}$
OC-FIT Calibrator (for OC-SENSOR Ceres)	V-PH02	6 × 1 mL (concentration in 6 levels)
OC-FIT Control LV1	V-PH53	2 × 5 mL
OC-FIT Control LV2	V-PH54	2 × 5 mL
OC-FIT Control LV3	V-PH59	2 × 5 mL


■ Common


Product name	Product code	Package
OC-SENSOR Sample Diluent	V-PH08	2 × 20 mL

■ Reagents for calprotectin testing

Product name	Product code	Package
OC-FCa Reagent (for OC-SENSOR Ceres)	V-PH09	2 × 8 mL
OC-PCa Reagent (101 OC-SENSOR Ceres)		2 × 15mL
OC-FCa Calibrator	V-PH12	6 × 1 mL (concentration in 6 levels)
OC-FCa Control LV1	V-PH13	2 × 5 mL
OC-FCa Control LV2	V-PH14	2 × 5 mL
OC-FCa Control LV3	V-PH15	2 × 5 mL

1.8 System Dimensions

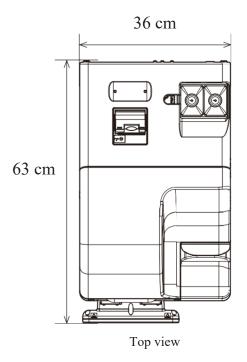


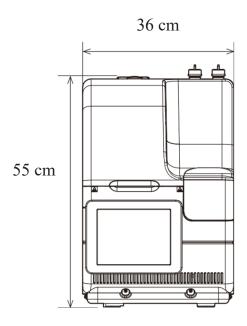


Front view

1.9 Names and Functions of Parts


1.9.1 Exterior Parts

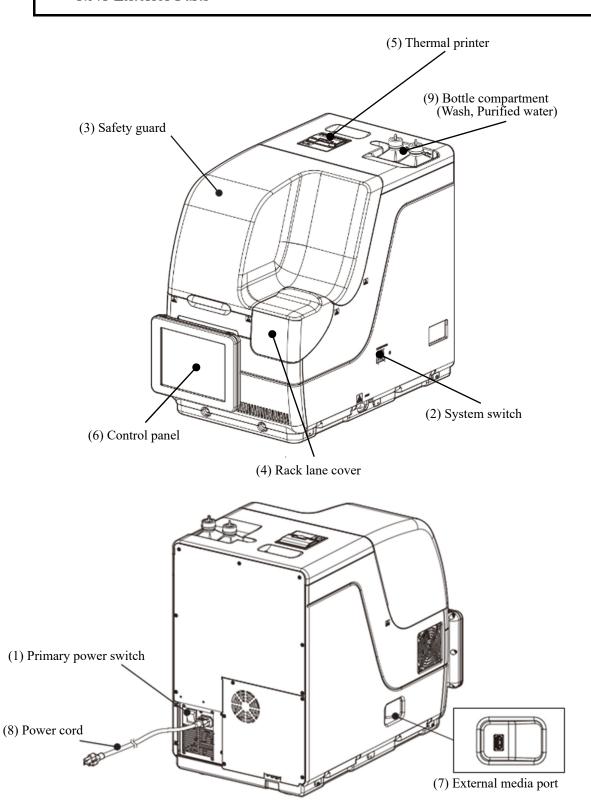


Washing solution bottle (Wash Solution)

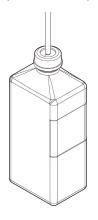

① Purified water bottle (Purified Water)

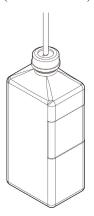


12 Drain tank (Drain)

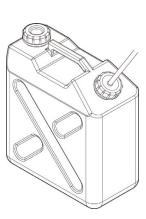

Name	Function	
① Primary power switch	When turning on the primary power switch, the system is powered on. Usually,	
© 11mmily period switch	keep the system power on.	
	Switch OFF Switch ON	
	Note After turning on the primary power switch, cold storage of the reagent starts when	
	the application starts up.	
	When turning off the primary power switch, cold storage of the reagent cooler is	
	turned off.	
② System switch	When turning on the system, the application starts, and the reaction table	
	temperature is controlled.	
③ Safety guard	The cover to prevent accidents during the system operation. It is automatically	
	locked during the system operation.	
④ Rack lane cover	The cover to prevent accidents during the system operation. It is automatically	
	locked while using the rack.	
⑤ Thermal printer	Prints setup conditions, test results, error messages, or other data.	
Control panel	Shows operation screens and analytical results. Operate by directly touching the	
(Including the touch panel)	screen.	
⑦ External media connector	The external media to save the test data is connected.	
Power supply cord	The power cord for this system.	
Washing solution and purified	Section to set the washing solution bottle and the purified water bottle.	
water setting section		
(1) Washing solution bottle	Fill with wash solution.	
(Wash Solution)		
① Purified water bottle	Fill with purified water.	
(Purified Water)		
② Waste liquid tank(Drain)	The wasted liquid is stored from the sample nozzle, reagent nozzle, and mixer.	


1.9.2 Internal Parts

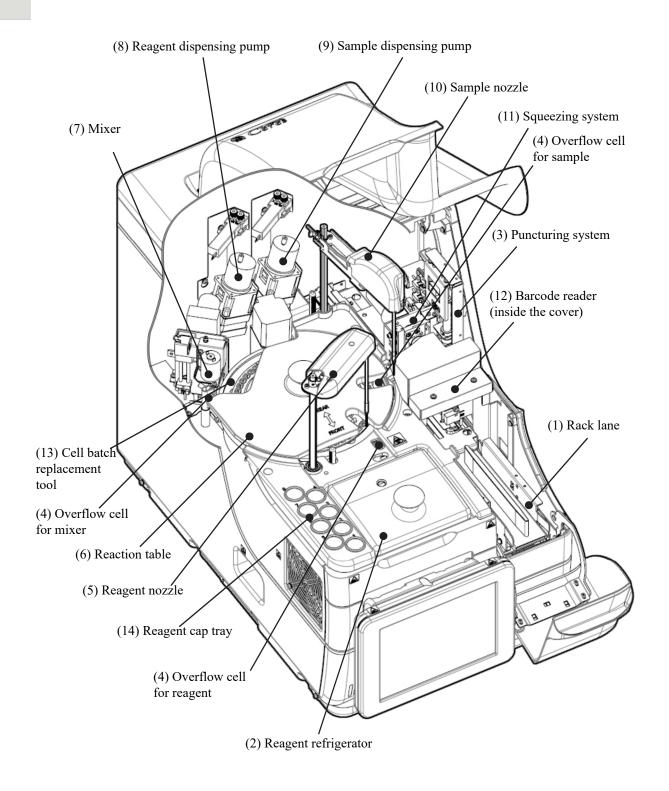


Name	Function
① Rack lane	Rack setting location. Up to two racks can be set.
② Reagent refrigerator	Reagent (20 mL special container) location. The temperature is
	always kept at 9 to 15 °C.
③ Puncturing system	Punctures the double aluminum seals on the sample bottle.
④ Overflow cell (OF)	For sample nozzle
	Washes the sample nozzle inner and outer walls with purified
	water and washing solution. Waste liquid is sent to the drain tank.
	For reagent nozzle
	Washes the reagent nozzle inner and outer walls with purified
	water. Waste liquid is sent to the drain tank. For mixer
	Washes the mixer needle with purified water and washing
	solution. Waste liquid is sent to the drain tank.
© D	
⑤ Reagent nozzle	Dispenses the reagent.
6 Reaction table	Location of the antigen-antibody reaction and photometry.
7 Mixer	Mixes the sample and the reagent.
Reagent dispensing pump	Absorbs and flushes samples.
Sample dispensing pump	Absorbs and flushes samples.
10 Sample nozzle	Dispenses samples.
① Squeezing system	Raises the sample level in the sampling bottle.
② Barcode reader	Reads barcodes on reagent bottles and racks.
Cell batch replacement tool	Used to replace test cells in a batch.
4 Reagent cover storage place	Used to store the reagent cover.

1.9.3 Accessories and Others

① Sample rack (light blue)

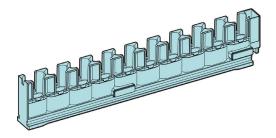

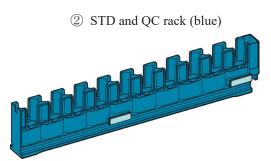

Name	Function
① Sample rack	
② STD and QC rack	Holds sample bottles and sample cups.

Chapter 2 Basic Operation

- 2.1 Basic Screen Operation
- 2.2 Starting the System
- 2.3 Logging In
- 2.4 Initial Settings
- 2.5 Daily Operation
- 2.6 Inspections and Cleaning After Use
- 2.7 Shutting Down the System (Shut Down Mode)
- 2.8 Processing the Drain Tank



Chapter 2 Basic Operation


2 Basic Operation

2.1 Basic Screen Operation

This section describes the functions provided on the [MENU] screen, as well as the buttons on the screen and their basic operations.

2.1.1 [MENU] screen and functions

On the [MENU] screen, functions (a) through (h) can be performed.

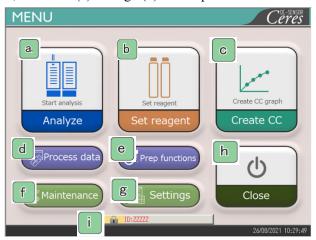


[MENU] screen

a.	Analyze	Starts analyzing. Sets conditions to start analyzing.
b	Set reagent	Sets the reagent. The barcode is automatically read. Touching {Set complete} ends the setting of the reagent.
C	Create CC	Creates CC. Checking and editing CC are also available here.
	Process data	Performs "Process data" or "Process quality control."
d		To open the [Process data] screen, a password is required (if a password was set).
е	Prep functions	Performs prep functions (Initialize, Prime, Cleaning).
f	Maintenance	Performs inspections, replacement parts check, and maintenance.
g	Settings	Performs "System setting" and "Protocol setting" for the system.
h	Close	Turns off the system. After touching this button, the system switch automatically turns off.
i	ID	Used to log in.
		After log-in, the ID of the operator who logged in is displayed.

Screen Configuration and Button Operations 2.1.2

This section describes the [Buttons] and [Tabs] on the screen and how to enter settings.

Screen configuration

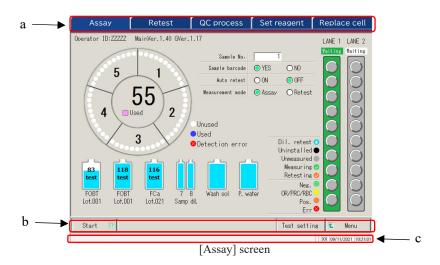

a	Tabs	Tabs are used to select functions. Tabs are provided at the top of the screen.
b	Buttons	Buttons are used to perform processes. At the bottom of the screen, buttons corresponding to the selected screen are displayed. ([Run/Cancel], [Register/Cancel], [Close], etc.)
		Page 18 "■ {Continue} buttons"
	Status bar	Displays the message and the date and time.


Note The screen is only an example.


■ {Continue} buttons

A {Continue} button for close/start is provided on the Time course data range change screen, and {Continue} for close/register is provided on the Test setting screen. In this section, the [Time course data range change] screen and [Test setting] screen are used as examples.

[Time course data range change] screen

When {Continue} is touched, a dialog box opens

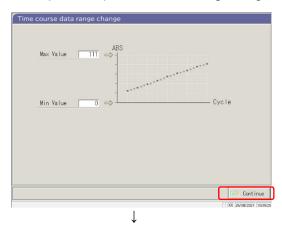

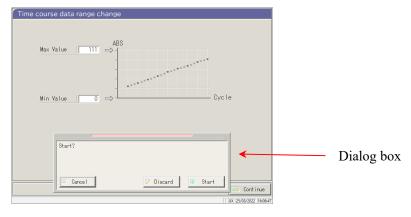


{Cancel}: Closes the dialog box.


{Discard}: Aborts the process and returns to the previous screen (in this case, the [Time course data range change] screen).

{Start}: Executes the process (in this case, the time course range is changed).

[Test setting] screen

Touching the {Register/cancel} button opens a dialog box

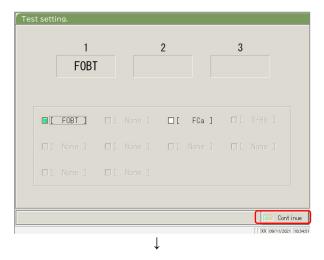

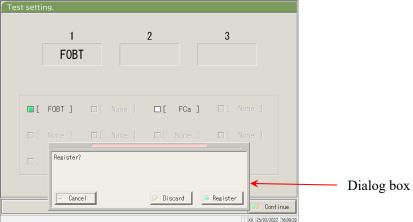

{Cancel} : Closes the dialog box.


{Discard} : Aborts the registration of settings and returns to the previous screen. (In this case, the [Assay] screen or [Output settings] screen.)

{Register}: Registers the settings (in this case, the analysis items are registered.)

Basic Operation

■ Entering a setting

This section describes how to enter, insert, and delete numbers. ([MENU] - [Settings] - [Protocol Settings] - [CC Protocol])

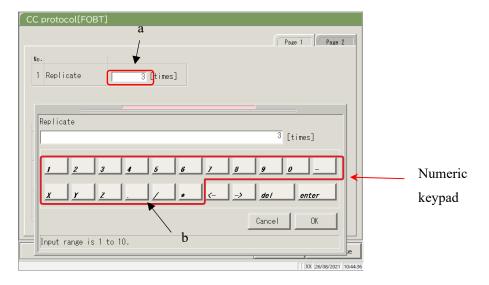

[CC Protocol] screen

Entering a numerical value

- ① Touch the entry field (a). The numeric keypad opens.
- 2 Touch the numeral buttons (b) to enter a numerical value.
- 3 Touch {OK} or {enter}. The numeric keypad closes, and the number is entered in the entry field.
- ④ To cancel the entry, touch {Cancel}. The numeric keypad closes.

Inserting one character

- ① Touch the entry field (a). The numeric keypad opens.
- ② Use {←} and {→} to move the cursor to the desired position to enter a character.

Deleting one character

- ① Touch the entry field (a). The numeric keypad opens.
- ② Use $\{\leftarrow\}$ and $\{\rightarrow\}$ to move the cursor to the desired position to enter a character.
- ③ Touch the {del} key.
 - When an asterisk (*) is entered, the value of the corresponding item can be omitted.


 Entering "*" is not available for some settings. Check the available range by checking the status bar at the bottom of the numeric keypad.
 - Note Entering characters from the keyboard is the same as entering numbers from the numeric keypad (next page).

Switching between capital letters and small letters is available by touching [A/a], and entering symbols is available by touching [Sign].

2 Basic Operation


Radio buttons and check boxes

Radio buttons and check boxes are used to select one or several conditions when setting the condition items.

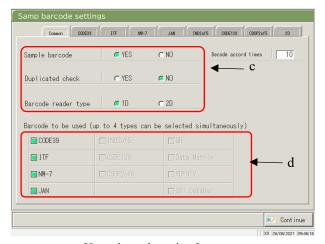

[Samp barcode settings] screen

c	Radio button	Used to select only one item.
d	Check box	Used to select one or more items.

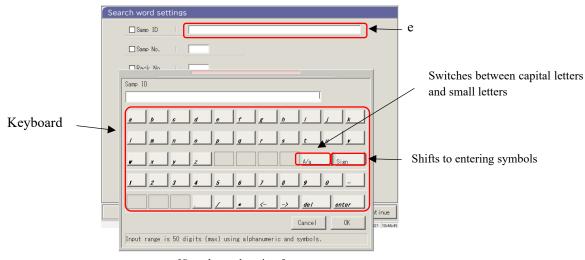


[Search word settings] screen

Entering characters

- ① Touch the entry field (e). The keyboard opens.
- ② Touch the keyboard keys to enter text value.
- ③ Touch {OK} or {enter}. The keyboard closes, and the entered value appears in the entry field.
- ④ To cancel the entry, touch {Cancel}. The keyboard closes.

■ Selecting from a pull-down window

A pull-down window is used to select one item from the provided list.

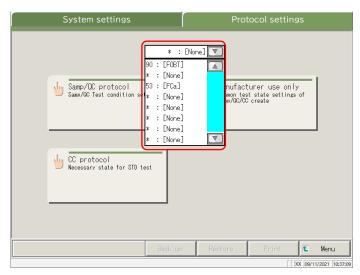

[Protocol settings] screen

Inputting characters


- ① Touch Tuch The pull-down window opens as shown above.
- ② If the desired item is not displayed in the selection list, find the item by scrolling with $\boxed{}$ and $\boxed{}$.
- ③ Touch the desired item in the selection list.
- ④ To cancel the entry, touch outside the pull-down window. The selection list closes.

2 Basic Operation

2.1.3 [Analyzing] Screen and Functions

When touching {Start} on the [Analyze start] screen, [Retest] screen, [QC process] screen, or [Create CC] screen, the [Analyzing] screen opens.

The following functions can be performed from the [Analyzing] screen.

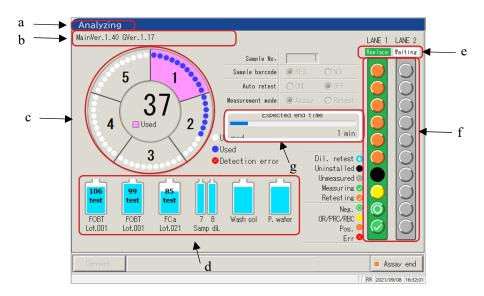


Analyzing] screen

a	Title	Displays the test mode.	
		Initial test, retest analysis	: Testing
		Retest, automatic retest	: Retesting
		QC test	: QC testing
		CC create	: CC creation
b	Operator ID software	The ID of the operator who is cu	rrently logged in is displayed. The system software
	version	version is also displayed.	
С	Cell information	The cell use status is displayed.	
d	Reagent information	The reagent used in the measurement is displayed.	
e	Rack info	The sample setting lane status is displayed.	
		When the rack is drawn in, the d	isplay changes from "Replaceable" to "Using."
		For the using lane, replacement	of the rack and setting a new rack are not available.
f	Sample information	The process status and measuren	nent result for each sample are displayed.
		Note If the rack is replaced duri	ng the analysis, QC process, or CC creation, the result will
		not be reflected.	
g	End schedule time	After dispensing the last sampl	e, the remaining time until the system stops will
	(min)	be displayed.	
		Note While sample dispensing of	ontinues, the time display does not change.

2.1 Basic Screen Operation

2 Basic Operation

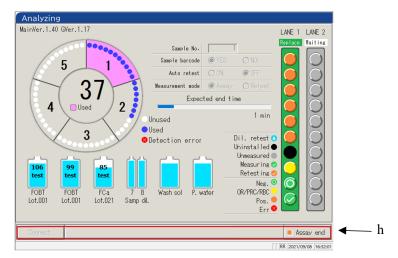

[Analyzing] screen

h Buttons

{Connect}

When the connection with the external computer fails and communication is shut down, {Connect} is displayed. Restart the connection by taking the following procedure.

- ① Remove the cause of the connection failure.
- ② Touch {Connect}.
 - * The connection restarts, and any data which was not output is output.



When the analysis is ended using {Assay end}, transmission ends even if there is data which was not output. In that case, specify the range of the measurement data on the [Test data] screen.

{Assay end}

Ends the testing.


2.1.4 [Set reagent] Screen and Functions

After logging in, the [Set reagent] screen opens.

R1 reagent, R2 reagent, and sample diluent are set in the reagent refrigerator.

Note The reagent can be set on the {Set reagent} tab on the [Create CC] screen.

Note Be sure to remove the reagent cap before executing the test.

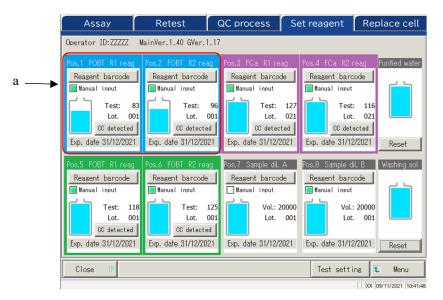

[Set reagent] screen

a	Reagent information	
	Paring condition	Paired R1 reagent and R2 reagent are displayed in different colors.
		There are three colors: blue, purple, and green.
	Reagent barcode	The reagent barcode can be edited manually.
		When edited, the check box for "Manual input" will be selected (■). When the
		check mark is removed, the barcode returns to the previous barcode.
		Page 39 " <entering a="" barcode="" keypad="" numeric="" reagent="" the="" using="">"</entering>
		Note) Manual input and editing are available only when the user is logged in using an
		administrator ID, or when the operator control is "No."
	{CC detected} {No CC}	When there is a CC for the reagent lot, "CC detected" is displayed.
	{NO CC}	When there is no CC, "No CC" is displayed.
		Touching this button displays the CC.
		Note When there is no CC, the test cannot be started. In that case, create a CC.
	Exp. date	An expiration date is displayed.
		When there is a sample condition error, it is displayed here (expired, volume
		shortage, etc.).
	Test/Vol.	The sample volume is displayed with a graphic and a numerical value.
		(The displayed number of tests correspond to the use count.)
	Lot.	Displays the sample lot.

2.1 Basic Screen Operation

Replace cell QC process Set reagent Retest Operator ID:ZZZZZ MainVer.1.40 GVer.1.17 Reagent barcode Reagent barcode Reagent barcode Reagent barcode ■ Manual input Manual input Manual input Manual input Test: Test: Test: 127 Lot. 021 Test: 116 001 Lot. 001 Lot. Lot. 021 b CC detected CC detected CC detected CC detected Exp. date 31/12/2021 Exp. date 31/12/2021 Exp. date 31/12/2021 Exp. date 31/12/2021 Pos.7 Sample dil. A Reagent barcode Reagent barcode Reagent barcode Reagent barcode Manual input Manual input Test: 118 Vol.: 20000 Vol.: 20000 Lot. 001 Lot. 001 Lot. 001 Lot. 001 CC detected CC detected Exp. date 31/12/2021 Exp. date 31/12/2021 Exp. date 31/12/2021 Exp. date 31/12/2021

Basic Operation

[Set reagent] screen

b	Washing solution/Purified water information		
	Remaining volume	Remaining volume Remaining volumes of the washing solution and purified	
		water bottles are displayed graphically. When the sensor detects zero volume,	
		"Empty" is displayed.	
	{Reset}	This is used after replacing the washing solution or purified water bottles.	
		Touch {Reset} and {Close} to reset the volume display for the bottles.	
С	Buttons		
	{Close}	Touch this after completing the reagent setting.	
		The system reads the barcode and updates the statuses of the reagent	
		refrigerator/washing solution and purified water bottles.	
	{Test setting}	Sets test items.	
		Page 198 "6.1.7 Data output – [Test Setting]"	
	{Menu}	Returns to the [MENU] screen.	
d	Tabs	Moves to other screens.	

Note

If {Close} is not touched after setting a reagent or editing a barcode, the screen cannot be changed using the tabs or buttons.

2 Basic Operation

2.2 Starting the System


Start the system by turning on the switch.



[Startup] screen



[Message] screen

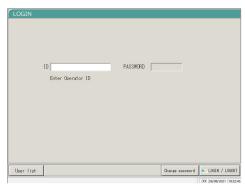

[LOGIN] screen

Note) When "Operator mode" is set to "No," the [Set reagent] screen opens.

1 Check that the primary power switch is on.

If the system is not on, turn on the primary power switch.


- 2 Turn on the System switch.
 - * The startup screen opens.

- 3 If "Exchange buffer and wash sol to P. water" were set to "Yes" in the previous shutdown of the system, the message screen opens. Set the washing solution bottle.
 - ① Remove the washing solution bottle filled with purified water.
 - ② Pour the washing solution into the washing solution bottle.
 - ③ Touch the {Close} button.
 - * The [LOGIN] screen opens.

2.3 Logging In

To use all functions of the system, logging in is required.

After the ID information is entered on the [LOGIN] screen, the operator can log in to the system. The ID of the operator who logged in is displayed.

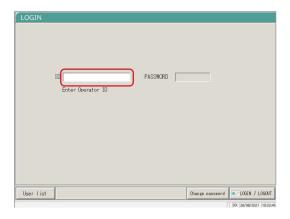

Enter an operator ID.

①Touch the operator ID field.


Number of registration available: Number of operator IDs which can be additionally registered.

{User list}: The list of registered operator IDs is displayed.


[LOGIN] screen

Note Entering an operator ID is also available by selecting from the {User list}.

Page 30 "■ User list"

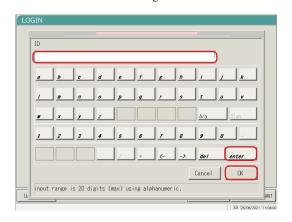

- ② Enter an operator ID and touch {OK} or {enter}.
 - * The keyboard dialog closes.

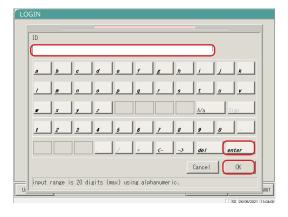


- 2 Enter a password.
 - ① Touch the password entry field.
 - ② Enter a password and touch {OK} or {enter}.
 - * The keyboard dialog closes.



3 Touch {LOGIN/LOGOUT}.

2 Basic Operation

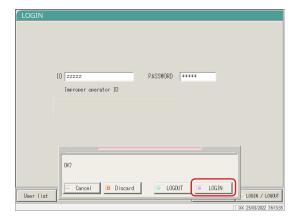

4 Touch {LOGIN}.

* The user is logged in to the system. The [Set reagent] screen opens.

{Cancel}: Closes the dialog box. {Discard}: Returns to the [MENU] screen while keeping the current logged in/out condition.


{LOGOUT}: Logs out. The screen returns to the [MENU] screen. {LOGIN}: Logs in.

2.3 Logging In


■ User list


On the [LOGIN] screen or "User account setting" of the System setting, selecting an operator ID from the User list is available instead of entering an operator ID.




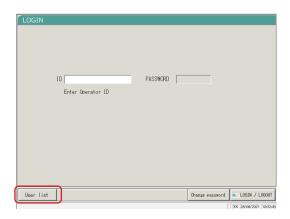

[LOGIN] screen

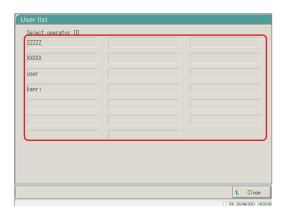

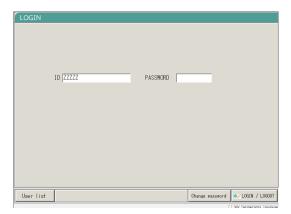

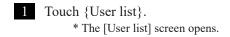

From the displayed list, touch an operator ID to login.

{Close}: The screen returns to the previous screen.

The operator ID is displayed.

For the procedure to enter the password, see page 28 "2.3 Logging In."

2.4 Initial Settings

Before using the system for the first time, initial settings are required.

The initial settings are roughly divided into "System settings" and "Protocol settings."


For both settings, touching {Settings} on the [MENU] screen displays the corresponding screen to make each setting. This section describes the outline of the system settings and the protocol settings.

For detailed operating procedures, see "Chapter 6 Settings" on page 182.

Note

When the user is logged with an administrator ID, or when "No" is selected for "Operator mode," settings cannot be changed.


2.4.1 System Settings

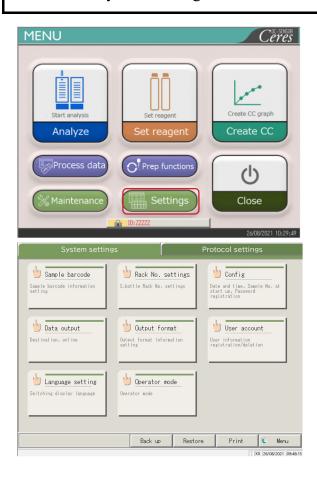

1 Touch {Settings}.

- 2 Touch the button for the item to be set.
 - * The corresponding setting screen opens. (See the next page.)


2 Basic Operation

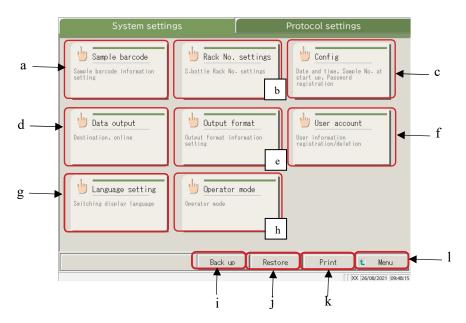

[System settings] screen


a	Sample barcode	Set conditions for reading sample barcodes.
	•	
b	Rack No. settings	Register racks for new and former sampling bottles.
c	Config	Set the date and time.
		Register a password.
		Select the purified water supply method and waste water drain method.
d	Data output	Select a data output destination.
		Select formats for the printer output, external media output, and online output.
		Set the online communication control.
		Set the analysis items.
e	Output format	Select the output format.
		Select items for the printer output and online output.
		Set the output items and output order for external media.
f	User account	Register, change, and delete the IDs of the operators who can log in to the system.
g	Language setting	Switch the system display language.
h	Operator mode	Switch the system operator mode.
i	Back up	Save the system settings and protocol settings to external media.
j	Restore	Restore the system settings and protocol settings saved on external media.
k	Print	Print the system settings.
1	Menu	Return to the [MENU] screen.

2.4.2 Protocol Settings

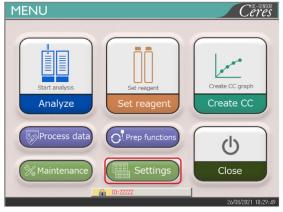


1 Touch {Settings}.



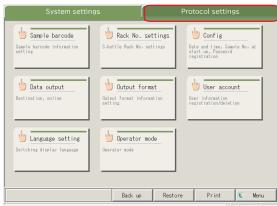

2 Touch the {Protocol settings} tab.

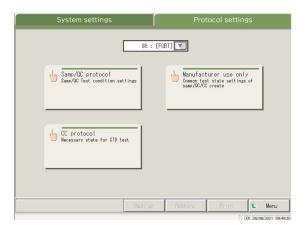


Touch the button of a setting item.

* Corresponding setting screen opens. (See the next page.)

2.4 Initial Settings

2 Basic Operation

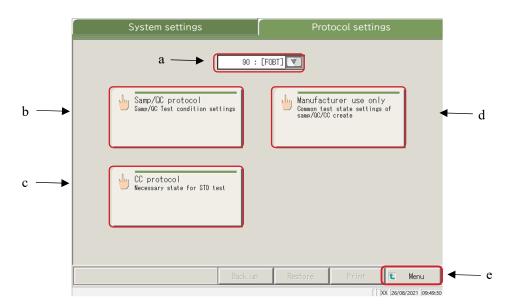

Protocol Settings screen


a	Test item	Select the test item to be set.
ь	Samp/QC protocol	Set the test conditions used when measuring samples/QC samples.
С	CC protocol	Set the conditions used when creating a CC.
d	Manufacturer use only	Set the common conditions necessary for measuring samples, QC
		samples, and STD samples (for maintenance).
e	Menu	Return to the [MENU] screen.

2.5 Daily Operations


This section describes the flow of daily operation and the operation procedures.

2.5.1 Flow of Daily Operations

Note Complete the initial setting of the system before starting operations.

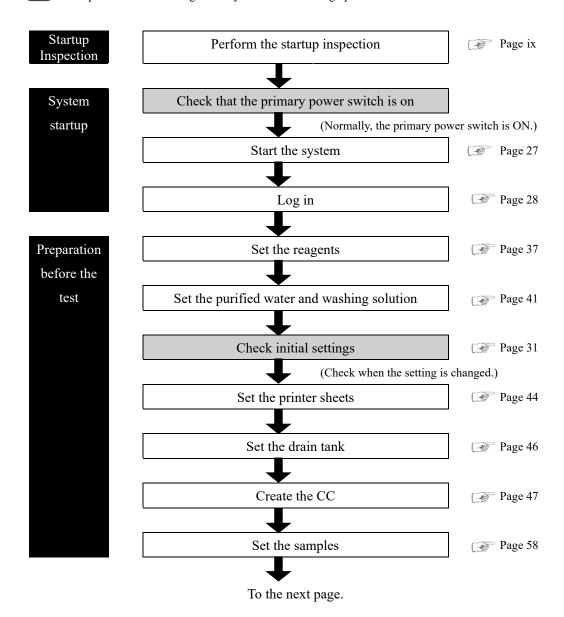

2 Basic Operation

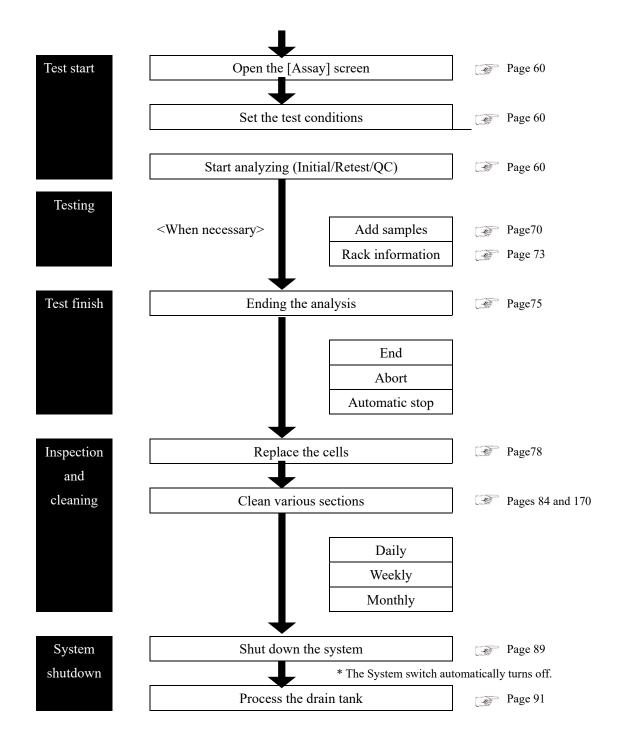

2.5.2 Setting the Reagent

R1 reagent, R2 reagent, and diluent are set in the reagent refrigerator.

A maximum of three sets of reagents can be set for one item.


 \triangle

Warning




· Wear protective gears (gloves, goggles, etc.) when handling the reagents. Failure to observe this precaution may lead to infection from the reagents.


Note The reagents can also be set in the {Set reagent} tab on the [Analyze] screen or [Create CC] screen.

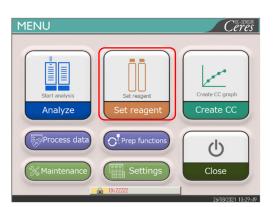

1 Touch {Set reagent}.

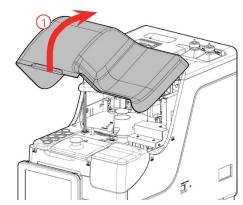


- 2 Set the reagent bottles into the reagent refrigerator.
 - ① Open the safety guard.

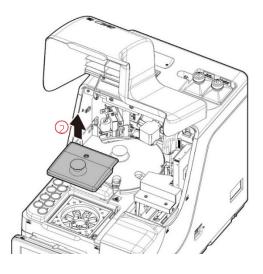



② Open the reagent refrigerator cover.


Preparation before the test



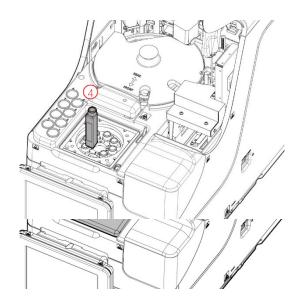

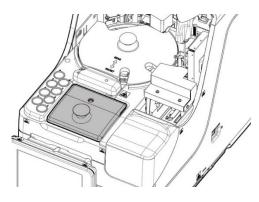


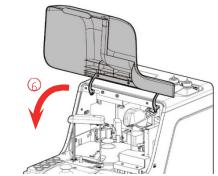

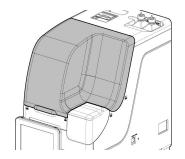

- ③ Remove the cover of the reagent bottle to be set.
- ④ Remove the reagent bottles currently set. Set the new reagent bottles in the proper positions.

Set R1 or R2 reagents in Pos. 1 to 6. Set the diluent in Pos. 7 and 8.

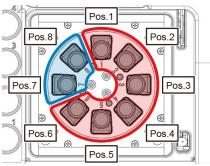

Top view

⑤ Set all reagents and close the reagent refrigerator cover.

6 Close the safety guard.

(Condition that the safety guard is closed.)

⚠

Caution




After setting the reagent bottles, close the reagent refrigerator cover and the safety guard.

Failure to observe this precaution could result in a reagent nozzle contacting the cover.



- <Entering a reagent barcode using the numeric keypad>
- ① Touch {Reagent barcode}.
 - * The numeric keypad opens.
- ② Check the 24-digit barcode on the reagent bottle. Enter the first 23 digits (exclude the last digit, the check digit) using the numeric keypad.
- ③ Touch {OK} or {enter}.
 - * The numeric keypad closes.
 - * A check mark is placed in the [Manual input] check box.

{Reagent barcode} becomes active only when the user is logged in with an administrator ID or when the Operator mode is set to "No."

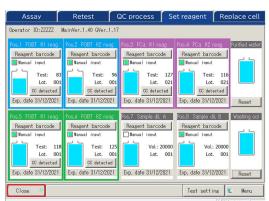

- 3 Touch {Close}.
 - * The reagent barcode is automatically read.
 - * The reagent information is updated.
 - * If the reagent barcode could not be read on the first attempt or if the reagent bottle was not set, the reagent barcode will be read multiple times.

This may take longer to complete the reagent set than a single successful read.

4 Check the reagent information (barcode, pairing condition, etc.)

{CC detected}: Displays CC.

{Test setting}: Sets a test item.

Page 198

{Menu} : Returns to the [MENU]

- Note If there is no CC for the set R1 and R2 reagents, "No CC" is displayed. In that case, create a CC.
- Note Pairing indicates the condition in which R1 and R2 reagents in the same lot in the same test item are set as a pair.
- (Note) After the reagent barcodes are read, the system automatically conducts pairing.
- Note Do not open the reagent refrigerator cover while reading the reagent barcode. If the cover is opened, reading the barcode fails, and an error (ERR#2-103) is output.
- Note) Expired reagents are not tested.

Preparation before the test

⚠

Request

Reagent barcode configuration

- The barcodes on the reagent bottles are 23-digit ITFs (excluding the check digit).
- · When entering a barcode, follow the configuration below.

KKKAABBCRYYMMDDXXXNNNNNM (barcode)

KKK: 3-digit manufacturer code (026)

AA: 2-digit item code (01 to 99)

90:FOBT 53:FCa

BB: 2-digit product code (01 to 99)

C: 1-digit bottle capacity (1: 20 mL)

R: 1-digit reagent type (1: R1 2: R2 3: diluent)

YYMMDD: 6-digit expiration date (YYMMDD)

XXX: 3-digit lot number (000 to 999)

NNNNN: 5-digit bottle number (00001 to 99999)

M: Check digit

Switching between a reagent bottle and a diluent bottle is as follows.

Reagent

Before the first dispensing of each sample, compare the reagent test number and the number of sample replications. If the number of reagent tests in use is too small, a shortage is judged, and the bottle automatically changes. The detailed conditions for the bottle change vary for each reagent lot number.

Measurement	In case of the same lot number	In case of different lot numbers
Sample (retest)	The bottl	e changes.
QC	The bottle changes.	The bottle does not change.
CC	The bottle does not change.	

(Note) It will not switch to a lot for which CC was not created. Set the reagent of the lot for which a CC has been created.



If the reagent test number becomes "0" due to trouble such as a detection failure of the reagent liquid level, further loading of the samples stops. Instead, the measurement results for the samples for which dispensing has completed are output.

· Diluent

Bottles are switched automatically. However, if the liquid amount in a bottle becomes zero while creating the diluent system, the detailed conditions for the bottle change depending on the reagent lot numbers.

In case of the same lot number	In case of different lot numbers
The bottle changes.	The bottle does not change.

2.5.3 Preparing Purified Water

Fill the purified water bottle with purified water.




Request

- · Always use the provided purified water bottle.
- · Always dismount the bottle from the system, and then refill it.
- · Always check that the bottle is sufficiently filled.


Basic
Operation

Note Purified water bottle size: 500 mL.

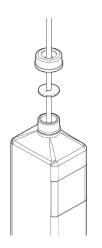

- Loosen the purified water bottle lid (white). Remove the hose.
- Dismount the purified water bottle from the system.

 Drain remaining purified water from the bottle.
- 3 Refill the purified water bottle.

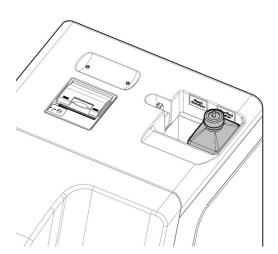

- 4 Set the purified water bottle at the right-hand side ("Purified water") of the purified water installation system.
- Insert the hose into the purified water bottle opening. Close the bottle lid.

Preparation before the test

Basic

Operation

2.5.4 Preparing Washing Solution

Pour washing solution diluted to 3% into the washing solution bottle.



Request

- · Before using the system, always check that the washing solution is sufficient.
- · Always use the attached washing solution bottle (accessory).
- · Always remove the bottle from the system before refilling the washing solution.




Warning




· Wear protective gear (gloves, goggles, etc.) when handling the washing solution.



Washing solution bottle: 500 mL.

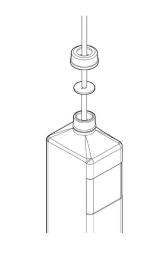

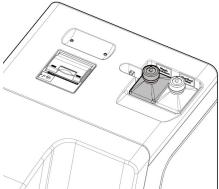

- Loosen the washing solution bottle lid (yellow) and remove the hose.
- Dismantle the washing solution bottle from the system.


 Drain the remaining washing solution from the bottle.
- 3 Dilute the washing solution to a 3% concentration.
 - \bigcirc Pour 15 mL of washing solution (undiluted) into the washing solution bottle.
 - ② Pour purified water into the bottle to bring the total washing solution amount to 500 mL.
- 4 Set the washing solution bottle on the left side (washing solution) of the washing solution/purified water setting area.
- 5 Insert the hose into the washing solution bottle opening. Close the bottle lid.



Washing solution without an indication of "(undiluted solution)" is the washing solution diluted to 3%.


2.5.5 Washing Solution and Purified Water Settings

Set up the washing solution bottle and the purified water bottle, and update the system.

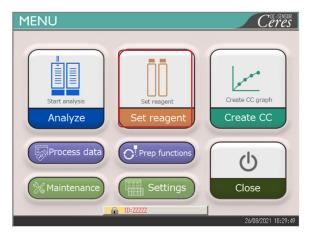


Touch {Set reagent} on the [MENU] screen.



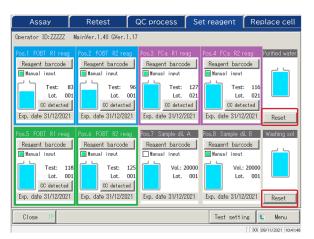

- Touch {Reset} for the purified water and the washing solution.
 - * {Reset} turns to green.

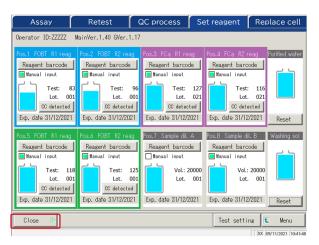

- 3 Touch {Close}.
 - * Information on the washing solution and purified water is updated.


Preparation before the test

Preparation before the test

Basic Operation

2.5.6 Setting Printer Sheets

Place the printer paper into the printer.

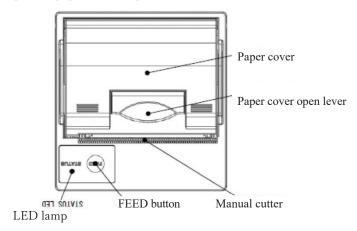




Caution




Pay close attention to the blade of the manual cutter.

Failure to observe this precaution may lead to injuries.



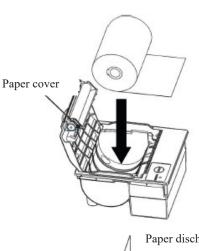

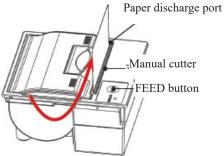

- ① Raise the paper cover open lever, and open the cover.
- ② Set the paper roll as shown in the drawing.(If the roll is set upside down, printing is not possible.)
- ③ Draw out the end of the roll from the paper discharge port.
- ④ Bring down the paper cover.
- ⑤ Push down on both ends of the paper cover to close it. Check that the cover is locked.
- ⑥ Press the FEED button.
- 7 Cut the sheet with the manual cutter.

Basic Operation

■ Printer LED

The LED on the printer indicates the printer status, including errors.

<Normal conditions>

LED	Printer condition
Green	Stand-by, ready for printing
Blinking green	Initializing

<Recoverable Errors>

LED	Printer condition
Red	Out of paper
Blinking red	Temperature abnormal (70 °C or higher)

<Unrecoverable Errors>

LED	Printer condition
Blinking red and green	Voltage too high
Blinking red and green	Voltage too low

2.5.7 Checking the Drain Tank

(The tank connection with the system is set up by our service personnel.) Check that the hose attached to the drain tank is connected to the system.

 \triangle

Warning




Wear protective gear (gloves, goggles, etc.) when working with the drain tank. Failure to observe this precaution may lead to infection.




Caution



- Before using the system, empty the drain tank.
 Failure to observe this precaution could cause waste liquid to overflow.
- Install the drain tank at a position lower than the system.
 If the drain tank is installed at a position higher than the system, waste liquid may not drain properly.

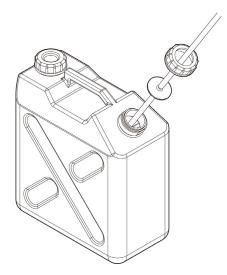

1 Connect the hose to the drain tank.

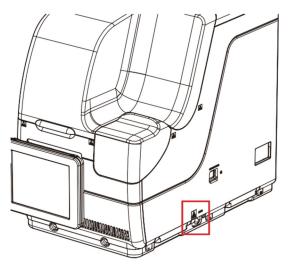

2 Check that the hose is connected to the drain connect or at the right side of the system.



Preparation before the test

2

Basic Operation

2.5.8 Creating CC

When R1 reagent, R2 reagent, and sample lots are newly installed, create a CC which corresponds to the lot.



Warning



Biohazard

Wear protective gear (gloves, goggles, etc.) when handling the calibrator and the QC samples.

Failure to observe this precaution may lead to infection.

1 Setting the STD Sample and QC Sample to the Rack



Request

- When pouring the calibrator and the QC sample into the sample cup, check that there are no bubbles.
- Note When measuring the STD sample and the QC sample, use the "STD/QC rack." Set only the STD sample and the QC sample into the rack. When measuring only QC, use the "QC process."

Page 65 "2.5.12 Starting Analysis (QC Sample)"


Set the calibrator and the QC sample into the STD/QC rack.

(Note) Numbers in the figure are rack positions. The rightmost position is No.1.

Preparation before the test

2 Basic Operation

2 Setting the STD/QC Rack into the System

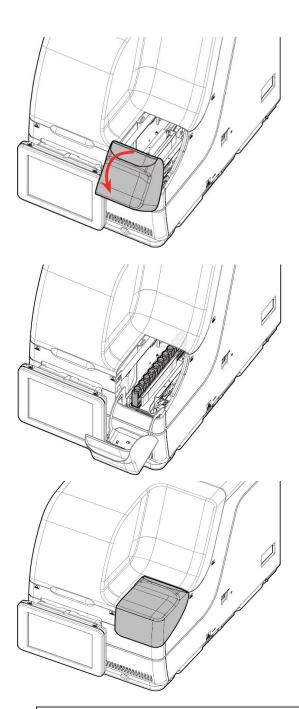

1 Open the rack lane cover.

- 2 Set the rack into rack lane 1 or 2 with rack position No. 1 at the inner-most side of the system. Be careful set the rack in properly.
- Note When setting the rack, be careful with the front/rear orientation. Push the rack in until its end touches the inner section of the system.
- 3 Close the rack lane cover.



Caution




Set the rack in the system with rack position No. 1 at the innermost end.

Failure to observe this precaution may cause problems.


48

3 Creating CC

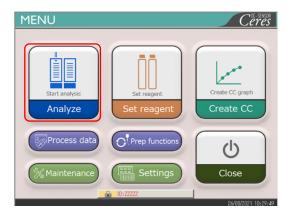


1 Touch {Analyze}



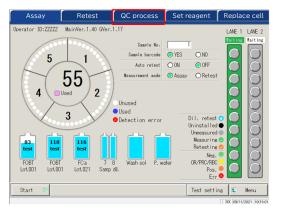


2 Touch the {QC process} tab.

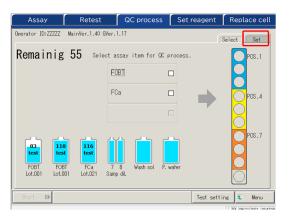

3 Touch the {Set} tab.



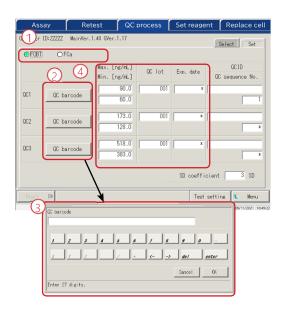

- 4 Set each QC item.
- ① Select a radio button (•) for the test item to create the CC.
- ② Touch {QC barcode}.
- ③ Read the QC barcode corresponding to the selected test item or QC1 3 using the handy barcode reader. When entering a barcode using the numeric keypad, enter the 27-digit QC barcode excluding the last digit (check digit).
- Max./Min. values, QC lot, and expiration date are automatically entered from the barcode.
- ⑤ Enter the remaining settings. (See the table on page 68.)
- Note Expired QCs cannot be used.



Request

About QC barcode configuration

- The QC barcode is a 27-digit ITF (excluding the check digit).
- · When entering a barcode, follow the configuration below.

KKKAABBCYYMMDDXXXYYYYYZZZZZM (barcode)

KKK: 3-digit manufacturer code (026)

AA: 2-digit item code (01 to 99)

90:FOBT 53:FCa

BB: 2-digit product code (01 to 99)

C: Concentration type


YYMMDD: 6-digit expiration date (YYMMDD)


XXX: 3 digit lot number (000 to 999)

YYYYY: Min. value of reference range (00001 - 99999)

ZZZZZ: Max. value of reference range (00001 - 99999)

M: Check digit

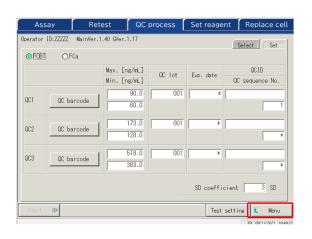

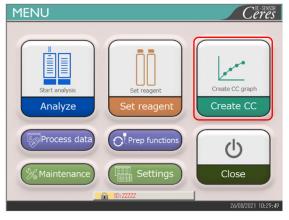

Replace cell Select Set 1 Set 2 Remaining 55 Specify the lot to create CC. CC detected 83 FOBT 001 001 118 CC detected FOBT 116 CC detected Lot. Expiry date 31/12/2021 101 Cal. barcode CC list


5 Touch {Menu}. Returns to the [MENU] screen.

> 2 Basic Operation

6 Touch {Create CC}.

7 Touch a radio button • to select the item/lot to create the CC.



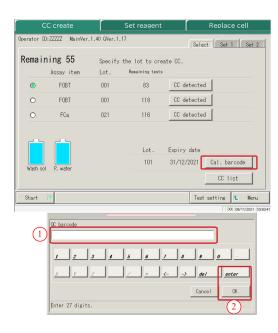

8 Touch {Cal. barcode} to enter the barcode of the calibrator.

 $\{CC\ detected\}$: Displays the currently registered CC.

- ① Read the calibrator barcode with the handy barcode reader. When entering a barcode using the numeric keypad, enter a 17-digit calibrator barcode excluding the last digit (check digit).
- ② Touch {OK} or {enter}.
 - * The numeric keypad closes.
 - * The lot. and expiration date are displayed.

Note An expired calibrator cannot be used.

{CC list}: Displays a list of CCs.

Page 57

 $\{Test\ setting\}$: Sets the test item.

Page 198

{Menu}: Returns to the [MENU] screen.

 \triangle

Request

Calibrator barcode configuration

- The 17-digit barcode on the bottle is an ITF (excluding the check digit).
- When generating a barcode, follow the configuration below.

KKKAABBYYMMDDXXXNM (barcode)

KKK: 3-digit manufacturer code (026)

AA: 2-digit item code (01 to 99)

90:FOBT 53:FCa


BB: 2-digit product code (01 to 99)

YYMMDD: 6-digit expiration date (YYMMDD)

XXX: 3-digit lot number (000 to 999)

N: Spare (O)

M: Check digit

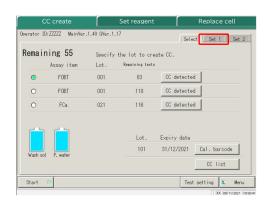


9 Touch the {Set 1} tab to set the number of replications for STD.

(Note) When the number of remaining cells is smaller than the number of STD measurement tests, a message about the cell shortage will appear. Check the number of remaining cells before performing the STD tests.

Operation

Make settings other than those concerning STD replication on the {Set 1} and {Set 2} tabs as needed.

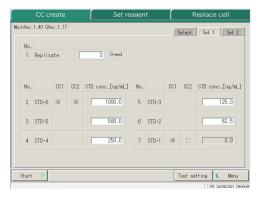


{Set 1} tab

For details on the settings, see page 224 "6.2.2 CC Protocol Setting" – {Page 1} tab.


GVer.1.17 rator's gment	€ YE	ES	C NO		Se	elect Set	: 1 Set 2
gment	€ YE	ES	C NO				
gment	€ YE	ES	C NO				
Snar S							
Snac S							
OPEC C	STD-6 Min	DA [2000	12	STD-1 Min	DA	-50
S	STD-6 Max	DA [4000	13	STD-1 Max	DA	50
S	STD-6-3 ba	nckfit±	5 9	6 14	STD-2 back	kfit±	10 g
			STD-6 Max DA			, , , , , , , , , , , , , , , , , , , ,	

{Set 2} tab

For details on the settings, see page 224 "6.2.2 Protocol Settings" – {Page 2} tab.

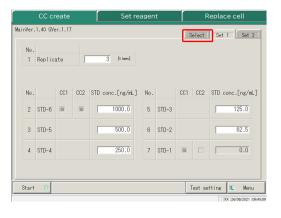

10 Touch the {Select} tab.

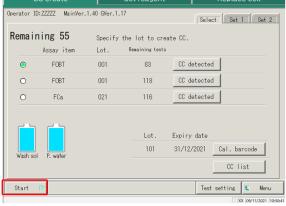

- . 12 Register or cancel CC.
- ① When the test ends, the [Check CC] screen opens.


* Start STD/QC measurement.

The screen shifts to the [CC create] screen.

② Check the CC.

11 Touch {Start}.

For details on the [Check CC] screen, see page 56 "■CC check."

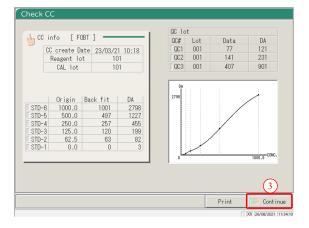

- OC lot


 OC# Lot

 OC1 001

 OC2 001


 OC3 001 CC info [FOBT] DA 121 231 901 | CC create Date | 23/03/21 10:18 | Reagent lot | 101 | CAL lot | 101 | 77 141 STD-6 STD-5 STD-4 STD-3 250.0 125.0 STD-2 STD-1 - Cancel Cont inue
- <When registering the CC >
- ③ Touch {Continue}.
- 4 Touch {Register}. The tested CC is registered in the system.
- < When not registering the CC >
- ③ Touch {Continue}.
- 4 Touch {Discard}. Create the CC again.
 - Page 47 "2.5.8 Creating CC"
 - * CC creation ends, and the screen returns to the [CC create] screen.
- In the CC protocol setting, if "Operator's Judgment" after STD/QC test is set to "No," the tested (Note) CC is automatically registered in the system, and the [Check CC] screen does not open. Page 224 "6.2.2 CC Protocol Settings"

4 Removing a Rack

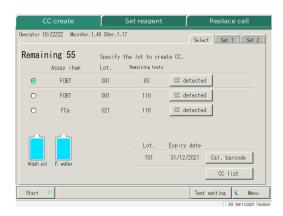


After the STD/QC test ends, the screen returns to the [Create CC] screen.


2 Basic Operation

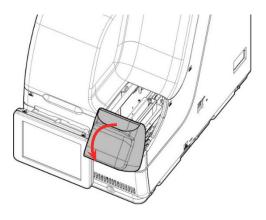

2 Open the rack lane cover.

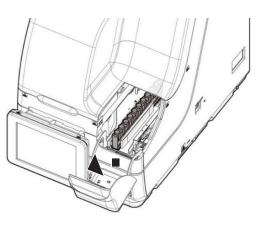


3 Remove the STD/QC rack.

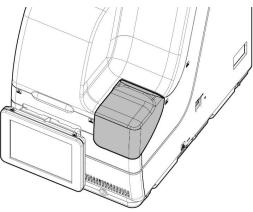


4 Close the rack lane cover.

■ CC check

If the CC has been created, touching {CC detected} on the [Create CC] screen or the [Set reagent] screen displays the CC information (test date and time, reagent lot, etc.) and the CC graph.

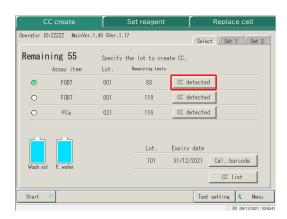

1 Touch {CC detected}.

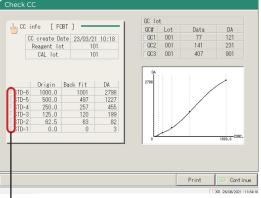

2 Check the CC information and the graph.


{CC info}: Edit and recalculate the CC. Page 128

{CC list}: Display CC list.

Page 57

{Print} : Print the CC information and the graph.

Turns red when the value is outside of the specified range.

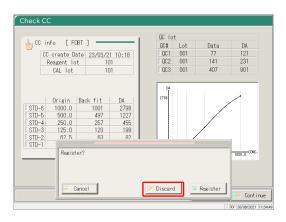


- Touch {Continue}.
- 4 Touch {Discard}.
 - * The display returns to the previous screen.

- (Note) Origin values and DA values can be changed on the [Edit and Recal CC] screen.
 - Page 128 "3.3.8 Editing and Recalculating CC"
- Note) When the measurement result of the STD sample is out of the standard range, the box at the left side of the STD-# turns red.
 - Page 224 "6.2.2 CC Protocol Settings"


Preparation before the test

When {CC list} is touched on the [Create CC] screen, the [CC list] screen opens. Detailed CC information can be referred from the list.

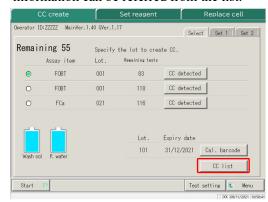

1 Touch {CC list}




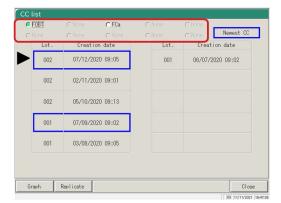

Lot

001


06/07/2020 09:02

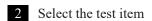

Note FOBT uses the common CC.

3 Check the CC list.


Items in blue frames □ are the latest CC or each lot.

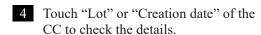

The CC with the ▶ mark is the selected CC.

{Graph}: Displays the selected CC.

{Replicate}: Displays the STD replication. Page 118

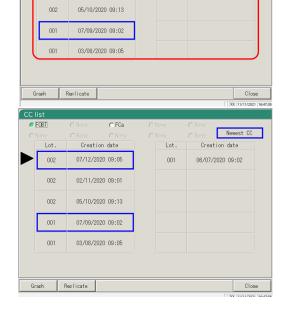

⁵ Touch {Close}.

Preparation before the test

CC list

002

07/12/2020 09:05

02/11/2020 09:01

^{*} The display returns to the previous screen.

Basic Operation

2.5.9 Setting Samples

Set the rack with the samples in the rack lane.

When racks are set in both Lanes 1 and 2, processing starts from Lane 1.



Warning




• Wear protective gear (gloves, goggles, etc.) when handling the samples.

Failure to observe this precaution may lead to infection.

1 Setting samples (sampling bottles, sample cups) in the rack

Note Set the sampling bottles vertically.

· If a sampling bottle is set at an angle piercing might not be conducted properly.

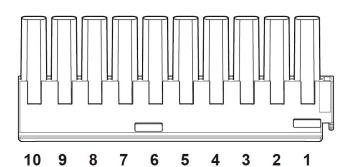

Up to ten sampling bottles or sampling cups can be set in one rack.


Set the sampling bottles in the rack in a way that the barcodes face the user.

Note Numbers in the figure are rack positions. No. 1 is on the right.

Preparation before the test


(Note)

2 Setting a sample rack in the system

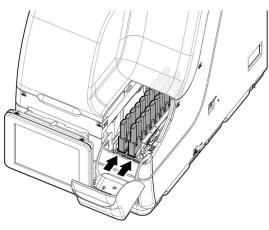

1 Open the rack lane cover.


Basic Operation

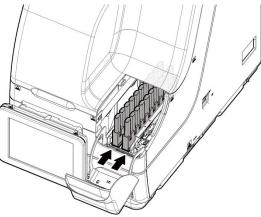

2 Set the rack into the rack lane with rack position No.1 at the innermost end. Pay close attention to set the rack properly.

Note When setting the rack, pay close attention to the front and rear orientation of the rack. Push the rack in until the rack contacts the innermost section of the system.

3 Close the rack lane cover.

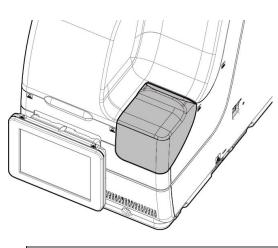

 \triangle

Caution




• Set the rack in the system with rack position No. 1 at the innermost end.


Failure to observe this precaution may cause problems.

2.5.10 Starting Analysis (Initial Test)

Before starting the first sample measurements, perform the initial inspection.

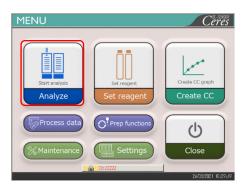

1 Touch {Analyze}.

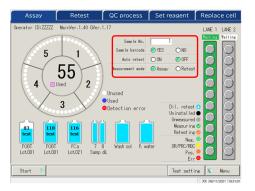

2 Set each item (see the following table).

{Test setting}: Sets the analyzing items.

Page 198

 $\{Menu\}$: Returns to the [MENU] screen.


The changed settings are saved for each setting.

Item	Range/selectio	Content
Sample No.	1 - 99999 The sequence No. of the first sample.	
(When the system power is turned off, the		
sample No. returns to the "Sample start		
No." set in [System Settings] - [Config.])		
Sample barcode		Whether or not the sample barcode is used.
(Even when the system power is turned		This setting is also reflected in [Settings] -
off, this setting is kept.)		[System Settings] - [Sample barcode settings].
	Yes:	The sample barcode is used.
	No:	The sample barcode is not used.
Auto retest		Whether or not automatic retest is performed
(Even when the system power is turned		for error samples.
off, this setting is kept.)		For error samples, see page 217 "6.2.1
		Sample/QC Protocol Settings."
	On:	Automatic retest is performed for error samples.
	Off:	Automatic retest is not performed for error
		samples.
Measurement mode		The measurement mode is selected.
(This setting is kept until the system power		
is turned off.)	Assay:	Assay is selected when the test is conducted for
		the first time.
	Retest:	Retest is selected when the sample is retested.

Note Retest means testing a sample that was already tested. Piercing the sample bottle is not performed again. All of other operations will be the same as those of the "first test."

Note For automatic retest, the same sample sequence number is allocated for both the first test and the retest.

Test start

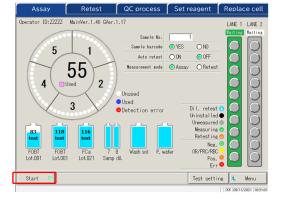

Touch the {Start} button.
When the priming completes, dispensing of the sample starts.


2 Basic Operation

2.5.11 Start Analyzing (Retest)

When retesting a sample, the sample is re-analyzed. Set the dilution factor for each test item and perform the test.



1 Touch {Analyze}.

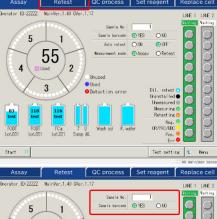

2 Touch the {Retest} tab.

{Test setting}: Sets the item to be analyzed.

Page 198

{Menu}: Returns to the [MENU] screen.

The settings are saved for each item.

Assay Retest OC process Set reagent Replace cell

Ocerator ID: 22222 Mainfert, 1, 40 Ger. 1, 17

Seal is Sec. 1


Seal is Seal


3 Set each item.


Item	Range/selection	Content
Sample No.	1 - 99999	The sequence No. of the leading sample.
(When the system is turned off, the		
sample No. returns to the "Sample start		
No." set in [System Settings] - [Config.].)		
Sample barcode		Whether or not the sample barcode is used.
(The setting remains even after the system		• The setting is reflected to [Settings] -
is turned off.)		[System Settings] - [Samp barcode settings].
	Yes:	Use the sample barcode.
	No:	Don't use the sample barcode.

(Note)

To return to the [MENU] screen, touch {Menu}. The display returns to the [MENU] screen while the setting is unchanged.

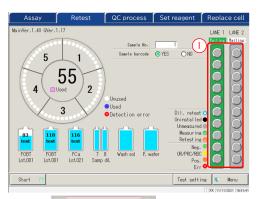

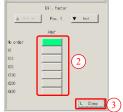

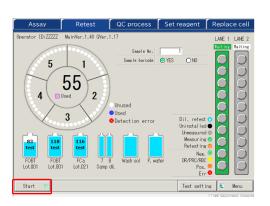

- 4 Set the sample dilution factor.
- ① Touch a sample position on the rack graphic. A dialog opens.
- ② Set the dilution factor.

In the drawing on the left, the following dilution factor is set.

> FOBT : Diluted by a factor of 400

Operation

- {▲Previous}: Selects the previous sample (moves up)
- { ▼ Next}: Selects the next sample (moves down)
- ③ Touch {Close} to close the dialog. "Dil. retest () " is displayed for samples which the dilution factor is set.
- 5 Touch {Start}.


When the pre-test priming completes, sample dispensing starts.

_

■ Operation flow of the dilution retest

On the [Retest] screen, the "Dilution factor" is set. After the test starts, the sample is diluted in the cell, and the test starts.

The cell consumption is higher as compared to the regular tests.

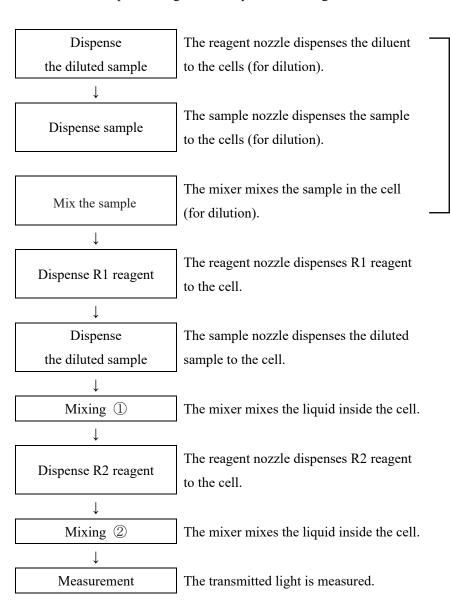

The dilution count varies depending on the set magnification ratio

x10, x20: 1 time x100, x200, x400: 2 times

One cell is used for one dilution.

2.5.12 Starting Analysis (QC Sample)

When testing the QC sample only, the QC process is used.

The [QC process] is configured on two pages. Switch pages by touching the {Select} tab and the {Set} tab.

2 Basic Operation

Note If testing STD and QC at the same time, perform "CC create." Page 47 "2.5.8 Creating CC"




Warning




• Wear protective gear (gloves, goggles, etc.) when handling the QC samples.


Failure to observe this precaution may lead to infection.

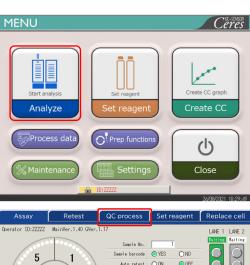

Touch {Analyze}.

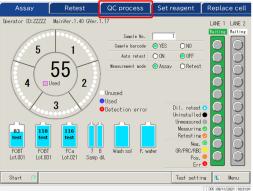

2 Touch the {QC process} tab.

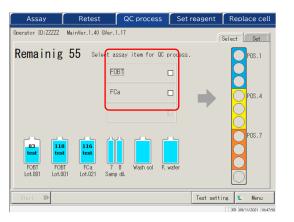


- 3 Set the QC sample.
- ① Select the test item to be measured (■). The QC sample setting positions in the rack are displayed in different colors.


{Test setting}: Sets the item to be analyzed.
Page 198

{Menu}: Returns to the [MENU] screen.
Changed settings are saved.


Note Items for which R1 and R2 samples are set are displayed on the screen. If there is no item to be tested, set the reagent of the corresponding item.

Page 37 "2.5.2 Setting the Reagent"

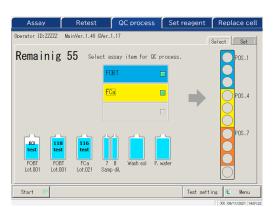


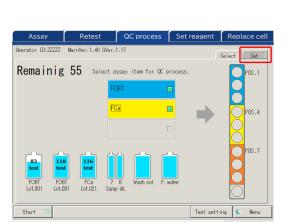

- ② Prepare the STD and QC racks.
- ③ Set the QC samples in the positions corresponding to the colours. In the case of the left figure, set the QC samples as follow.
 - FOBT for Pos. 1 3
 - FCa for Pos. 4 6

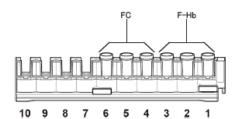

4 Touch the {Set} tab.

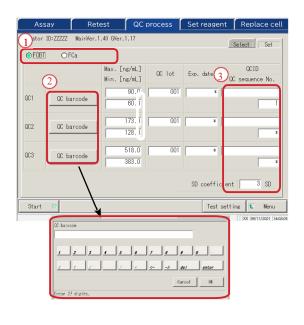


- 5 Set each item.
- ① Select the test item.
- ② Touch {QC barcode}. Enter the QC barcode that corresponds to the selected test item or QC1-3.
- For QC barcode entry, see page 50.
- ③ Enter the remaining settings (see the table below.).

(Note)

Be careful with the positions of the STD and QC racks and handling of QC1 - 3. In the following example, set as follows.

Pos.1 : [FOBT]-QC1 Pos.2 : [FOBT]-QC2 Pos.3 : [FOBT]-QC3 Pos.4 : [FCa]-QC1 Pos.5 : [FCa]-QC2 Pos.6 : [FCa]-QC3

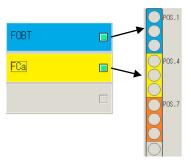

(Note)

Set the QC replication on the [Protocol settings] screen.

Page 217 "6.2.1 Sample/QC Protocol Settings"

(Note)

If the number of remaining cells is smaller than the number of QC process tests, a message indicating insufficient cells is displayed. Before performing the QC process, check the number of remaining cells.

2 Basic Operation {Set} tab


Item	Range/selection	Content
Test item		Selects the test item to be set.
QC barcode	27 digits	Enter the QC barcode here. The max. value/min. value, QC lot, and expiration date are automatically set, and they can also be entered and edited manually.
QCID	0 – 9, X, Y, Z	As the tested QC1-3 ID, it is output to the external media, printer, and the high-order system. Can be omitted.
QC sequence No.	1 – 99999, *	QC sequence No. When a numerical value is entered for QC1 and "*" is entered for QC2-3, a serial number from the QC1 value is automatically allocated every time the QC is measured. Ex) If QC1 = 3, QC2 = *, and QC3 = *, the tested QC \rightarrow QC sequence No. QC1 \rightarrow 3 QC2 \rightarrow 4 QC3 \rightarrow 5 QC1 \rightarrow 6 QC2 \rightarrow 7 QC3 \rightarrow 8 When numerical values are entered for each of QC1-3, the sequence number is allocated in a fixed way every time QC is measured. Ex.) If QC1 = 3, QC2 = 10, QC3 = 15, the tested QC \rightarrow QC sequence No. QC1 \rightarrow 3 QC2 \rightarrow 10 QC3 \rightarrow 15 QC1 \rightarrow 3 QC2 \rightarrow 10 QC3 \rightarrow 15
SD coefficient □ SD	1-9	Sets the SD coefficient used for the calculation of X-R control drawing for the accuracy control process.

(Note)

Expired QCs will not be analyzed.

Note

When the QC lot is *, the expiration date, max./min. control limit values are hidden, and the corresponding QC is not tested.

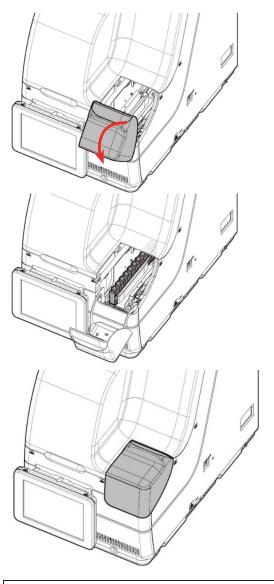

6 Open the rack lane cover.

7 Set the rack at lane 1 or lane 2 with rack position 1 innermost. Pay attention to set the rack properly.

8 Close the rack lane cover.




Request



When setting the rack in the lane, rack position 1 must be innermost.

Failure to observe this precaution may lead to trouble.

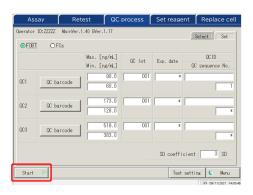

9 Touch {Start}.
When the pre-set priming completes, dispensing of the sample starts.

2.5.13 Additionally Setting the Sample

While executing the initial analysis, the procedure to additionally setting the rack is explained.

\triangle

Warning




• Wear protective gears (gloves, goggles, etc.) when handling the samples.


Failure to observe this precaution may lead to infection.

The lane status consists of three types: "In use," "Replace," and "Waiting." (See page 74.) For lanes displaying something other than "In use" while testing, new setting or replacement of the rack is possible.

However, after an "In use" rack is discharged to the rack lane, if a new rack and/or replacement is performed while there is no rack "In use," the test of the set rack does not start until a new test starts.

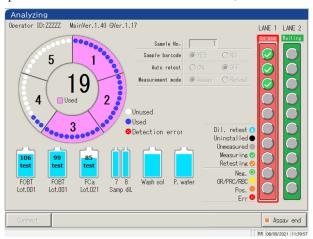

1 Check that there is a rack displaying "In use" in red in the lane information.

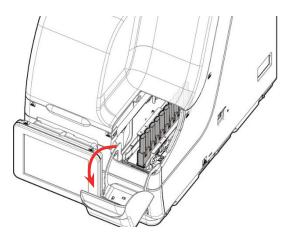


2 Open the rack lane cover.




In the following cases, the rack lane cover is locked; therefore, inserting a new rack or replacement of a rack cannot be done:

- · Initializing
- · Priming before analysis
- Rack discharging from the transfer lane to the rack lane
- · Retest measuring
- · Auto retesting
- · QC processing
- CC creating

*For Retest, QC test, and CC create, the rack lane cover is unlocked and the rack can be removed from the lane when all dispensing processes are completed and the rack is discharged.

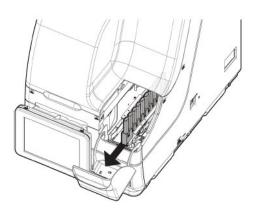

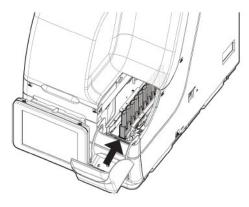

Remove the rack from the rack lane for which the lane information is "Replace" or "Waiting."
However, if no rack is set, unloading is not necessary.

4 Set the rack to be added to the rack lane. Be careful to set the rack properly.

Note When setting the rack, put the rack in with position 1 innermost, and push it in until the rack contacts the innermost section of the system.

Note Replace the rack as quickly as possible.

Otherwise, dispensing could fail for some cells, and efficient cell use is hindered.

For lanes indicated as "In use," do not set a

rack.


 \triangle

Required



Set the rack in the system with rack position No. 1 at the innermost end.

Failure to observe this precaution may lead to troubles.

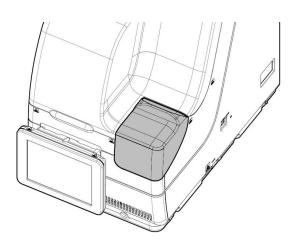

5 Close the rack lane cover.


*Dispensing starts automatically.

2 Basic Operation

2.5.14 Checking the Analysis Information


■ Checking the reagent volume

On the [Assay] screen, the [Retest] screen, the [QC process] screen, and the [Analyzing] screen, remaining volumes of the washing solution, purified water, and regent are displayed.

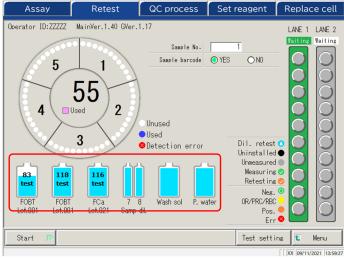



For the diluent, washing solution, and purified water, the remaining volumes are displayed using a bottle graphic.

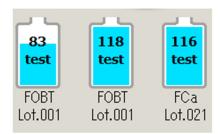


For the R1 and R2 reagents, the remaining volume is displayed with bottle graphics. In addition, the test item, lot number, and available test count are displayed.


* On the [Set reagent] screen, only reagents for which pairing is established are displayed.

Lot.XXX: Lot numbers of R1 and R2 reagents (3 digits)


XX test: Process-able test count

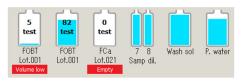

If an error occurs, the error information is displayed under the bottle.

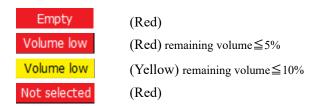

Rack info is displayed on the [Assay] screen, [Retest] screen, and [Analyzing] screen. The conditions of the sample rack and the samples in the system can be checked.

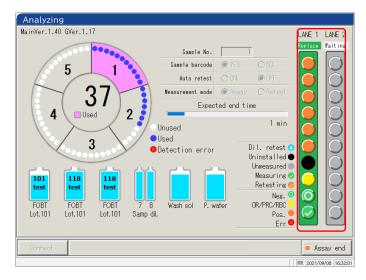

<Sample information>

The request condition and analysis result of each sample.

(Light blue)	Re-specification of diluting	Dilute retest is set. Displayed only on the [Retest] screen.
(Black)	Not set	Sample not set.
(Gray)	Not measured	Not measured
(Green)	Measuring	Currently measuring. Displayed only on the [Analyzing] screen.
(Orange)	Remeasurement	Currently retesting. Displays only on the [Analyzing] screen.
(Green)	Normal	Measurement result: Negative
(Yellow)	OR/PRC/RBC	Measurement result, OR/PRC/RBC.
(Orange)	Positive	Measurement result: Positive
(Red)	Error	Error occurred during the measurement, and the result was not output.

<Rack info>

Request condition and analysis result of Lanes 1 and 2 rack status are displayed.

	Waiting	
Waiting	(Gray)	The rack is not set or the state before the sample is
		dispensed.
Waiting	(Green)	Rack replacement is available
In use	In use	
	(Red)	Dispensing the sample
Replace	Replace	
	(Green)	Dispensing the sample on the rack completed. Rack
		replacement is available.

2.5.15 Ending the Analysis


When testing all of the set samples is finished, the analysis automatically ends. This section describes other ways to finish the analysis.

Normal ending of the analysis

After finishing the tests of all dispensed samples, the test ends.

Samples not dispensed when the normal end is selected remain as "Not measured.".

When selecting "Abort" for the analysis, see "■ Abort analysis."

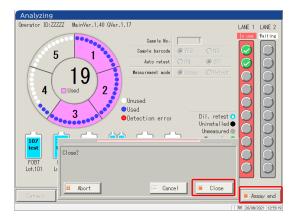


- 1 Touch {Assay end}.
- Touch {Close}.* Results of the measured samples are printed.

{Abort}: Ends the testing in the middle of the process.

{Cancel}: Closes the dialog box. {Close}: Ends the testing.

Abort analysisThe analysis is aborted.

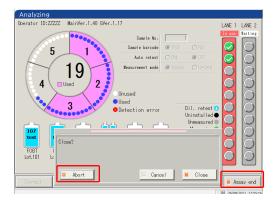

- 1 Touch {Assay end}.
- Touch {Abort}.* Results for the samples are discarded.

{Abort}: Ends the test in the middle of the process.

{Cancel}: Closes the dialog box.


{Close}: Ends the test.

Basic

Operation

■ Aborting the testing


If an error occurs during testing, a popup window for aborting opens. The testing of all dispensed samples is finished, and then the test ends.

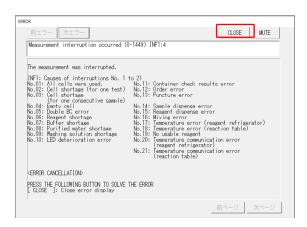

1 Close the screen with {Close}.

Resuming the test

Only when the testing was ended or ended by aborting the sample measurement, pressing [Start] on the [Assay] screen resumes the testing from the position after the measurement result output. When the Start button is pressed, the following dialog box opens. Starting a new test or resuming the testing can be selected.

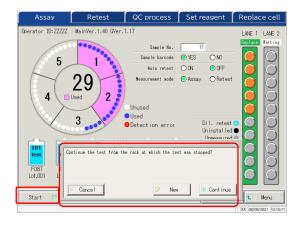

1 Touch {Start}.

{Cancel}: Closes the dialog box.

{New}: Begins testing from Lane 1 Position 1.

* Pierced samples will be pierced again.

{Continue}: Continues testing from the position next to the one for which the measurement results have been output.

* Pierced samples are not pierced again.



- When a discharged rack is removed, the rack info is cleared. In that case, resuming the testing is not possible.
- · When the test mode is changed before resuming, resuming the testing is not possible.
- · When automatic retest is set to "Yes," resuming the testing is not possible.

Test finish

2.6 Inspection and Cleaning After Use

2.6.1 Removing the Racks

The racks are removed after the testing is finished.



Warning




• Wear protective gear (gloves, goggles, etc.) when handling the samples.

Failure to observe this precaution may lead to infection.

Biohazard

1 Open the rack lane cover.

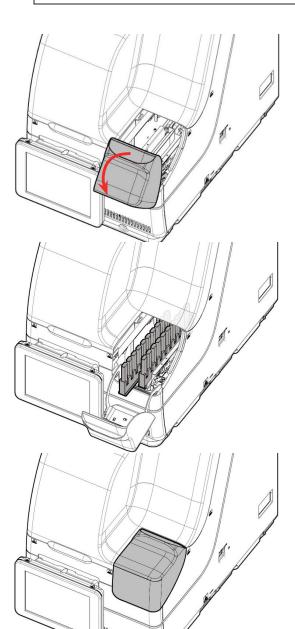

2 Remove the racks which testing is finished.

3 Close the rack lane cover.

2.6.2 Replacing Cells

The used test cells are replaced.


\triangle

Warning




· Wear protective gear (gloves, goggles, etc.) when handling the used cells.


Failure to observe this precaution may lead to infection.

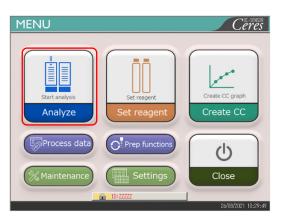

1 Touch {Analyze}.

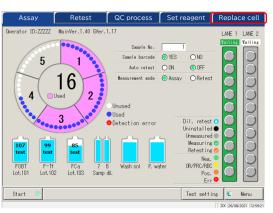


Touch the {Replace cell} tab.
The [Replace cell] screen opens.

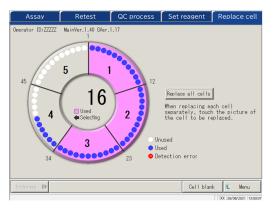

{Cell blank}: Displays the cell blank value of the cell which is currently set.

Page 83


{Menu}: Returns to the [MENU] screen.

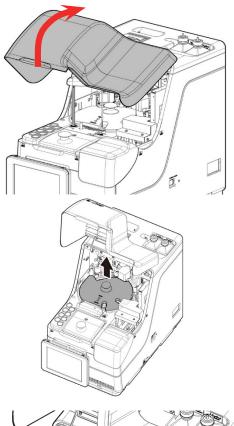



3 Open the safety guard.

Basic Operation

4 Remove the Light-shielding cover.

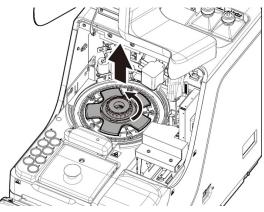


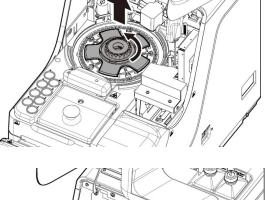

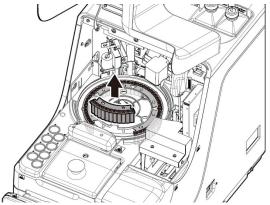

5 Remove the cell retainer.


Rotate the center handle counterclockwise and align the ▼ mark with the unlock mark

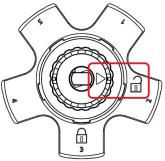

6 Remove the used cell and replace it.

<Replace one by one>

① Remove the cell and replace it with a new one.

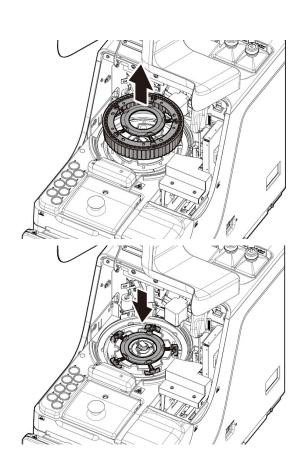

- <Replacing five cells in a batch>
- ① Pull up the cell batch replacement tool. All five cells can be removed with the tool.
- ② Remove the cells from the cell batch replacement tool.
- ③ Attach the cell batch replacement tool to the system.
- 4 Attach five new cells to the system.


Request



Required


• When setting a test cell, check that the cell legs are not floating. After setting the test cell, rotate the reaction table by hand. Check that the cell and the reaction chamber are not contacting. If they are contacting, an abnormal sound can be heard.

After setting the test cells, if any abnormal sound occurs, immediately stop the operation and check the test cell condition. The test cell might not be properly set.

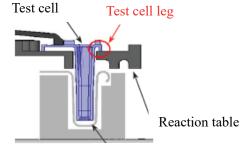


This can cause trouble.


Reaction chamber

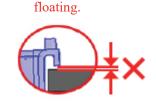


Test cell leg is not

80

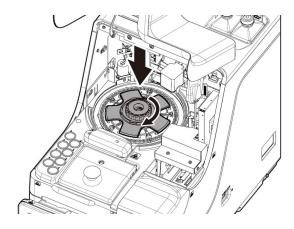


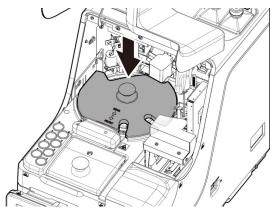

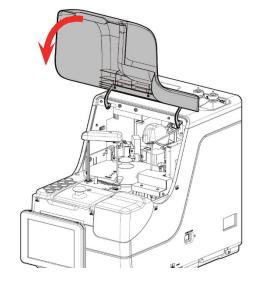

7 Return the cell retainer.

Turn the center handle counterclockwise and align the ▼ mark with the lock mark.

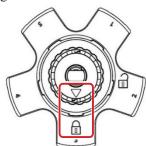


8 Replace the Light-shielding cover.


9 Close the safety guard.

81

Basic Operation Operation

16 Replace all cells 3 Exchanged

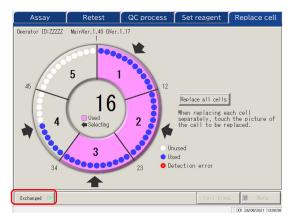

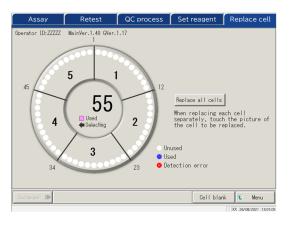


- 10 Touch the replaced cells. If all cells have been replaced, touch the {Replace all cells} button. The arrows ▶ are displayed.
- * Touching the selected cell again cancels the cell
- * Touching {Cancel all cells} cancels selection of all the
- * Buttons at the lower section of the screen cannot be operated while any cells are selected. (Buttons become inactive.)
- 11 Touch {Exchanged}.
- * The cell blank test starts.

- 12 Cell replacement is completed.
- 13 Discard the used cells.


Checking the cell blank value

Testing the cells without dispensing is called cell blank testing.


This value is used for the calculation of measured data. In addition, it is also used to judge whether or not the cells can be used without problems. This section describes the way to check the cell blank values.

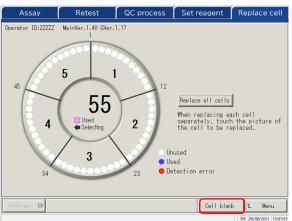

On the [Replace cell] screen, touch {Cell blank}.

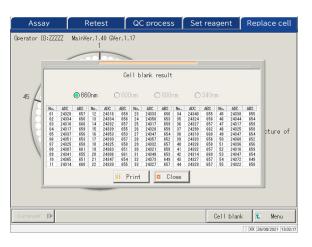

- 2 Check that no errors are output. The cells with an error is displayed in red.
 - *Cells with an error cannot be used for testing. If there are many error cells, replace the cells again.
- 660 nm / 600 nm / 800 nm / 340 nm : Switches the cell blank test wavelength.
- * Usually, it is checked at 660 nm. {Print}: Prints the test result.

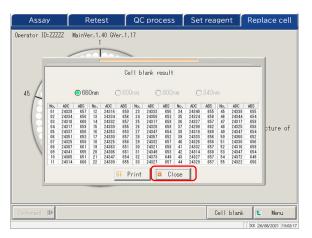

3 {Close} closes the Cell blank result screen.

2.6.3 Cleaning Various Sections

On the [MENU] screen, touching {Maintenance} opens the [Maintenance] screen.

"Inspection and cleaning" items are displayed. Follow the instructions on the screen to clean each section.




Warning




· Wear protective gear (gloves, goggles, etc.) when cleaning the system.

Failure to observe this precaution may lead to infection.

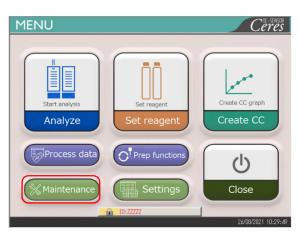

1 Touch {Maintenance}.

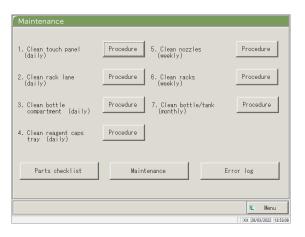

2 The [Maintenance] screen opens.


Clean each section.

Sections to be cleaned are as follows.

- 1. Touch panel (daily)
- 2. Rack lane (daily)
- 3. Washing solution and purified water setting positions (daily)
- 4. Reagent cap space (daily)
- 5. Nozzle (weekly)
- 6. Rack (weekly)
- 7. Clean bottle/tank (monthly)

For details, see page 170 "5.1 Inspection and Maintenance"

Note When there is a part for which the replacement date has passed, a message appears explaining this, with instructing to check the Parts checklist screen.

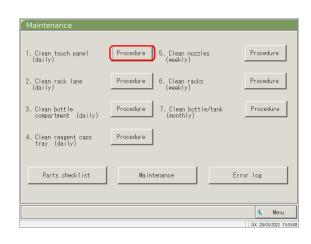

3 If the cleaning procedure is unknown, touch each {Procedure}. The cleaning procedure is displayed. After cleaning completes, touch {Close} to close the cleaning procedure screen.

2 Basic Operation

{Parts checklist}: List of parts required for inspection and replacement is displayed.

- Page 86 "2.6.4 Opening Parts Checklist"
- {Maintenance}: Adjust the system and/or checks the performance (for service personnel)


{Error log}: Display the error log. A maximum of 1000 error logs can be checked.

Page 88 "2.6.5 Opening the Error Log"


{Menu}: Return to the [MENU] screen.

2.6.4 Opening Parts Checklist


On the [Parts checklist] screen, part names, last replaced date, and number of months used (maximum use count) are displayed.

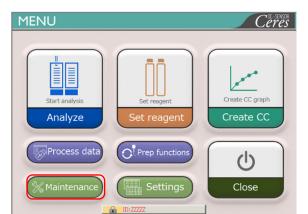

1 Touch {Maintenance}.

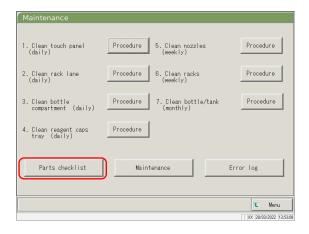


2 Touch {Parts checklist}.

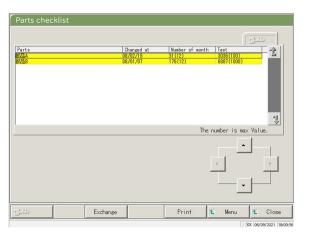


Check if there are any parts which require replacement. Displayed content as the table on the next page.

If there are any parts or which the number of month or tests has exceeded the max. value, the line becomes yellow.

No.	Item	Content	Remarks
1	Part	Name of the part	Max. of 15 digits of half size
			alphanumeric characters or 7 digits
			of full size characters
2	Changed at	Date when the part was replaced (YY/MM/DD)	
3	Number of months	Number of months from the latest replacement to the	
		current date	
	()	Max. number of months	
4	Test	Number of tests from the latest	
		replacement to the current date	
	()	Max. number of use	

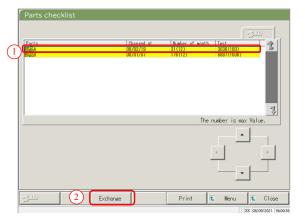

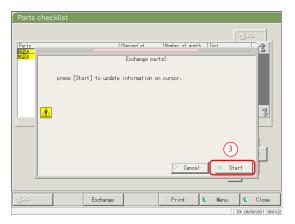

- 4 Replace the parts.
- ① Select the part (yellow line) to be replaced.
- ② Replace the part.
- ③ Touch {Exchange}.
 - * A dialog box asking "Exchange parts?" opens.

{Exchange}: Touch this button after replacing the part.

{Print}: Print a list of parts for inspection and replacement.

{Menu}: Return to the [MENU] screen. {Close}: Return to the [Maintenance] screen.

- 4 Touch {Start}.
 - * The latest replacement date is updated to today.
 - * The number of months and use count are reset to "0."
 - * Closes the dialog box.


{Close}: Close the dialog box.

 $\{Start\}\colon Update\ the\ parts\ replacement\ information.$


Note Parts replacement is also available for lines not colored in yellow.

2.6.5 Opening the Error Log


On the [Error log] screen, the date and time of errors, error codes, and error names (content) are displayed. A maximum of 20 errors can be displayed on one screen. A maximum of 1000 errors can be displayed in the entire list. If there are more than 20 errors, use the scroll bar to move through the list.

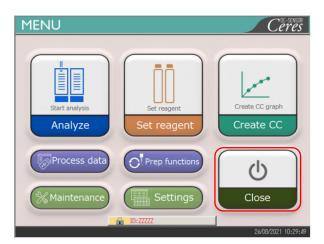

1 Touch {Maintenance}.

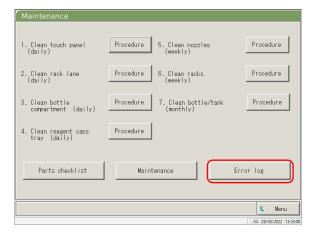

2 Touch {Error log}.

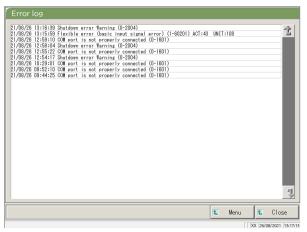


3 Check the error information.

{Menu}: Return to the [MENU] screen.


 $\{Close\}\colon Return\ to\ the\ [Maintenance]\ screen.$


2.7 Shutting Down the System

To properly shut down the system, use the "shut down mode." The System switch turns off automatically (the primary power switch does not turn off.)

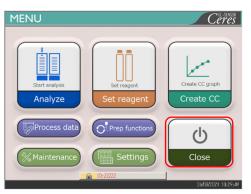


1 Touch {Close}.

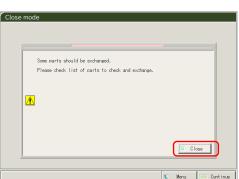


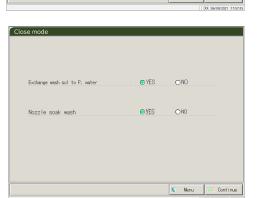

- The [Close mode] screen opens.
 If there are any expired items, a
 message appears saying "Some parts
 should be exchanged. Please check the
 list of parts to exchange" is displayed.
 - Touch {Close} and then {Menu} to return to the [MENU] screen.
- Page 86 "2.6.4 Opening Parts Checklist"
- 3 If there are no expired parts, select an operation to be conducted at the end of the operation.

Item	Range/selection	Content
Exchange wash sol to P.	Yes: Replace the purified water.	
water	No:	Do not replace the purified water.
Nozzle soak wash	Yes:	Soak for storage.
	No:	Do not soak for storage.

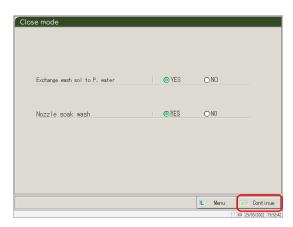

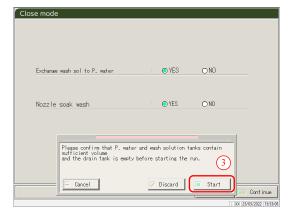

4 Touch {Continue}.

- 5 Executes the Close mode.
 - ① Check that the purified water bottle and the washing solution bottle are not empty.
 - ② Check that the "drain tank" is not full.
 - ③ Touch {Start}.

{Cancel}: Close the dialog box.

{Discard}: Abort the closing mode and returns to the [MENU] screen.

{Start}: Execute the Close mode.

- Replace the washing solution pipe inside the purified water bottle (when "Exchange buffer and wash sol to P. water" is set to "Yes").
 - ① Remove the pipes connected to the washing solution bottle.
 - ② Connect the pipe in ① to the container filled with purified water.
- 7 The nozzle is stored with the nozzle tip soaking (when "Yes" is selected for "Nozzle/cell soak wash")
 - * The System switch shuts off.
 - * The setting which was executed in the Close mode is saved.
- 8 The system power turns off.

2.8 Processing the Drain Tank

Disconnect the hose attached to the drain tank and drain the waste liquid from inside the tank.




Warning



• Wear protective gear (gloves, goggles, etc.) when handling the drain tank.

Failure to observe this precaution may lead to infection.

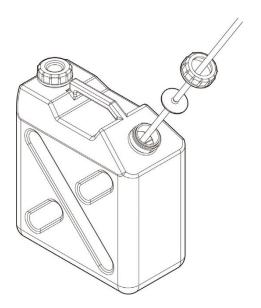

1 Remove the hose from the drain tank.

- 2 Drain the waste liquid from the drain tank.
- 3 Connect the hose to the drain tank.

2 Basic Operation

Chapter 3 Applied Operations

- 3.1 Processing Test Data
- 3.2 Editing Sample IDs
- 3.3 Replication
- 3.4 Accuracy Control
- 3.5 LOG IN/LOG OUT



Chapter 3 Applied Operations

3.1 Processing Test Data

On the [Process data] screen, test data can be selected, edited, and deleted.



3

Applied

Operations

- Test data are saved for each item.
- Samples for up to 5000 tests can be saved on the hard disk for each test item. If the number of tests exceeds 5000, the old data are overwritten.
- Up to 1000 tests of the QC samples for each control can be saved on the hard disk. If the number of tests exceeds 1,000, the old data are overwritten.


3.1.1 Opening the [Process Data] Screen

The [Process data] screen can be opened by touching {Process data} - -Process data} on the [MENU] screen.

If a password has been set in Config., the password will be required.





1 Touch {Process data}.

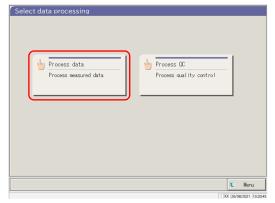


2 Touch {Process data}.



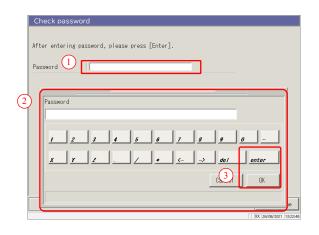

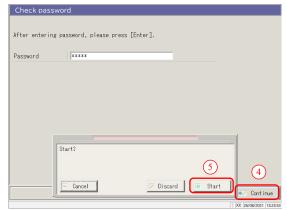

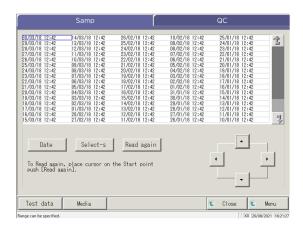

- 2 If a password has been registered in [System settings] -Config.], or if {Process data} is used for the first time with the currently logged in account, enter the password.
 - ① Touch the entry field for the password.
 - ② Enter a password from the keyboard.
 - ③ Touch {OK} or {enter}. The dialog closes.
 - 4 Touch {Continue}.
 - ⑤ Touch {Start}.

{Cancel} : Closes the dialog box. {Discard} : Returns to the [Select data processing] screen.

- 3 [Process data] screen opens.
 - * {Samp} tab is selected.
 - * The list of test date and time is displayed.

 $\overline{\text{Note}}$ The test date and time are when the system was started up.


If no data was processed, the date and time are not displayed.

3.1.2 Opening the [Test Data] Screen


The test data are selected by switching between the {Samp} and {QC} tabs on the [Process data] screen. When {Test data} is touched after specifying the test date, the specified data is displayed. (the [Test data] screen is different when the {Samp} tab is selected and when the {QC} tab is selected.)

- Note The displayed date and time are the ones when the System switch is turned on (time when the system is started up).
- Note The data measured until the System switch is turned off are saved on the hard disk.

 When displaying the [Process data] screen, see page 94 "3.1.1 Opening the [Process Data] Screen."

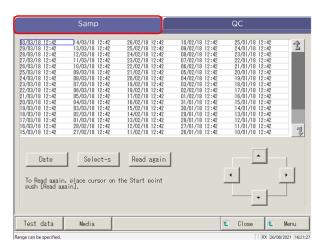

Touch the displayed test data tab. {Samp} tab {QC} tab

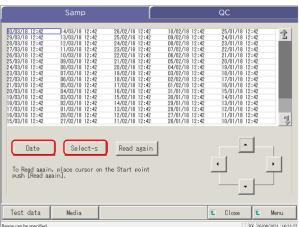


- 2 Specify the test data by date and time or by range.
- <When specifying by the test date and time>Touch {Date} to specify the test date.
- Page 100 "3.1.3 Specifying a {Date} for the Test Data" 2
- <When specifying by range>


Touch {Select-s} and specify the test date range.

Page 102 "3.1.4 Selecting {Select-s} for Test Data"


Note When {Date} or {Select-s} is not selected, all displayed test data becomes the target of data processing. (max. 5,000 data).

3.1 Processing Test Data

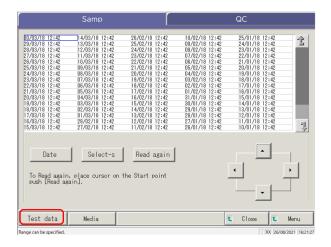

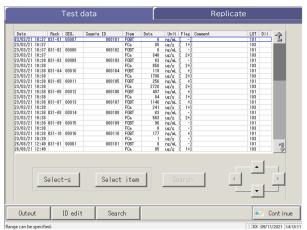

When the {Samp} tab is selected

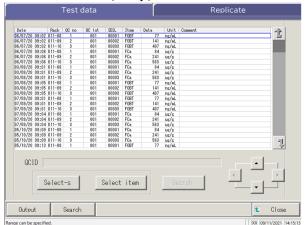

When the {QC} tab is selected

3 Touch {Test data}.

{Close} : Returns to the [Select data

processing] screen.

{Menu} : Returns to the [MENU] screen.

4 The test data is displayed.

3 Applied Operations 3

Applied Operations

Contents displayed on the [Test data] screen

Display item	Content	aj serecii	Remarks
Date	Date and time of the sample test		
Rack	Rack No Tack Position No.		
SEQ.	Sample sequ	uence number	
Sample ID	Barcode on	the sample bottle	
Item	Test item		
Data	Test result (concentration)	
	Units of test data		
Units	FOBT : ng/1	mL	
	FCa: µg/g		
Flag	-, +, 1+, 2+, 3+		
	Error inforn	nation (excluding read errors)	
	UR	Under range	Outputs a blank for the
Comment			measured data and the
			flag
	OR	Over range	Outputs a flag only
	PRC	Prozone	Outputs a flag only
LOT	Lot of the sample used for the test		
	Dilution information		
	Space	No dilution (Analyze, retest)	
	A	No dilution (retest)	
D.1	A10	Dilution by a factor of 10	
Dil		(retest)	
	A20	Dilution by a factor of 20	
		(retest)	
	A100	Dilution by a factor of 100	
		(retest)	
	A200	Dilution by a factor of 200	
		(retest)	
	A400	Dilution by a factor of 400	
		(retest)	

Content of the [Test data] screen (QC)


Display item	Content		Remarks	
Date and time	Date and t	ime of the sample test		
RACK	Rack No	Rack Position No.		
QC No.	QC No.1 -	QC No.1		
QC lot	QC lot (3	QC lot (3 digits)		
Seq.	QC seque	QC sequence No.		
Item name	Test item	Test item		
Measured data	Test result	(concentration)		
Units	Units of te FOBT: ng FCa: μg/g	/mL		
Error information (excluding the read error		rmation (excluding the read errors)		
Comment	UR	Under range		
	OR	Over range		
	PRC	Prozone		
QC ID	ID of QC	sample at the cursor position		

3.1.3 Specifying a {Date} for the Test Data

The date of the test data to be displayed is specified on the [Process data] screen.


- Note If a part of the year, month, and date is omitted when entering the test date, a search is conducted. The latest test data that matches the entered conditions are extracted.

 Next page 3 (Example)
- Note If a part of the year, month, and date is omitted when entering the test date, a search is conducted. The latest test data that matches the entered conditions are extracted.

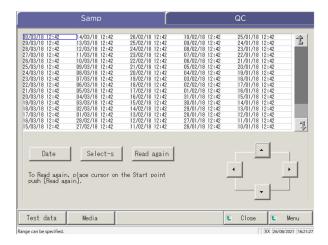



Touch the tab of the test data to be displayed.


{Samp} tab {QC} tab

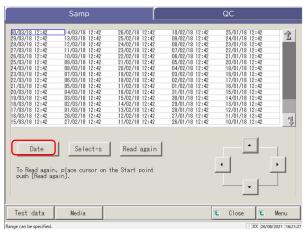

2 Touch {Date}.

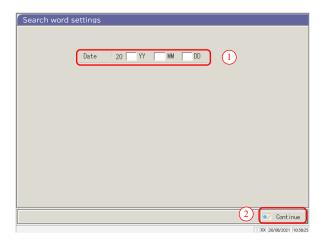

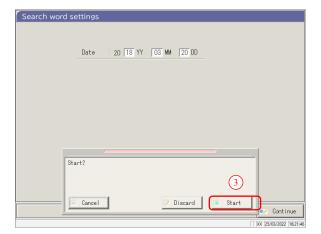

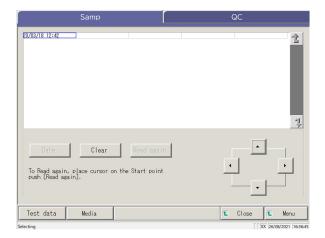

3 Specify the Date.

① Touch the entry field and enter the year, month, and date.

If a part of the year, month, and date is omitted, a search is conducted. The latest test data that matches the entered conditions are extracted.

(Example) If October 5 is entered while omitting the year, only data of the current year are displayed. For example, if the current year is 2020, the displayed data will be as follows.

2020/10/05 10:10 (displayed) 2020/10/05 10:15 (displayed) 2020/10/05 10:20 (displayed)

.

2019/10/05 14:25 (not displayed) 2019/10/05 14:30 (not displayed) 2019/10/05 14:35 (not displayed)

② Touch {Continue}.

③ Touch {Start}.

{Cancel} : Closes the dialog box.

{Discard} : Returns to the [Select data processing]

screen.

 $\{Start\}$: The data is searched within the

specified date range.

4 The data of the specified test date are displayed.

Note To re-specify the test date, touch {Clear}.


All tests are displayed and the displayed condition returns to the condition when no range is specified.

3.1.4 Selecting {Select-s} for Test Data

The test data to be displayed are specified within a test date range (starting point and ending point). The range is specified by touching test dates on the [Process data] screen or by operating the cursor buttons.


When the starting point is specified and {Select-s} is touched, the test data starting point is set. Then, when the ending point is specified and {Select} is touched, the test data ending point is set. When the starting and ending points are finalized, the test data in-between becomes the range.

- (Note) When opening the [Process data] screen, see "[3.1.1 Opening the [Process Data] Screen" on page 94.
- Note When not specifying the range, all data (max. 5000) becomes the target of processing.

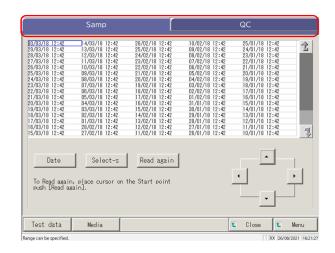

Touch the tab of test data to be displayed.


{Samp} tab {QC} tab

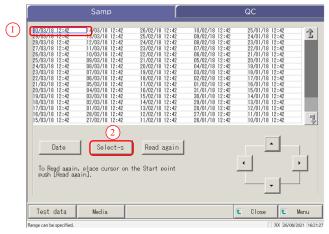


- 2 Specify the test date of the starting point.
 - ① Touch the date to be used as the starting point.
 - ② Touch {Select-s}.
 - *The test date of the starting point has been specified. (The letters turn blue.)
 - * When the starting point is finalized, {Select-s} changes to {Select}.

Note The starting and ending points can also be specified using the cursor buttons.



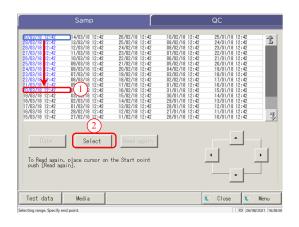

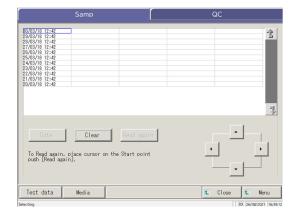

- 3 Specify the test date and time of the ending point.
 - ① Touch the date and time to be used as the ending point.
 - ② Touch {Select}.
 - * The test date of the end point is finalized.
 - * When the end point is finalized, {Select} changes to {Clear}.

{Clear}: Clears the range specification.

The test data within the specified range are displayed.

Note

- When the same dates are specified for the starting point and ending point, only one test data is specified.
- To respecify the range, touch {Clear}.
 All test data are displayed, and no range is specified.


3.1.5 Reading "Test Data" Not Displayed in the List (Re-reading)

The maximum number of sample data sets that can be displayed and edited in the list is 5,000.


To edit sample data not displayed in the list, it is necessary to read the data.

Specify the date to use as a starting point. Select a former date or a later date to read the data to be edited.

Note When opening the [Process data] screen, see "3.1.1 Opening the [Process Data] Screen" on page 94.

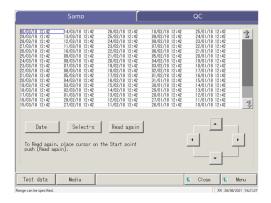

1 {Samp} tab is displayed.

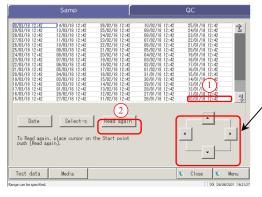


- The test data is read by specifying the starting point.
- ① Touch the test date and time to be used as a starting point.
 - (Selecting is also possible using the cursor buttons at the bottom right of the screen.)
- ② Touch {Read again}.
- ③ Select the reading method (see the following table).
- 4 Touch {Start}.
 - * Starts reading the test data.

{Cancel}: Closes the dialog box.

{Close}: Returns to the [Select data processing] screen.
Or returns to the [Analyzing] screen.

(When the {Test data} button is touched on the AssayMainView screen)

{Menu}: Returns to the [MENU] screen.

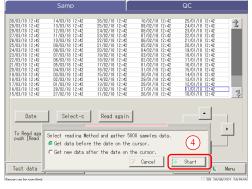


Select	Content
 Get data before the date on the cursor. 	The data is acquired before the date of the cursor
	position (date and time at the cursor position are not
	included).
Get new data after the date on the	Reads the data before the date and time of the
cursor.	cursor position (date and time at the cursor
	position are not included).

Reading "Test Data" in External Media (Switching the External Media) 3.1.6

Test data saved on external media can be read and displayed on the screen. This is not possible for QC sample test data.


When displaying the [Process data] screen, see "3.1.1 Opening the [Process Data] Screen" on page 94

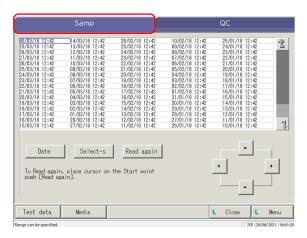

1 Touch the {Samp} tab.



- Read again Test data
- Touch {Media}. {Media} changes to {HD change}.
 - * The data is read from the external media.
 - * The test data saved on the external media are displayed (max. 5000).
 - * The data are displayed from the latest date back.

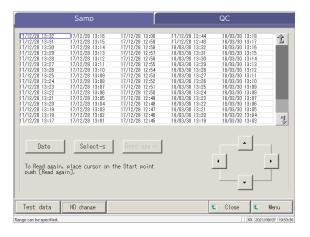



{HD change}: Returns to the display of test data saved on the system hard disk.


The test data cannot be searched by rack position No. (Note)

(Note) When displaying the [Process data] screen, see "3.1.1 Opening the [Process Data] Screen" on page 94.

Select-s Read again To Read again, place cursor on the Start point push [Read again].

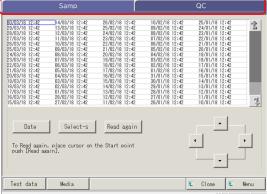

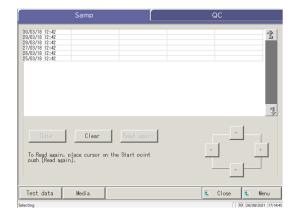

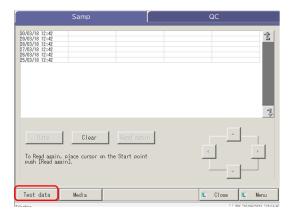



1 Touch the test data tab to be displayed.

{Samp} tab {QC} tab

- 2 Specify the test data by test date or range.
- <When specifying by test date> Touch $\{Date\}$ to specify the test date. Page 100 "3.1.3 Specifying a {Date} for the Test Data" 2
- <When specifying by range> Touch {Select-s} to specify the test date range. Page 102 "3.1.4 Selecting {Select-s} for Test Data" 2
- Touch {Test data}.

3 Applied Operations

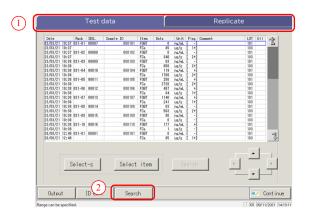


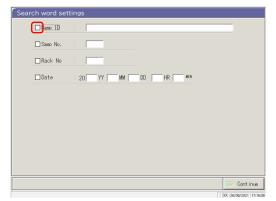



- 4 Search the target sample.
 - ① Touch the corresponding tab.
 - ② Touch [Search].

- 5 Select a keyword type to be searched.
 - □ Samp ID (only for the {Samp} tab)
 - □ Samp No.
 - □ Rack No.
 - □ Date

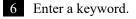

- ① Touch the entry field next to a keyword.
- ② Enter a value using the keyboard.
- $\ensuremath{\ensuremath{\Im}}$ Touch $\ensuremath{\ensuremath{\{OK\}}}$ or $\ensuremath{\ensuremath{\{enter\}}}.$

{A/a}: Switches between large/small letters.

{Sign}: Enters symbols.


(Note) When the test date is selected, the year, month, date, hour, or minute must be entered.



7 The search condition has been entered in an entry field.


3 Applied Operations

3.1 Processing Test Data

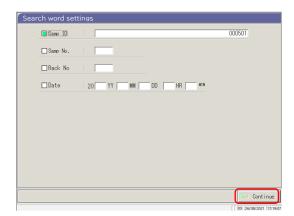


8 Touch {Continue}.



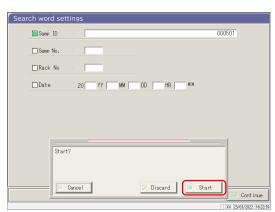

9 Touch {Start}.

* Searching starts.

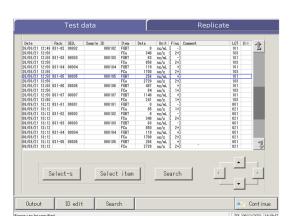


10 The search results are displayed.

{Search}: Continues searching.

3.1.8 **Outputting Test Data**

The specified test data is output to the selected destination.


- Prints the test data
- Saves the test data on external media.
- Outputs the test data to the host computer.

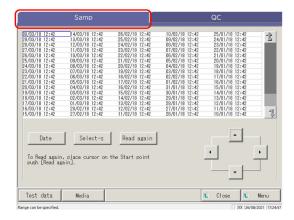

Touch the tab for the test data to be output (printed).

{Samp} tab



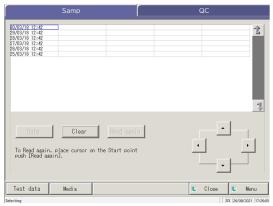

- 2 Specify the test data by date or range.
- <When specifying by test date>

Touch {Date} to specify the test date.


Page 100 "3.1.3 Specifying a {Date} for the Test Data" 2

<When specifying by range>


Touch {Select-s} to specify the test date range.

Page 102 "3.1.4 Selecting {Select-s} for Test Data" 2

3 Touch {Test data}.

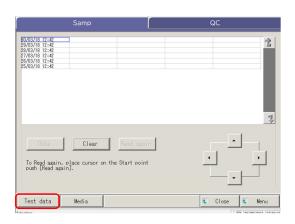

3.1 Processing Test Data

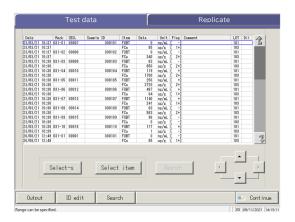


4 Specify the output range.


The procedure is the same as for the screen for selecting data.

Page 102 "3.1.4 Selecting {Selects} for Test Data"

3 Applied Operations

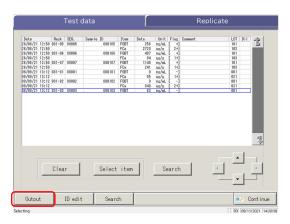

5 Touch {Output}.

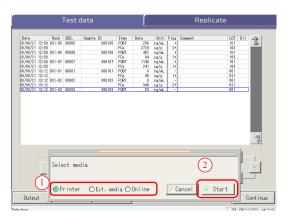

- 6 Outputs the test data.
 - ① Select the destination.
 - Printer
 - o Ext. media
 - o Online
 - ② Touch {Start}.

{Cancel}: Closes the dialog box.

(Note) If an error occurs in the test result communication, selecting "Network" in the [Select media] dialog is unavailable.




Request



If the external media becomes full in the course of outputting the data, replace it with another external media and touch {Retry}.

3.1 Processing Test Data

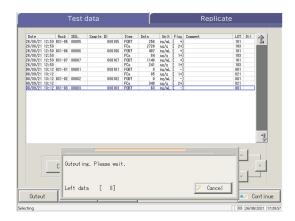

- * During output, the [Output] dialog opens.
- * When the output completes, the [Output] dialog closes.

3 Applied Operations


3.2 Editing a Sample ID

The sample ID of the data selected on the [Test data] screen can be edited.


Note In [System Settings] - -Samp barcode settings] screen - -Sample barcode], if [Duplicated check] is set to "YES," a redundancy check on the Sample IDs is conducted.

However, if the sample ID of Retest data is edited, the redundancy check is not conducted.

3 Applied Operations

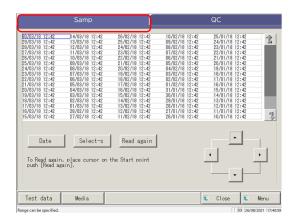

1 Touch the {Samp} tab.

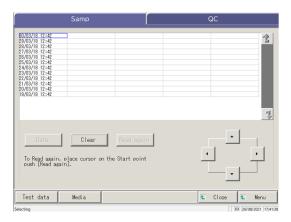


2 Specify the test data by date or range. <When specifying by test date> Touch {Date} to specify the test date.

Page 100 "3.1.3 Specifying a {Date} for the Test Data" 2


<When specifying by range>
Touch {Select-s} to specify the test date range.

Page 102 "3.1.4 Selecting {Selects} for Test Data" 2

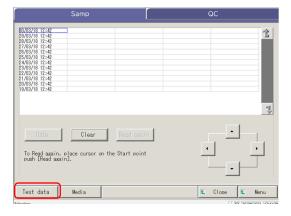


3 Touch {Test data}.



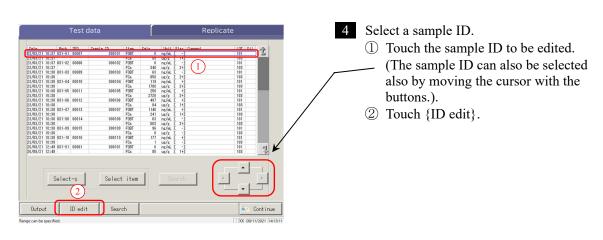

Note When the cursor is on a sample ID with 16 or more digits in the 2D code, the ID edit function becomes unavailable.


([ID edit] becomes inactive.)

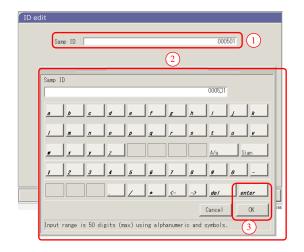

- 5 Enter a sample ID.
 - ① Touch the entry field for the sample ID. The Samp ID dialog opens.
 - ② Enter a sample ID using the keyboard.

 $\{A/a\} \hspace{1cm} : Switches \ between \ large/small \ letters.$

 $\{Sign\} \qquad : Enter\ symbols.$

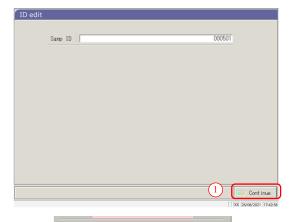

- 6 Update the sample ID.
 - ① Touch {Continue}.
 - ② Touch {Start} on the dialog box.
 - * The edited sample ID is recorded.

{Cancel}: Closes the dialog box.

{Discard}: Aborts editing of the sample ID, and returns to the [Test data] screen.

- Cancel

Applied

Operations


3.3 Replication

The replication data of the test data specified on the [Test data] screen is displayed.


3.3.1 Displaying the [Sample Replication] Screen (Sample)

The replication data of the sample is displayed.

Note If the range is not specified or the item is not selected, all data are displayed.

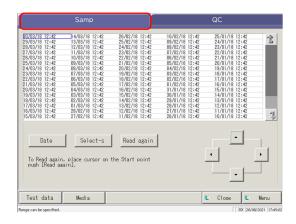



1 Touch the {Samp} tab.

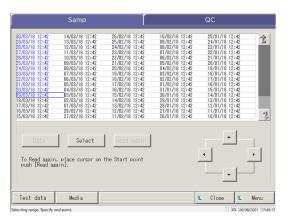


- 2 Specify the test data by date or range.
- When specifying by test date>
 Touch {Date} to specify the test date.
 Page 100 "3.1.3 Specifying a {Date} for the Test Data"
- <When specifying by range>
 Touch {Select-s} to specify the test date range.
 Page 102 "3.1.4 Selecting {Select-s} for Test Data"



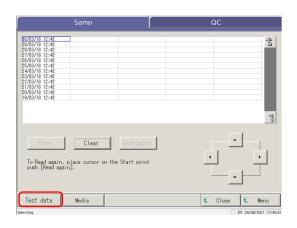

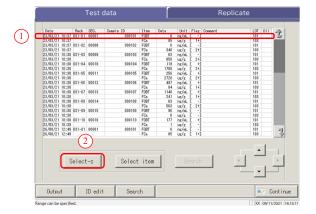

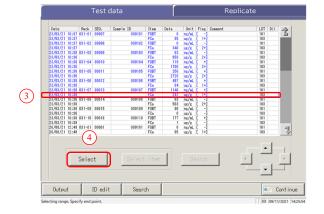



3 Touch {Test data}.


- 4 Specify the test data within the test date range.
 - ① Touch the test date of the starting point (the cursor buttons at the bottom right of the screen can also be used).
 - ② Touch {Select-s}.
 - * The line of the specified date changes to blue (starting point)
 - * {Select-s} changes to {Select}.
 - ③ Touch the test date of the ending point (the cursor buttons at the bottom right of the screen can also be used).
 - * The lines of the specified range changes to blue (ending point).
 - 4 Touch {Select}.
 - * The data in the specified range is displayed.
 - * When the ending point is finalized, {Select} changes to {Clear}.
- {Select-s}: Finalizes the starting point data for range specification.
- {Select}: Finalizes the ending point data for range specification.
- {Clear}: Clears the specified range.

3.3 Replication

3
Applied
Operations

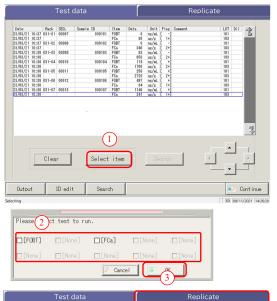

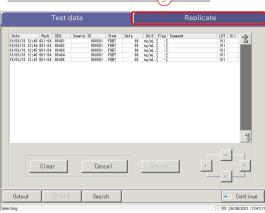

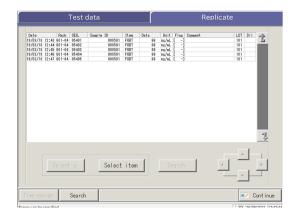

- 5 Select an item of test data.
 - ① Touch {Select item}.

 The item select dialog is displayed.
 - ② Select an item on the dialog.
 - ③ Touch {OK}.
 - * {Select item} changes to {Cancel}.

{Cancel}: Clears the range specification and item selection.

6 Touch the {Replicate} tab.

7 The [Replicate (sample)] screen opens.

{Time course}: Displays the time course data.

Page 131 "3.3.9 Displaying and Printing Time Course (Sample)"

{Search}: Searches the sample by the sample ID, sample No., rack No., and test date.

Page 126 "3.3.7 Searching Replication Data"

Content of the [Replicate] screen

Display item	Content		Remarks	
Date	Date and ti	me of the sample test		
Rack	Rack No	Rack Position No.		
SEQ.	Sample sec	quence No.		
Sample ID	Barcode or	the sample bottle		
Item	Test item			
Data	Test result	(concentration)		
Units	Units of m FOBT: ng/ FCa: μg/g	easured data mL		
Flag	-, +, 1+, 2+	-, 3+		
	Error infor	mation (excluding read errors)		
Comment	UR	Under range	Outputs a blank for the measured data and the flag (displayed for dilution retest)	
	OR	Over range	Outputs a flag only	
	PRC	Prozone	Outputs a flag only	
LOT	Lot of the	Lot of the sample used for the test.		
	Dilution in	formation		
	Space	No dilution (analyze, retest)		
	A	No dilution (retest)		
Dil	A10	Dilution by a factor of 10 (retest)		
	A20	Dilution by a factor of 20 (retest)		
	A100	Dilution by a factor of 100 (retest)		
	A200	Dilution by a factor of 200 (retest)		
	A400	Dilution by a factor of 400 (retest)		

3


Applied Operations

3.3.2 Displaying [Replicate (STD)] Screen


STD replication data is displayed.




1 Touch {Create CC}.

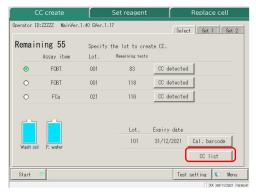


- 2 Touch {CC list}.
 - * The [CC list] screen opens.

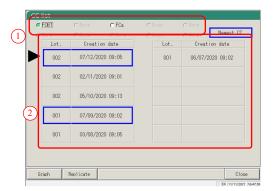


- 3 Select CC.
 - ① Select an item.
 - ② Touch the lot or creation date of the CC to be edited or recalculated.
 - *The selected CC is indicated by the mark.

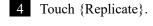

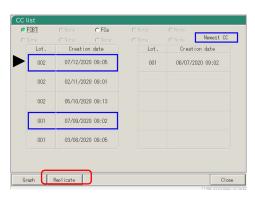



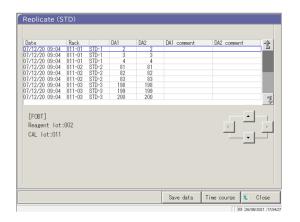

4 Data of the selected CC are displayed.

Content of the [Replicate (STD)] screen

Display item	Content	Remarks
Date	Date and time when STD was	
	measured	
Rack	Rack No Rack Position	
(Blank)	STD concentration (STD-1 - STD-6)	
DA1	ABS (A#) change amount A3-A1	Page 236 "1.2 DA Value
DA2	ABS (A#) change amount A2-A0	Calculation"
DA1 comment	DA1 comment	
DA2 comment	DA2 comment	

{Save to media}: Saves the replicate data to an external media.

Page 120 "3.3.3 Saving Replicate Data (STD) to External Media"

{Time course}: Opens the [Time course data] screen.

Page 132 "3.3.10 Displaying and Printing Time Course (STD)"

Page 135 "3.3.12 Changing the Time Course Range (STD)"

{Close}: Returns to the [Select data] screen.


3.3.3 Saving Replication Data (STD) to External Media

By opening the [Replicate (STD)] screen, the replication data displayed on the screen can be saved to external media.

The explanation starts from the condition at which the [Replicate (STD)] screen is displayed.

Page 118 "3.3.2 Displaying [Replicate (STD)] Screen"

Save data Time cours

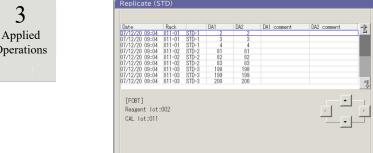

- On the [Replicate (STD)] screen, touch the {Save data} button.
 - * The STD replication data is saved on the external media.


If external media is not connected, the "External media is not connected" message will be displayed. Connect (Note) external media and touch {Retry}.

Applied Operations

3.3.4 Displaying the [Replicate (QC)] Screen

The QC replication data is displayed.

Note If a range is not specified, all data are displayed on the screen.

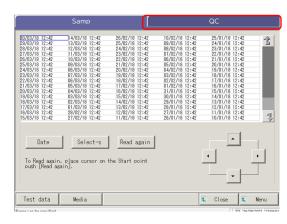

1 Touch the {QC} tab.



2 Specify the test data by date or range.

<When specifying by test date>

Touch {Date} to specify the test date.


Page 100 "3.1.3 Specifying a {Date} for the Test Data"

<When specifying by range>


Touch {Select-s} to specify the test date range.

Page 102 "Selecting {Select-s} for Test Data" 2

3 Touch {Test data}.

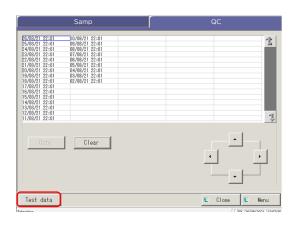


3.3 Replication

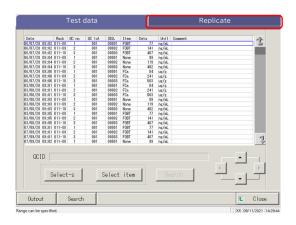

4 Touch the {Replicate} tab.

3 Applied Operations

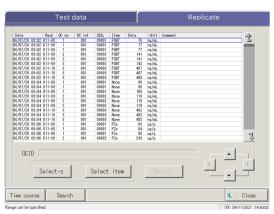

The [Replicate (QC)] screen is displayed.

* QC IDs are displayed.

{Close} : Returns to the process data start screen.

Content of the [Replicate (QC)] screen

Display item	Content	Remarks
Date	Date and time when QC was measured	
Rack	Rack No Rack Position No.	
QC no.	QC sample No.	
QC lot	QC sample lot number	
SEQ.	QC sequence number	
Item	Test item	
DATA	QC sample measured data	
Units	Units of measured data	
	FOBT: ng/mL	
	FCa: μg/g	
Comment	Barcode read error	
QC ID	QC ID of the QC sample at the cursor position	

Note When the QCID has 16 or more digits of 2D code, only 15 digits from the beginning are displayed.

{Time course}: Displays the time course data.

Page 131 "3.3.9 Displaying and Printing a Time Course (Sample)"

{Search} : Searches the sample by the sample ID, sample No., rack No., and test date.

Page 126 "3.3.7 Searching Replication Data"

3.3.5 Specifying {Select-s} for Replication Data

The replication data can be specified within a test date range (from starting point to ending point) by touching the test date or by operating the cursor buttons on the [Replicate] screen.


Specifying the starting point and touching {Select-s} sets the replication data starting point.

When the ending point is specified and {Select} is touched, the replication data ending point is set.

When both starting point and ending point are finalized, the replication data between the starting and ending points are specified.

The explanation starts from the condition at which the [Replicate] screen is displayed.

Page 114 "3.3.1 Displaying the [Sample Replication] Screen (Sample)"

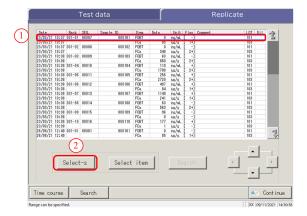

Test data

| Description | Section | Description | Descrip

- 1 Specify the <u>replication data</u> starting point.
 - ① Touch the date and time to be used as a starting point.
 - ② Touch {Select-s}.
 - * The starting point is set. (The line of specified date turns blue.)
 - * When the starting point is set, {Select-s} changes to {Select}.
- 2 Specify the <u>replication data</u> ending point.
 - ① Touch the date and time to be used as an ending point.
 - * The line of the specified range turns blue.
 - ② Touch {Select}.
 - * The data of the specified range is displayed.
 - * When the ending point is specified, {Select} changes to {Clear}.


{Select-s}: Finalizes the starting point of the range specification.

{Select}: Finalizes the ending point of the range specification.

{Clear}: Clears the specified range.

3.3 Replication

- Note When the dates and times specified as the starting point and ending point are the same, only one test data set is specified.
- Note To redo the specification of the replication data range, touch {Clear}. All replication data are displayed, and no range is specified.

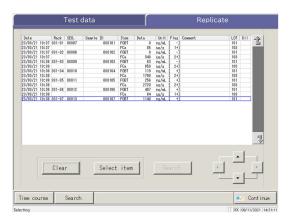


- The replication data is specified.
 - * The specified replication data is displayed.

3 Applied Operations }

To further limit the test items to be processed on the [Replicate] screen, select an item.

The explanation starts from the condition at which the [Replicate] screen is displayed.


Page 114 "3.3.1 Displaying the [Sample Replication] Screen (Sample)" 1 - 5

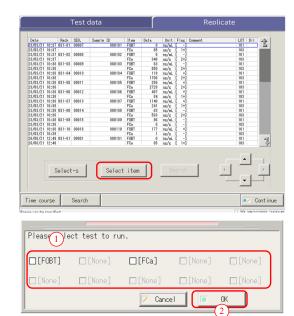

Touch {Select item}. The item selection dialog opens.

3 Applied Operations

- 2 Select a test item.
 - Select an item in the dialog.
 - ② Touch {OK}.

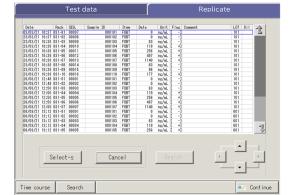

- 3 The selected item data are displayed.
 - * {Select item} changes to {Cancel}.

{Cancel} : Clears the range specification and item selection.


3.3.7 Searching Replication Data

The replication data can be searched by sample ID, sample number, rack number, and test date.

The explanation starts from the condition at which the [Replicate] screen is displayed.

- Page 114 "3.3.1 Displaying the [Sample Replication] Screen (Sample)" 1 5
- Note When a range is not specified, all error samples do not become the target of the error sample search.

 The replication data cannot be searched by rack position number.
- The replication data range can also be specified by {Select-s} on the [Replicate] screen. The procedure is the same as "3.3.5 Specifying {Select-s} for Replication Data" on page 123.

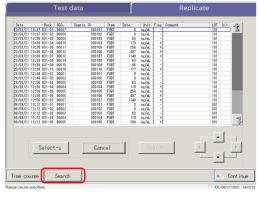

1 Touch {Search}.




- 2 Select the keyword to be searched.
 - □ Samp ID (only for the {Samp} tab)
 - □ Samp No.
 - □ Rack No.
 - □ Date




- 3 Enter a value.
 - ① Touch inside the "entry field" next to a keyword.
 - ② Enter a value using the keyboard.
 - ③ Touch {OK} or {enter}.

{A/a}: Switches between large/small letters.

{Sign}: Used to enter symbols.

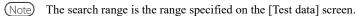

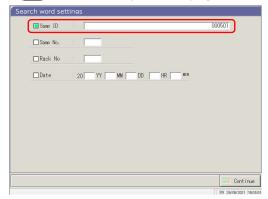

4 A search condition is entered in the entry field.



5 Touch {Continue}.

Search word settings
Samp ID 000501
☐ Same No.
□Rack No
Date 20 YY MM DD HR MM
Start?
- Cancel Discard Start
- Cancel Discard Start Continue
XX 25/03/2022 16:25:36

6 Touch {Start}.

* The search starts.

Date	Rack	SEO.	Sample ID	Iten	Data	10-14	Fire	Comment	LOT DI	
1/12/17 12:44 1			000501	FOBT	Data 86	ng/nL	Flag	Lossent	101	1
1/12/17 12:44 1			000501	FORT	86	ng/m.	-		101	
1/12/17 12:46 1			000501	FOBT	88	ng/aL	1 -1		101	-
1/12/17 12:47 1		01005	000501	FOBT	88	ng/aL	1 -		101	-
1/12/17 12:48 1	12-04	01005	000501	FOBT	88	ng/aL	1 -1		101	-
1/12/17 12:48 1		01005	000501	FOBT	88	ng/aL	1 -1		101	-
		01007	000501	FOBT	86	ng/aL	1 -1		101	-
1/12/17 12:51 1			000501	FOBT	86	ns/st	1 -1		101	
1/12/17 12:52 1		01010	000501	FOBT	86	ng/al.	1 -1		101	
1/12/17 12:53 1		01011	000501	FORT	88	ng/aL	i -i		101	-
1/12/17 12:54 1		01012	000501	FORT	88	ng/aL	i -i		101	
1/12/17 12:55 1		01013	000501	FORT	66	ng/eL	i -i		101	
1/12/17 12:56 1	13-84	81814	000501	FOBT	88	ng/eL	[-]		101	
1/12/17 12:57 1	13-04	01015	000501	FOBT	88	ng/eL	[-]		101	
1/12/17 12:58 1	13-04	01016	000501	FOBT	88	ng/nL	[-]		101	
1/12/17 12:58 1	13-04	01017	000501	FOBT	88	ng/nL	[-]		101	
1/12/17 13:00 1	14-04	01018	000501	FOBT	88	ng/nL	[-]		101	
1/12/17 13:01 1	14-04	01019	000501	FOBT	88	ng/nL	[-]		101	
1/12/17 13:02 1	14-84	01020			86		[-]		101	
1/12/17 13:03 1					86		[-]			
							[-]			211
1/12/17 13:05 1	14-84	01023	000501	FOBT	86	ng/nL	[-]		101	1
/12/17 13:02 1 /12/17 13:03 1 /12/17 13:04 1 /12/17 13:05 1	14-04 14-04 14-04	01020 01021 01022 01023	000501 000501 000501 000501	FOST FOST FOST FOST	66	ng/aL ng/aL ng/aL ng/aL	[-]			->

7 The search results are displayed.

3 Applied

Operations


3.3.8 Editing and Recalculating CC

Recalculation of CC origin value and DA value by editing and saving the recalculated CC in the hard disc are available.

(Note) While logging in with an administrator ID or when "NO" is set to Operator mode, editing and recalculation are available.


The explanation starts from the condition which the [CC list] screen is displayed.

Page 57 "■ CC list"

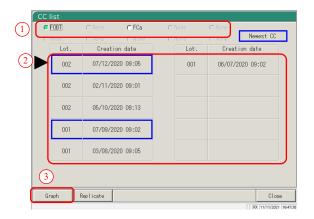


1 Select CC.

- ① Select an item.
- ② Touch the lot or creation date of the CC to be edited/recalculated.
 - * The selected CC is indicated by the 🕟 mark.
- 2 Touch {Graph}.

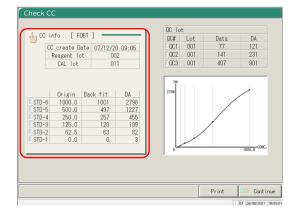

3 Touch {CC info}.

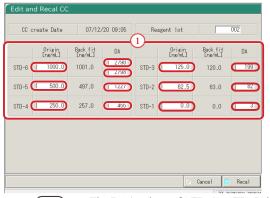


- 4 Edit CC.
 - ① Touch the entry field of the item to be edited.

The keyboard is displayed.

Note The DA1 values of STD-1 - -TD-5 should be entered in DA fields.


The DA values of STD-6 are the entered value of DA1 for the upper level and the entered value of DA2 for the lower level.

- Note DA2 for STD-6 is referred to for the PRC method check.
 - Page 238 "1.4 Prozone Check"

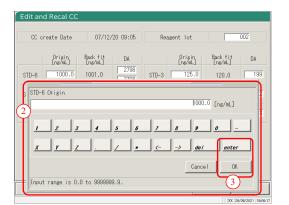


② Enter a number.

③ Touch {OK} or {enter}.

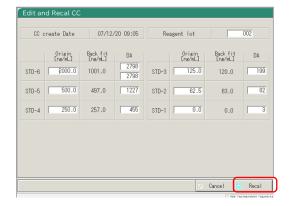


- 5 Touch {Recal}.
 - * The recalculated CC will be displayed.

3.3 Replication

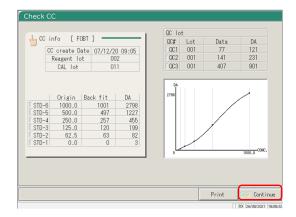

6 Touch {Continue}.



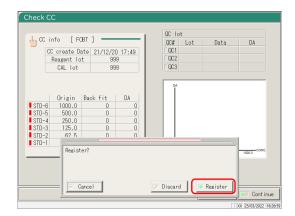

7 Touch {Register}.

* The CC is registered as the latest CC.

{Register}: Registers the CC data.

{Discard}: Aborts registration and returns to the [Check CC] screen.


{Cancel}: Closes the dialog box.

3.3.9 Displaying and Printing a Time Course (Sample)


The time course of the replication data (sample) can be displayed. On the screen, the time course (graph), the number of cycles (Cycle), and the absorbance (ABS) are displayed.

The explanation starts from the condition at which the [Replicate] screen is displayed.

Page 114 \[\int 3.3.1 \] Displaying the [Sample Replication] Screen (Sample)" \[1 \] - \[3 \]

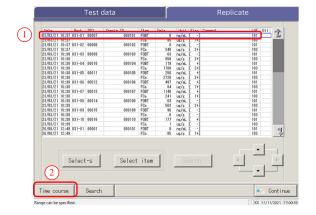

- 1 Display the time course.
 - ① Touch the date field.
 - * The specified data is encircled by a blue frame.
 - ② Touch {Time course}
 - * The time course of the specified data is displayed.

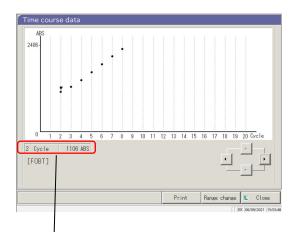

2 The time course is displayed.

When printing the time course, touch {Print}.

{Print} : Prints the time course.


{Range change} : Changes the time course range.

{Close}: Returns to the [Replicate] screen.

Note The "▼" mark on the screen can be moved by the cursor buttons.

The numeral information of the cursor is displayed in

Each item indicates the following meanings.

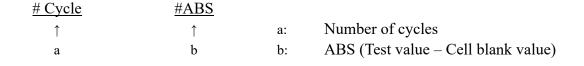


3.3.10 Displaying and Printing a Time Course (STD)

The time course of the replicate data can be displayed.


On the screen, the time course (graph), the number of cycles (Cycles), and the absorbance (ABS) are displayed.

The explanation starts from the condition at which the [Replicate (STD)] screen is displayed.

Page 118 "3.3.2 Displaying [Replicate (STD)] Screen"

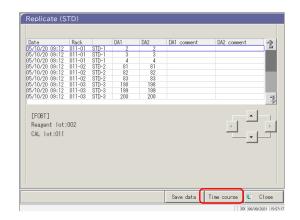

- 1 Touch {Time course}.
 - * The time course of the specified data is displayed.

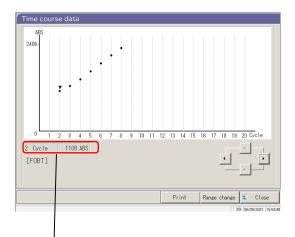

2 The time course was displayed.

When printing the time course, touch {Print}.

{Print} : Prints the time course.

 $\{Range\ change\}: Changes\ the\ time\ course\ range.$

{Close}: Returns to the [Replicate] screen.

Note

The "▼" mark on the screen can be moved by the cursor buttons.

The numeral information of the cursor is displayed in

Each item indicates the following meanings.

3 Applied Operations

3.3.11 Changing the Time Course Range (Sample)


On the [Time course data range change] screen, the range can be changed by entering "Max Value" and "Min Value."

The explanation starts from the condition at which the [Replicate] screen is displayed.

Page 114 "3.3.1 Displaying the [Sample Replication] Screen (Sample)" 1 - 3



The time course data for which the range was changed is not saved to the hard disk.

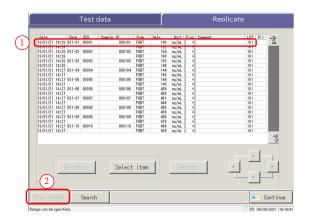


1 Display the time course.

Touch a Date field.

* The specified data is encircled by a blue frame.

② Touch {Time course}.

* The time course of the specified data is displayed.

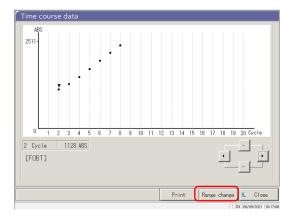

2 Touch {Range change}.

{Print} : Prints the time course.


{Range change}: Changes the time course range.


{Close}: Returns to the [Replicate] screen.


3.3 Replication

3

Applied Operations out range is 0 to 100000.

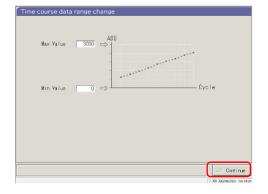

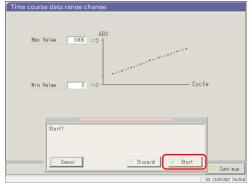

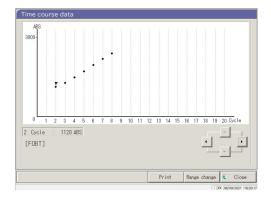

3 Set ABS.

(Available range: 0 - -0000)

- ① Touch the entry field of [Max Value] or [Min Value]. The numeric keypad opens.
- ② Enter a number.
- ③ Touch {OK} or {enter}.

4 Touch {Continue}.

5 Touch {Start}.

* The range changes.

{Cancel}: Closes the dialog box.

{Discard}: Aborts the range change and returns to the [Time course data] screen.

{Start}: Changes the range.

6 The time course after the range change has been displayed.

{Print} : Prints the time course. If {Close} is touched during printing, the printing is aborted.


{Close}: Returns to the [Replicate] screen.

3.3.12 Changing the Time Course Range (STD)


On the [Time course data range change] screen, the range is changed by entering the "Max Value" and the "Min Value."

The explanation starts from the condition at which the [Replicate (STD)] screen is displayed.

- Page 118 "3.3.2 Displaying [Replicate (STD)] Screen"
- The time course data for which the range was changed will not be saved to the hard disk.

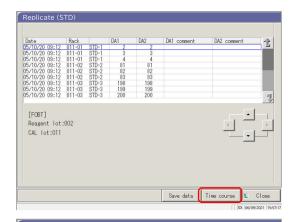

7 8 9 10 11 12 13 14 15 16 17 18 19 20 [FOBT]

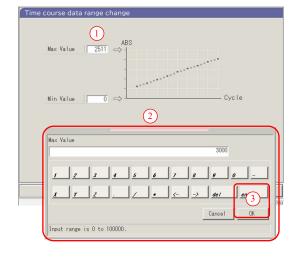

- 1 Display the time course.
 - 1 Touch a Date.
 - * The specified data is enclosed by a blue frame.

Applied Operations


- ② Touch {Time course}.
 - * The time course of the specified data is displayed.
- Touch {Range change}.

3 Set ABS.

(Available range: 0 - -0000)


- ① Touch the entry field for [Max Value] or [Min Value]. The numeric keypad opens.
- ② Enter a number.
- $\ensuremath{\mathfrak{G}}$ Touch $\ensuremath{\mathsf{OK}}$ or $\ensuremath{\mathsf{enter}}$.

3.3 Replication

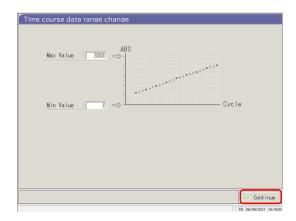

4 Touch {Continue}.

3 Applied Operations

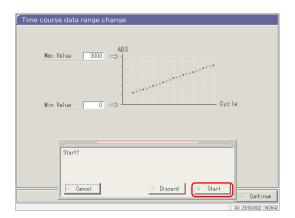


5 Touch {Start}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts the range change and returns to the [Time course data] screen.

{Start}: Changes the range.

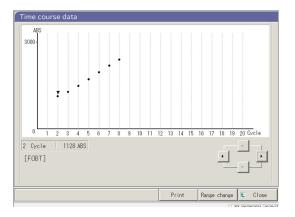

The time course after the range change has been displayed.

{Print}: Prints the time course. While printing,
 touching {Close} aborts printing.
{Close}: Returns to the [Replicate (STD)] screen.

On the accuracy control screen, QC lot select, Intra-day/Inter-day, and \bar{X} -R control are displayed. In addition, the Intra-day/Inter-day and \bar{X} -R control range can be edited.

3.4.1 Displaying the QC Lot List ([QC lot select] screen)


Touching {Process data} - -Process QC} on the [MENU] screen or touching {Process QC} tab on the [Analyzing] screen will display the QC lot select ([QC lot select] screen).

(Required to select a test item and to operate the {Start} button)


3 Applied Operations

On this screen, the {Intra-day/Inter-day} tab and the $\{\overline{X}-R \text{ control}\}\$ tab are provided in addition to the {QC lot select} tab.


Note) When there is no test data, switching from the {QC lot select} tab to another tab is not available.

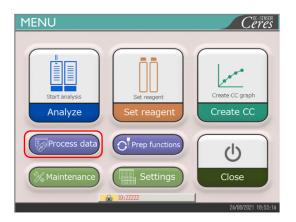


1 Touch {Process data}.

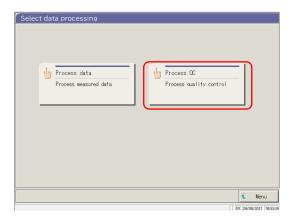

2 Touch {Process QC}.

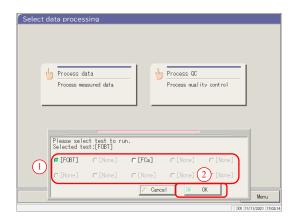

3
Applied
Operations

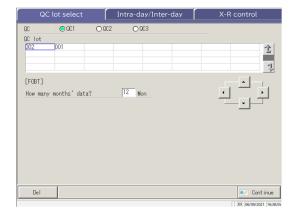


- 3 Select a test item.
 - ① Select a radio button (•).
 - 2 Touch $\{OK\}$.

{Cancel}: Closes the dialog box.

- 4 The [QC lot select] screen opens.
 - · {QC lot select} tab
 - {Intra-day/Inter-day} tab
 - $\{X-R \text{ control}\}\ tab$
 - * The currently used QC lots are displayed sequentially.

(The display order is from left to right and from top to bottom.)

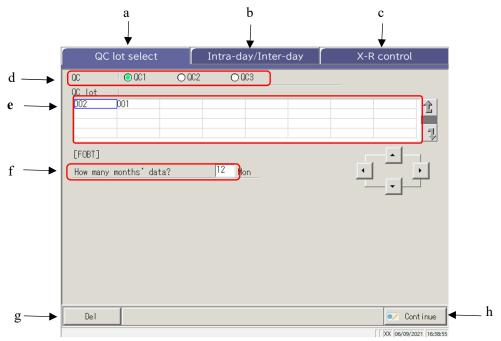

3 Applied Operations

Ξ

[Screen]: QC lot select screen

a	QC lot select	Displays the QC lots saved in QC1C3.
ь	Intra-day/Inter-day	Displays the specified QC# and the intra-day/inter-day data of the lot.
-	X-R control	Displays the \overline{X} -R control of the intra-day/inter-day data.
<u> </u>	A-K Collifor	Displays the A-R control of the intra-day/inter-day data.
d	QC	Selects a QC# for the displayed QC lot.
e	QC lot	Displays the QC lot of the selected QC#. Selects the QC lot to be used.
f	Mon	Specifies the QC lot to be used in units of months. Enter the number of months to process the data.
g	Del	Deletes the selected QC lot.
h	Continue	Registers the changed data.

3.4.2 Selecting a QC Lot

When the [QC lot select] screen is opened, a list of QC lots is displayed. The QC No. and QC lot selected in this list are the target data to be processed on the {Intra-day/Inter-day} tab and the $\{\bar{X}-R \text{ control}\}$ tab.

- Page 142 "3.4.4 Opening the [Intra-day/Inter-day] Screen"
- Page 151 "3.4.8 Displaying X-R Control"

The explanation starts from the condition at which the [QC lot select] screen is displayed.


Page 137 "3.4.1 Displaying the QC Lot List ([QC lot select] screen))"



- 1 Touch QC No. (QC1 -C3)
 - * The currently used QC lots are displayed sequentially. (The display order is from left to right and from top to bottom.)



Touch a QC lot (selecting is also available using the cursor keys).

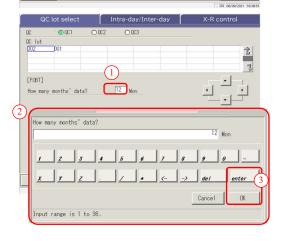

- Enter the number of months of data to process. (For an example entry, see Note) .)
 - ① Touch the entry field.
 - ② Enter a number using the numeric keypad.
 - ③ Touch {OK} or {enter}.
- (Note) In the QC lot list, the data of the latest month to the specified number of months are displayed (1 to 36 months in units of months).
 - (Example 1) When "one month" is entered on the current date of June 15, the data from June 1 to June 15 are displayed. (Example 2) When "three months" is entered on the current date of June 15, the data from April 1 to June 15 are displayed.

3.4.3 Deleting a QC Lot

The QC No. - - C lot selected in "3.4.2 Selecting a QC Lot" can be deleted.


After the QC lot to be deleted is selected, touch {Continue} and touch {Start} in the dialog box.

When the data for the selected QC No. - - C lot is deleted, the hard disk is updated.


* If {Start} is not touched, the hard disk is not updated.

The explanation starts from the condition at which operation 2 in "3.4.2 Selecting the QC Lot" is completed.

Page 140 "3.4.2 Selecting a QC Lot"

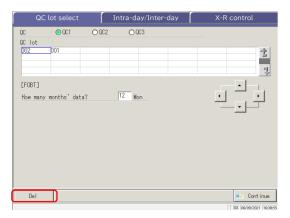


1 Touch {Del}.

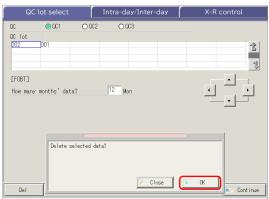

2 Touch {OK}.

* The data in memory is updated.

[Close]: Aborts deletion of the QC lot.

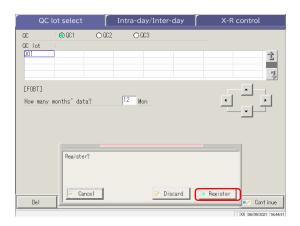

- 3 Save the updated data.
 - ① Touch {Continue}.
 - ② Touch {Register}.
 - * The updated data is saved on the hard disk.

 $\{Cancel\}$: Closes the dialog box.


{Discard}: Returns to the [Select data processing] screen.

3.4.4 Opening the [Intra-day/Inter-day] Screen


When the {Intra-day/Inter-day} tab is touched on the [QC lot select] screen, the [Intra-day/Inter-day] screen opens. On this screen, the intra-day/inter-day data of the QC No. - -C lot specified on the [QC lot select] screen is displayed.

Note If there is no test data, changing from the [QC lot select] tab to another tab is not available.




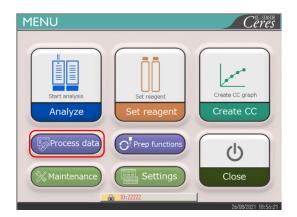

1 Touch {Process data}.

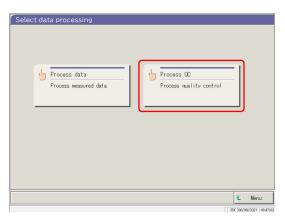


2 Touch {Process QC}.

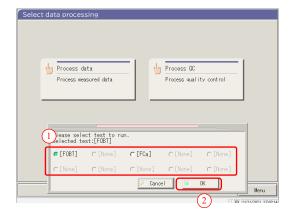



- 3 Select the test item.
 - ① Select the radio button for the corresponding item (•).
 - ② Touch {OK}.


{Close}: Returns to the previous screen.




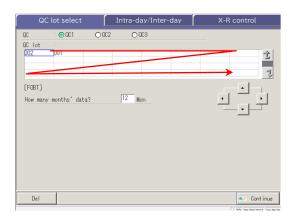


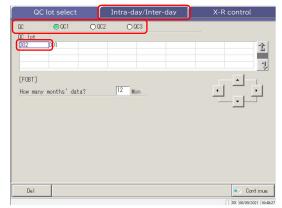

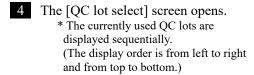


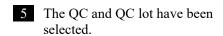

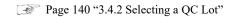

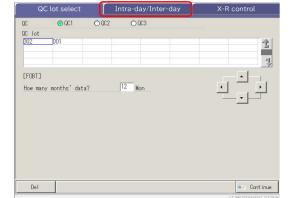

- 6 Touch the {Intra-day/Inter-day} tab.
 - * The [Intra-day/Inter-day] screen opens. (See the next page.)

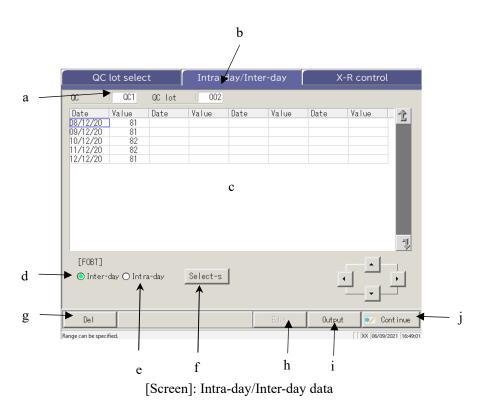

a	QC	Displays the QC (QC1C3) specified on the [QC lot select] screen.
b	QC lot	Displays the QC lot specified on the [QC lot select] screen.
С	QC data	When the intra-day data is selected
		Displays the test date and average value of the specified QC and QC lot.
	(Date, Value)	When the inter-day data is selected
		Displays the test time and average value of the specified QC and QC lot.
d	Inter-day	Displays the inter-day data (Test date, Value).
e	Intra-day	Displays the intra-day data (Test date, Value).
f	{Select-s}	Specifies the range of intra-day data or inter-day data.
		The data range can also be specified using the cursor buttons.
g	{Del}	Deletes the specified intra-day or inter-day data.
h	{Edit}	Edits the intra-day or inter-day data.
i	{Output}	Outputs the intra-day or inter-day data (printer, external media)
j	{Continue}	Registers the edited intra-day or inter-day data on the hard disk.
		Displays the data [registration check] screen. Touch {Register}.

3.4.5 Editing (Recalculating) Intra-Day and Inter-Day Data

When the {Intra-day/Inter-day} tab is touched on the [QC lot select] screen, the [Intra-day/Inter-day] screen opens. On this screen, the intra-day or inter-day data of the specified QC No.-QC lot is displayed. The radio buttons are used to switch between the intra-day and inter-day data.

Intra-day data: Edit the intra-day data. Inter-day data: Edit the replication data.

The explanation starts from the condition at which the [QC lot select] screen is switched to the [Intra-day/Inter-day] screen.

* The QC lot has already been selected on the [QC lot select] screen.


Page 142 "3.4.4 Opening the [Intra-day/Inter-day] Screen"

Note When there is no test data, switching from the {QC lot select] }ab to another tab is not available.

Note The maximum number of tests available for a day with one QC# and QC lot is ten.

Note The results of the 11th and later tests are not saved even if they are conducted.

When the user is logged with an administrator ID, or when "No" is selected for "Operator mode," editing is possible.

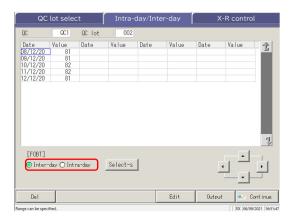



Touch {Edit}.


0


Select the data to be edited. Intra-day data


Inter-day data

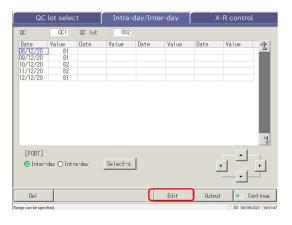

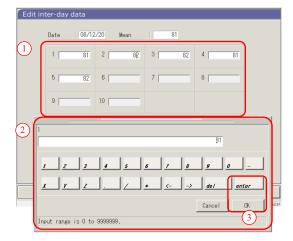

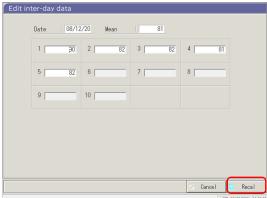

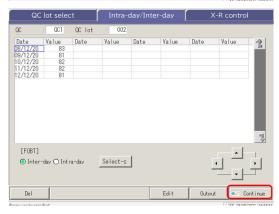

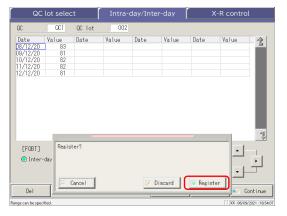

- 3 Edit the intra-day data (or inter-day data).
 - ① Touch an entry field.
 - ② Enter a number using the numeric keypad.
 - 3 Touch the [OK] or [enter] button.

4 Touch {Recal}.

{Cancel}: Aborts the editing of the intra-day or inter-day data. Returns to the [Intra-day/Inter-day] screen.

- 5 Touch {Continue}.
 - * The new average value is displayed.

- 6 Touch {Register}.
 - * The recalculated result is registered.

{Cancel}: Closes the dialog box.

{Discard}: Returns to the [Select data processing] screen.

3 Applied Operations

3.4.6 Deleting Intra-Day and Inter-Day Data

The intra-day or inter-day data can be deleted by specifying a data range on the [Intra-day/Inter-day] screen. After deletion, the specified data will no longer be displayed on the screen. However, the hard disk data is not updated until {Register} is touched.


The explanation starts from the condition at which the [QC lot select] screen is switched to the [Intra-day/Inter-day] screen.

* The QC lot has been selected as the [QC lot select] screen.


(All

Page 142 "3.4.4 Opening the [Intra-day/Inter-day] Screen"


Note When there is no test data, switching from the [QC lot select] tab to another tab is not available.

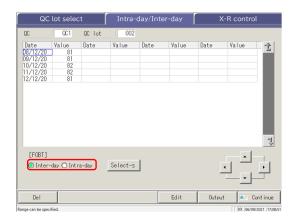

- 1 Select the data to be edited.
 - o Intra-day data
 - Inter-day data

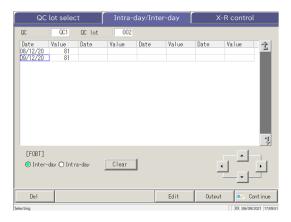


- 2 The range of data to be deleted is specified.
 - * Only the selected data is displayed.
 - Page 149 "3.4.7 Selecting Intra-Day and Inter-Day Data with {Select-s}"

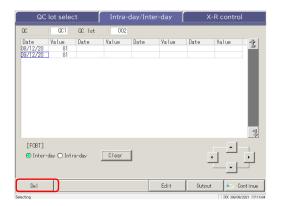


3 Touch {Del}.

3 Applied Operations When the {Del} button is touched, the data in the memory is deleted. However, the data on the hard disk is not deleted until {Register} is touched.

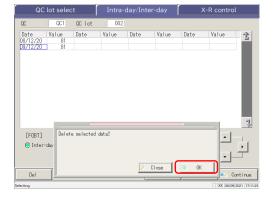


- 4 Touch {OK}.
 - * The data in memory is updated.
 - * The data specified on the screen is deleted.

{Close}: Aborts deleting the intra-day and inter-day

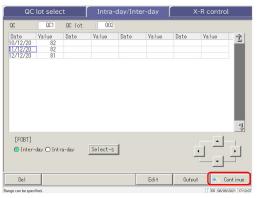

5 Touch {Continue}.

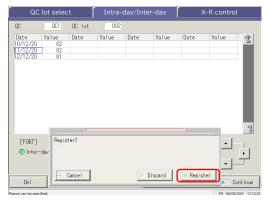

6 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Returns to the [Select data processing] screen.

3.4.7 Selecting Intra-Day and Inter-Day Data with {Select-s}


On the [Intra-day/Inter-day] screen, the data can be specified within a test date range (from starting point to ending point).

It is used when deleting a batch of intra-day or inter-day data or when displaying the X-R control.


The explanation starts from the condition at which the [QC lot select] screen is switched to the [Intra-day/Inter-day] screen.

* The QC lot has been selected already on the [QC lot select] screen.


Page 142 "3.4.4 Opening the [Intra-day/Inter-day] Screen"

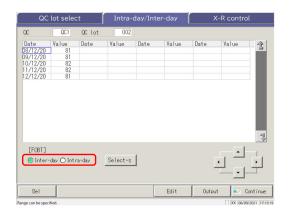

- 1 Select the data to be specified.
 - o Inter-day data
 - Intra-day data

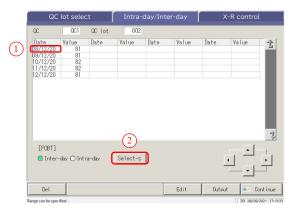

- 2 Specify the test date of the starting point.
 - ① Touch the test date to be the starting point.
 - ② Touch {Select-s}.
 - * The starting point is set.
 - * When the starting point is set, {Select-s} changes to {Select}.

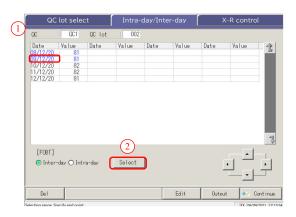

- 3 Specify the test date of the ending point.
 - ① Touch the test date to be the ending point.
 - ② Touch {Select}.
 - * The ending point is set.
 - * When the ending point is set, {Select} changes to {Clear}.

{Select-s}: Sets the starting point of the range.

{Select}: Sets the ending point of the range.

{Clear}: Aborts the range specification.

Note When the dates specified as the starting point and ending point are the same, only one test data set is specified.

To redo the range specification, touch {Clear}.

Note All test data are displayed, and no range is specified.


Note If the [X-R control] screen is opened instead of the [Intra-day/Inter-day] screen from the [QC lot select] screen, all the data are specified.

3
Applied
Operations

The X-R control of the intra-day and inter-day data can be displayed.


On the [QC lot select] screen, when the intra-day data is selected, \underline{X} -R control of the intra-day data opens. When the inter-day data is selected, $\underline{\overline{X}}$ -R control of the intra-day data is displayed.

(Note) When there is no test data, switching from the [QC lot select] tab to another tab is not available.

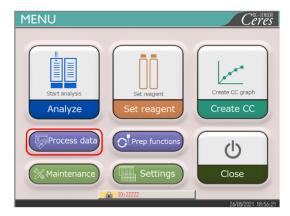


1 Touch {Process data}.



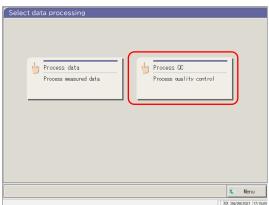


2 Touch {Process QC}.

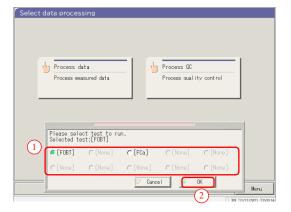


- 3 Select the test item.
 - ① Select an item with a radio button (•).
 - ② Touch {OK}.

{Close}: Returns to the previous screen.

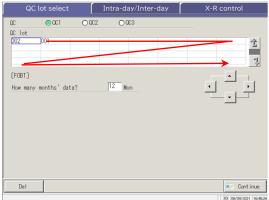



4 The [QC lot select] screen opens. * The QC lots used in recent days are displayed sequentially. (The display order is from left to right and from top to bottom.)

3 Applied Operations

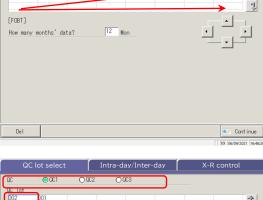


5 Select a QC and QC lot.

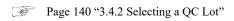

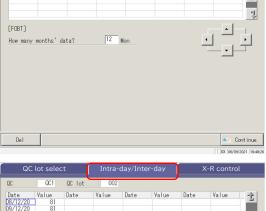


6 Touch the {Intra-day/Inter-day} tab.

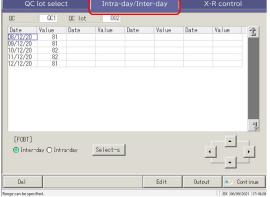


7 Select the \bar{X} -R control data.

```
0C1 0C lot
Date Value
08/12/20 81
09/12/20 81
                          Value Date
                 Date
                                             Value
 [FOBT]
 ● Inter-day ○ Intra-day
                           Clear
                                             Edit
```

Page 149 "3.4.7 Selecting Intra-Day and Inter-Day Data with {Select-s}"

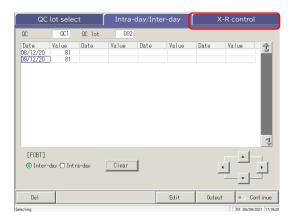


8 Touch the $\{\overline{X}$ -R control $\}$ tab.

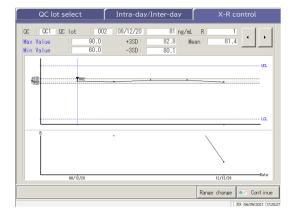

 $\frac{1}{2}$ $\frac{1}$

3 Applied Operations

Note The max value is determined by replicate number and measured value.

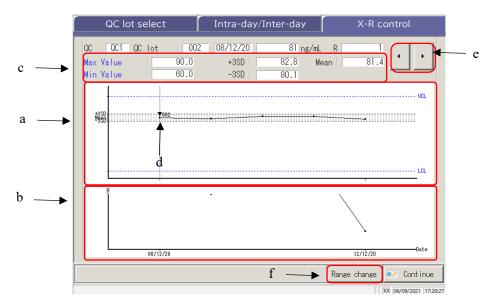

3
Applied
Operations

[Screen] : \bar{X} -R control


$ar{ ext{X}}^-$ graph	•	Displays the data for the test date or test date and time.				
(upper graph)	•	Displays the mean of all values and \pm #SD.				
	•	The max. value and the min. value set on the [Assay]QC process] -				
		-Set} tab are displayed in blue.				
	•	Data <u>less than min. value $+1$</u> and <u>more than max. value $+1$</u> are				
		displayed in red.				
R graph	•	Displays the data range for each test date or the test date and time.				
(lower graph)	•	Displays the R upper control limit				
Max Value	•	Displays the Max. Value set on the [Assay]QC process]Set} ta				
Min Value	٠	Displays the min. value set on the [Assay]QC process]Set} tab				
+ #SD	•	Displays the calculated $+$ #SD value and the SD coefficient (# = 1 – 9)				
		set on the [Assay]QC process]Set} tab.				
-#SD	•	Displays the calculated - #SD value and the SD coefficient (# = $1 - 9$)				
		set on [Assay]QC process]Set} tab.				
Mean	•	The calculated mean value				
▼ (In the graph)	•	Indicates that the CC has changed.				
	•	Displays the R1/R2 reagent lot next to the ▼ mark.				
◆ ▶	•	Moves the cursor (vertical line) displayed on the graph.				
{Range change}	•	Changes the range in \bar{X} -R control (max. and min. concentrations and R max. value)				
	(upper graph) R graph (lower graph) Max Value Min Value + #SD -#SD Mean ▼ (In the graph)	(upper graph) R graph (lower graph) Max Value Min Value + #SD -#SD Mean ▼ (In the graph) .				

3.4.9 Changing the Range of \bar{X} -R Control

The X-R control (graph) range can be changed.

The explanation starts from the [X-R control] screen.


Page 151 "3.4.8 Displaying X-R Control"

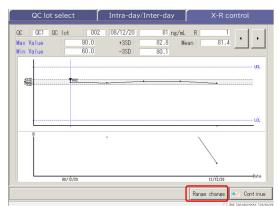


1 Touch {Range change}.




- 2 Change the range.
 - ① Touch the entry field of a setting.
 - ② Enter a number using the numeric keypad.
 - · Max. value of concentration
 - · Min. value of concentration
 - · R max limit.
 - ③ Touch {OK} or {enter}.

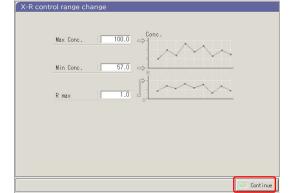

3 Touch {Continue}.

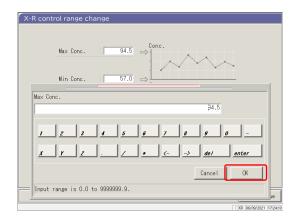

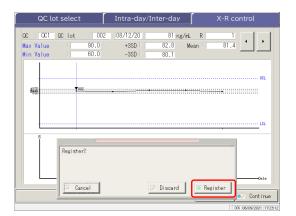

- 4 Touch {OK}.
 - * Displays the X-R control after the range change.

{Cancel}: Closes the dialog box.

{Close}: Aborts the range changing process and returns to the [X-R control] screen.

5 Touch {Continue}.

- 6 Touch {Register}.
 - * X-R control is registered.
 - * Returns to the [Select data processing] screen.

{Cancel}: Closes the dialog box.

{Discard}: Aborts the registration of X-R control and returns to the [Select data processing] screen.

3

Applied Operations

To use all the functions on the [MENU] screen, logging in is necessary. Enter the operator ID on the [LOGIN] screen and log in. The operator ID is then displayed.



3 Applied Operations

3.5.1 LOGIN

When the system starts up, the [LOGIN] screen opens.

For the login procedure, see the page 28 "2.3 Logging In."

3.5.2 LOGGING OUT



Touch the {Lock} mark at the lower section of the [MENU] screen (displaying the currently logged in operator ID).




2 Touch {LOGIN / LOGOUT}.



3 Touch {LOGOUT}.

 $\{Cancel\}$: Closes the dialog box.

{Close}: Returns to the [MENU] screen while keeping the current LOGIN/LOGOUT condition.

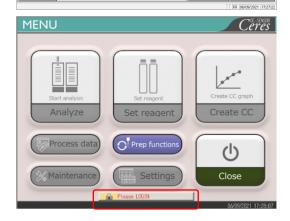


4 Returns to the [MENU] screen. The ID display changes to the "Login" message (indicating the logged-out condition).

3.5.3 Changing a Password

The password of the currently logged in operator can be changed.

The explanation starts from the logged in condition.

For the logging in procedure, see page 28 "2.3 Logging In."

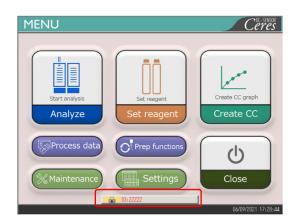


1 Touch the {Lock} mark at the lower section of the [MENU] screen (the operator ID of the currently logged in operator is displayed).



LOGIN			
	ID Enter Operator ID	PASSWORD	
	Enter operator ID		
User list			LOGIN / LOGOUT


2 Touch {Change password}.

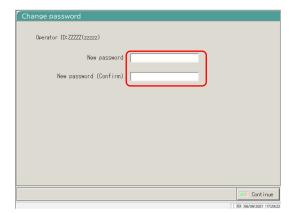

- 3 Set a new password.
 - ① Enter the "new password".
 - 2 Enter the same password again in the "New password (Confirm)" field.


zzzzz: Operator ID (zzzzz): Operator name

3.5 LOG IN/LOG OUT

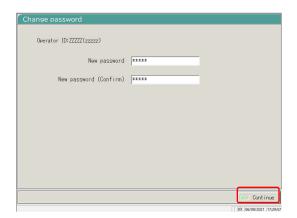

4 Touch {Continue}.



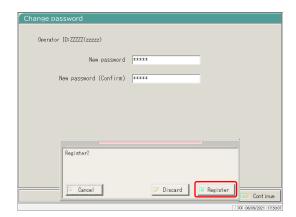

Touch {Register}.
The password is changed.

* The display returns to the [LOGIN] screen.

{Cancel}: Closes the dialog.

{Discard}: Returns to the [LOGIN] screen. The password is not changed.

Chapter 4 Prep Functions

- 4.1 Initialization
- 4.2 Priming
- 4.3 Washing




Chapter 4 Prep Functions

4.1 Initialization

This function initializes (moves to the standby position) each mechanism in the system. The following mechanisms will be initialized.

· Sample nozzle	· Reagent nozzle	• Mixer
Reaction table	· Puncturing system	· Squeezing system
· Rack setting position	· Reagent refrigerator	Rack transfer system

Touching {Prep functions} on the [MENU] screen opens the [Initialize] screen.

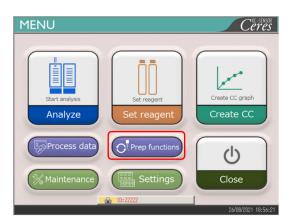

1 Touch {Prep functions}.



2 Touch {Start}.

{Menu}: Returns to the [MENU] screen.



3 Initialization is executed.

{Pause}: Pauses initialization. {Start}: Starts initialization.

When {Pause} is touched, the button changes to

{Start}.

{Abort}: Aborts initialization.




Start 🏬

4 Initialization has ended.

4.2 Priming

This function introduces washing solution and purified water into the pipes.

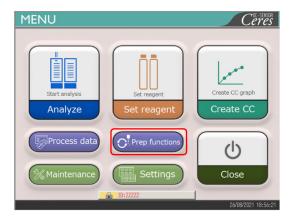


1 Touch {Prep functions}.



2 Touch the {Prime} tab.



- 3 Select the items to prime.
 - o YES: Executes priming
 - o NO: Does not execute priming
- 4 Touch {Start}.
 - * Priming starts.

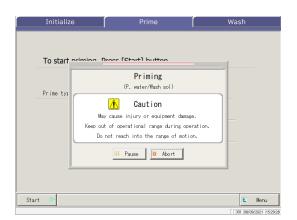

5 Priming is executing.

* When priming completes, the dialog box closes.


{Pause}: Pauses the process.

 $\begin{aligned} & \{Start\} \colon The \ process \ restarts. \ Touching \ \{Pause\} \\ & changes \ the \ button \ to \ \{Start\}. \end{aligned}$

{Abort}: Aborts the process in mid-course.

4.3 Washing

When the {Wash} tab is touched on the [MENU] – [Prep functions] screen, the [Wash] screen opens. Select the parts to be washed and the storage process (soak wash), and touch {Start}. Washing of each part starts.

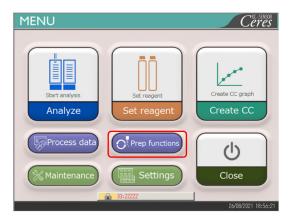

1 Touch {Prep functions}.



2 Touch the {Wash} tab.




3 Select "YES" or "NO" for each part to be washed. (See the table on the next page.)

Part to be washed	Range/selection	Content
Washing		
Sample dispensing nozzle	Yes:	Washes the sample dispensing nozzle.
	No:	Does not wash the sample dispensing nozzle.
Reagent dispensing nozzle	Yes:	Washes the reagent dispensing nozzle.
	No:	Does not wash the reagent dispensing nozzle.
Mixer	Yes:	Washes the mixer.
	No:	Does not wash the mixer.
Storage		
Nozzle and mixer	Yes:	Soak the nozzle and mixer for storage.
	No:	Does not soak the nozzle and mixer for storage.



4 Touch {Start}.

{Menu}: Returns to the [MENU] screen.

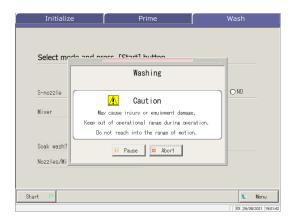

5 Washing is executed.


 $\{Pause\}\colon Pauses\ washing.$

{Start}: Starts washing.

Touching {Pause} changes the button to {Start}.

{Abort}: Aborts washing.

4.3 Washing

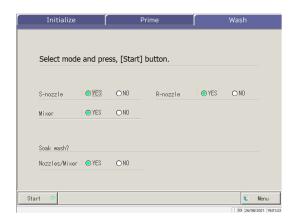

6 Washing has ended.

Chapter 5 Maintenance

- 5.1 Inspection and Maintenance
- 5.2 List of Parts to Check and Replace



Chapter 5 Maintenance


5.1 Inspection and Maintenance

On the [Maintenance] screen, inspection and maintenance items to be conducted periodically (daily, weekly, and monthly) are displayed.


In addition, touching {Procedure} enables inspections and maintenance orders.

5.1.1 Opening the [Maintenance] Screen

On the [MENU] screen, touching {Maintenance} opens the [Maintenance] screen.

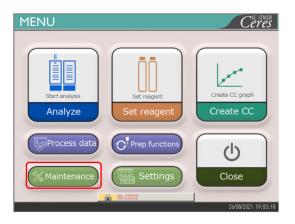


1 Touch {Maintenance}.

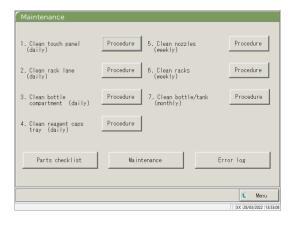

The [Maintenance] screen opens.

5.1.2 Cleaning Control Panel (Daily)

On the [Maintenance] screen, touching {Procedure} on the right side of [1. Clean touch panel] displays the cleaning procedure.

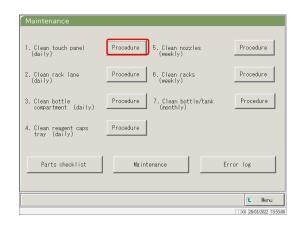



■ Procedure

Wipe the touch panel with cotton or a soft towel.

5 Maintenance





{Close}: Returns to the [Maintenance] screen.


5 Maintenance

5.1.3 Cleaning the Rack Lane (Daily)

On the [Maintenance] screen, touching {Procedure} on the right side of [2. Clean rack lane] displays the cleaning procedure.

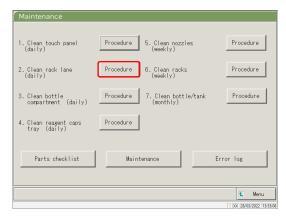

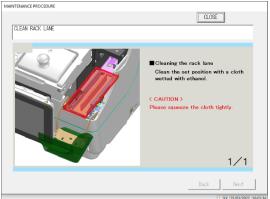

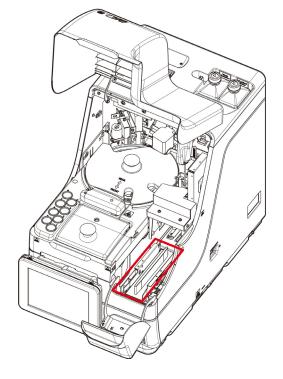


■ Procedure


- ① Wet a soft cloth with ethanol.
- ^ Caution: Wring the cloth tightly.
- ② Wipe the rack lane with the cloth described in ①.

{Close}: Return to the [Maintenance] screen.

5.1.4 Cleaning Washing Solution and Purified Water Compartments (Daily)

On the [Maintenance] screen, touching {Procedure} on the right side of [3. Cleaning Washing Solution and Purified Water Compartments] displays the cleaning procedure.

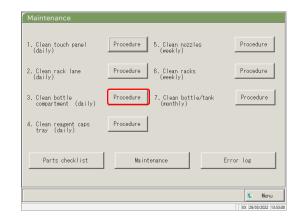

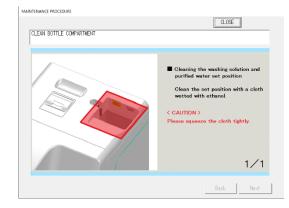


- Procedure
- ① Wet a soft cloth with ethanol.
- ⚠ Caution: Squeeze the cloth tightly.
- ② Wipe the compartment with the cloth described in ①.

5 Maintenance

{Close}: Returns to the [Maintenance] screen.

5.1.5 Cleaning the Reagent Cap Tray (Daily)

On the [Maintenance] screen, touching {Procedure} on the right side of [4.Clean reagent caps tray] displays the cleaning procedure.

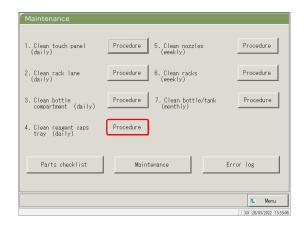



■ Procedure


- ① Clean the reagent cap tray using a cloth wetted with purified water.
- ② Clean the reagent cap tray using a cloth wetted with ethanol.
- ⚠ Caution: Please squeeze the cloth tightly.

{Close}: Returns to the [Maintenance] screen.


Maintenance


5.1.6 Cleaning the Nozzles (Weekly)

On the [Maintenance] screen, touching {Procedure} on the right side of [5. Cleaning nozzles] displays the cleaning procedure.

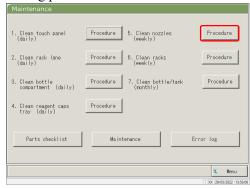



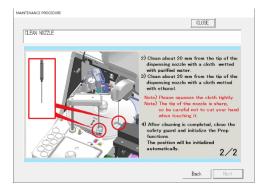

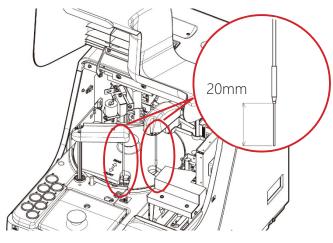

Procedure

① Hold the red circles on the covers of the sample nozzle and reagent nozzle, slowly lift them up, and rotate to a position where it is easy to clean.



Caution: Do not hold the nozzle itself to lift it. The nozzle may bend.



Caution: The tip of the nozzle may break so be careful not to touch other mechanical parts when moving it.

- 2 Clean about 20 mm from the tip of the dispensing nozzle with a cloth wetted with purified water.
- 3 Clean about 20 mm from the tip of the dispensing nozzle with a cloth wetted with ethanol.

Caution: Please squeeze the cloth tightly.



Caution: The tip of the nozzle is sharp, so be careful not to pierce your hand when cleaning it.

4 After cleaning is completed, close the safety guard and initialize the Prep functions. The position will be initialized automatically.

Page 162 "4.1 Initialization"


{Close}: Returns to the [Maintenance] screen.


: Changes the screen to the next page. {Next}

{Back}: Changes the screen to the previous page.

5.1.7 Cleaning the Racks (Weekly)

On the [Maintenance] screen, touching {Procedure} on the right side of [6. Clean racks] displays the cleaning procedure.

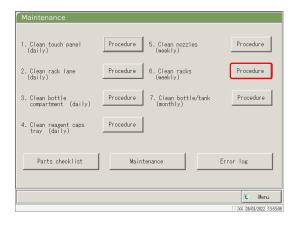

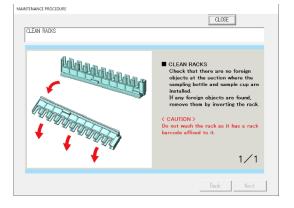

Procedure

Check that there are no foreign objects at the section where the sampling bottle and sample cup are installed. If any foreign objects are found, remove them by inverting and shaking the rack.




Caution: Do not wash the rack as it has a rack barcode affixed to it.

{Close}: Returns to the [Maintenance] screen.

5.1.8 Cleaning Bottles/Tanks (Monthly)

On the [Maintenance] screen, touching {Procedure} on the right side of [7. Clean bottle/tank] displays the cleaning procedure. Clean the tank once a month.

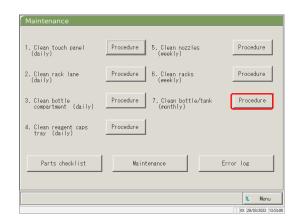



Procedure

Purified water bottle/washing solution bottle Rinse with purified water.

Drain tank

<When normal>

Wash thoroughly with tap water and rinse with purified water.

<In case of severe contamination>

- ① Wash thoroughly with tap water.
- ② Pour 2L of tap water and 20 mL of washing solution into the tank.

Tap water: 2L, Washing solution: 20 mL

- 3 Close the tank cap securely and shake the tank. (In case of severe contamination, leave for about one hour.)
- 4 Wash inside the tank thoroughly with tap water and rinse with purified water.

{Close}: Returns the screen to the [Maintenance] screen.

Caution: Insufficient rinsing with purified water may affect the test data.

5.2 List of Parts to Check and Replace


On the [Parts checklist] screen, the part names, last replacement date, number of months used, and number of use counts are displayed.

5.2.1 Opening the [Parts checklist] Screen


On the [Maintenance] screen, touching [Parts checklist] opens the [Parts checklist] screen. The parts that have expired (the number of months or use count has been exceeded) will be indicated in yellow.



1 Touch {Maintenance}.

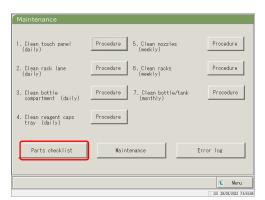

2 Touch {Parts checklist}.

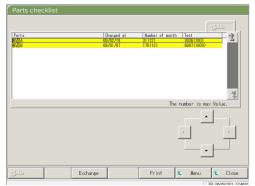


The [Parts checklist] screen opens.


Note The lines of the expired parts are yellow.

The expired parts can be replaced.

This explanation starts from the condition at which the [Parts checklist] screen is displayed.

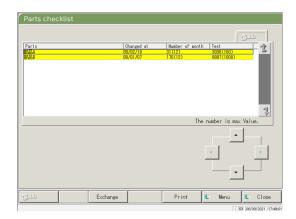


Replace the parts marked in yellow.

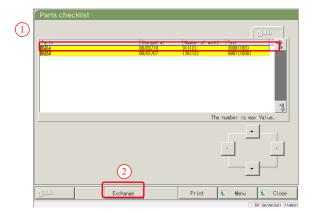

- 2 Reset the number of month and use count for the parts replacement.
 - ① Touch the line of the replaced part.
 - ② Touch {Exchange}.

5 Maintenance

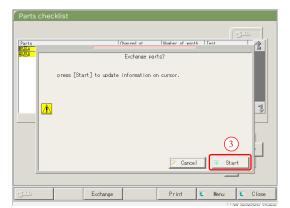

- ③ Touch {Start}.
 - * The number of months and use count become "0."
 - * The dialog box closes.

{Close}: Aborts the resetting of the months and use count.

Chapter 6 Settings

- 6.1 System Settings
- 6.2 Protocol Settings



Chapter 6 Settings


6.1 System Settings

When using the system for the first time, the system settings must be set. After the system settings are completed, no change is necessary for daily testing, unless other settings are needed. For an outline of the setting items in "System Settings," see page 31 "2.4.1 System Settings."

6.1.1 Sample Barcode Settings (Common)

This section describes the common settings which are used to read the sample barcodes.

When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," changing the settings becomes possible.

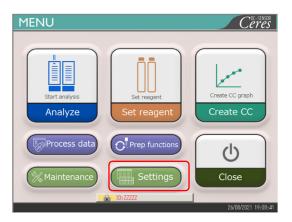

1 Touch {Settings}.




2 Touch {Sample barcode}.

6 Settings



3 Set the items on the {Common} tab.

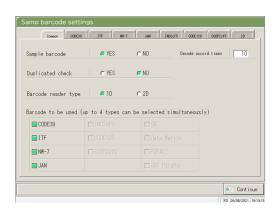

Setting item	Selection/setting	Content
Sample barcode	YES:	Barcodes are affixed to the samples.
	NO:	Barcodes are not affixed to the samples.
Duplicated check		From system startup to power off:
	YES:	Checks for duplicate sample barcodes.
	NO:	Does not check for duplicate sample barcodes
		However, even if this is set to "YES," if the test mode is retest or dilution and retest, the duplicate check is not conducted.
Barcode reader type	1D:	Uses only 1D codes.
	2D:	Uses both 1D and 2D codes.
Decode accord times	Number input	Sets 10 times as a rough target.
Barcode to be used		Select the barcodes to be used (■).
		Up to four types of barcodes can be selected.


Note Decode accord times

The barcode reader scans a maximum of 500 times when reading a sample barcode. In this process, the number which the barcode has been properly read is called the <u>decode times</u>.

"Decode accord times" is the number of reading which is sufficient for proper reading.

When the larger number is set to [Decode accord times], erroneous reading of the barcode can be prevented.

However, at the same time, occurrence rate of "barcode reading error" increases.

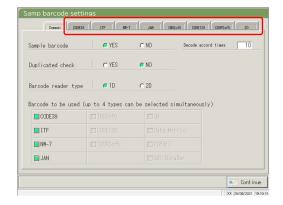

4 Detailed settings for each barcode type are set next.

Page 184 "6.1.2 Sample Barcode Settings (Detailed Settings for Each Barcode Type)"


6.1.2 Sample Barcode Settings (Detailed Settings for Each Barcode Type)

In [Samp barcode settings], there are items to set according to barcode types in addition to the common settings. Available barcodes are CODE39, ITF, JAN, NW-7, IND2of5, CODE128, and COOP2of5. The barcode settings can be seen by selecting the corresponding barcode name tabs.

Note When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," the settings can be changed.


The explanation starts from the condition at which the [Samp barcode settings] screen is displayed.

Page 182 "6.1.1 Sample Barcode Settings (Common)"

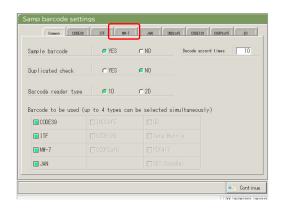


1 Touch the barcode tab to be set.

(Example)
Touch the {NW-7} tab.

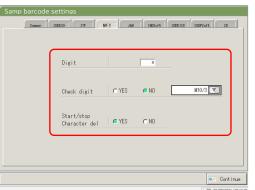

2 Set each item. (See Tables 6.1.1 - -.1.4.)


(Note) For the CODE39, JAN, and ITF barcodes, the check digit calculation method cannot be changed.

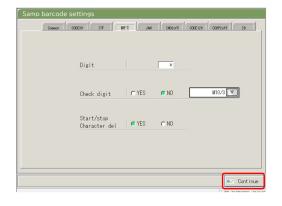

3 Touch {Continue}.

6 Settings



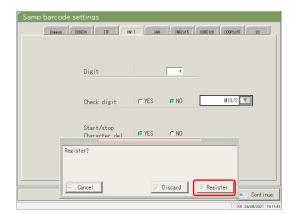

4 Touch {Register}.

 $\{Cancel\} \colon Closes \ the \ dialog \ box.$

{Discard}: Aborts registration of the Samp barcode settings and returns to the [System Settings]

screen.

{Register}: Registers the set contents.

When "YES" is selected for the check digit test, the last character of the barcode (one character before the start/stop character) is tested as a check digit.

Table 6.1.1 Samp barcode settings

Set item	Selection/setting	Content
Barcode digits		Sets the number of digits of sample barcode.
Digit		When "*" is input, the number of barcode digits is not
		checked.
		(Used when several number of digits exist for the sample
		barcode.)
		The range of input changes according to the "YES" or "NO"
		setting for "Start/stop character del."
		See Table 6.1.2 for details.
Start/stop		Sets whether the start/stop character is deleted.
character del		(Valid when NW-7 is selected.)
	YES :	Deletes the start/stop character.
	NO:	Does not delete the start/stop character.
Check digit test		Sets whether the check digit test is conducted or not.
	YES :	Conducts the check digit test.
	NO:	Does not conduct the check digit test.
Check digit calculation	See Table 6.1.3	Sets the check digit calculation method.
method		(Valid when NW-7 is selected.)

Note For the input range and the calculation method of the check digit, see Table 6.1.3.

6 Settings

Table 6.1.2 Range of input for barcode digits

Barcode type	Start/stop Input range		nge
	character deletion	Max.	Min.
	Yes	17	5
NW-7	No	15	5
ITF			
IND2of5	Invalid	15	6
COOP2of5			
CODE39			
JAN	Invalid	15	5
CODE128			
QR			
Data MatriX			
PDF417	Invalid	50	6
GS1 DataBar			
Omni-directional			

6 Settings

Table 6.1.3 Input No. and Calculation Method

Input No.	Calculation method	
1	Modulus 10/3 weight	
2	Modulus 16	
3	Modulus 11	
4	Modulus 10/2 weight	
5	Modulus 10/3 weight	
6	7 check DR	
7	Weighting modulus	
8	Loons	

Table 6.1.4 Barcode Type and Calculation Method

Barcode type	Calculation method
CODE39	Modulus 43 (fixed)
JAN	Modulus 10/3 weight (fixed)
ITF	Modulus 10/3 weight (fixed)
NW-7	Modulus 10/3 weight
	Modulus 16
	Modulus 11
	Modulus 10/2 weight
	7 check DR
	Weighting modulus
	Loons
IND2of5	No check digit
CODE128	No check digit
COOP2of5	No check digit
QR	No check digit
Data Matrix	No check digit
PDF417	No check digit
GS1 DataBar Omni-directional	No check digit


6.1.3 Rack Setting

The range of sample rack numbers.used for S sample bottle are set here.

During testing, racks with numbers in this range are processed as "sample racks."

Note When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," settings can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.

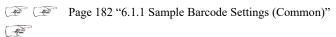




1 Touch {Rack No. settings}.




2 Set the range of sample rack numbers. (See page 189)

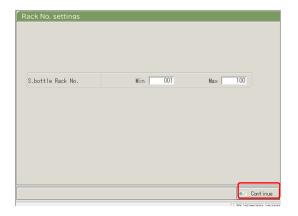

- 3 Touch {Continue}.
 - * The dialog for the change/registration check appears.

6 Settings

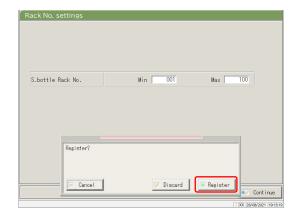

4 Touch {Register}.

{Cancel}: Closes the dialog box.

 $\{Discard\} \colon Aborts \ the \ rack \ range \ setting \ and \ returns \ to$

the [System Settings] screen.

{Register}: Registers the set contents.

Table 6.1.5 Rack setting

Setting	Value	Meaning
S. bottle Rack No.	1 - 99–	Sets the Rack No. range for S sample bottle.
Min.:		When "*" is entered, the range specification is
Max.:		disabled.

Note

To disable the rack No. range setting, enter "*."

After "*" is entered for the Min. or Max. value, touch the other value (Max. or Min. value). "*" is automatically set, and Rack No. range setting is disabled.

(If only one of Min. and Max. is "*," registration is not possible.)

6 Settings

6.1.4 Config.

In Config., the following items are set.

- Date and time
- Password


Note When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," settings can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.

Page 182 "6.1.1 Sample Barcode Settings (Common)"




1 Touch {Config.}

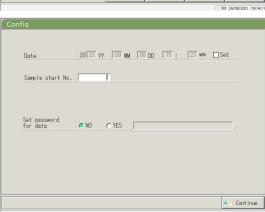

2 Set the Config. (See the next page.)


Note The screen differs depending on whether or not a password is registered.

When a password is not registered

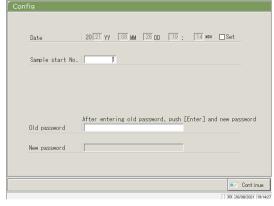


When a password is registered



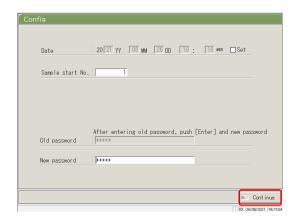

3 Touch {Continue}.

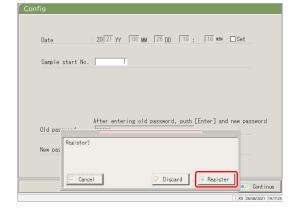

4 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts Config. and Returns to the [System Settings] screen.

 $\{Register\}$: Registers the set contents.

Setting	Value Meaning	
Date	□ Setting	To set the date and time, select the checkbox (1).
20(XX)YY - (XX)MM -	Year:0- 99	
(XX)DD ():()MIN		
	Month:1- 12	
	Day:1-31	
	Hour:0- 23	
	Minute:0- 59	
Sample start No.	1 - 99–99	Sets an initial value for the sample sequence No.
Password registration		Selects whether or not a password is registered.
	Yes:	Registers a password.
	No:	Does not register a password.
Password:	5 - 30 ch.	Enter the password here.
	(0 - 9,, X,	
	Y, Z, . ,/)	
Old password	5- 30 ch.	(When a password is set)
	(0 - 9,, X,	Changes the password.
New password	Y, Z,,/)	

Note To delete a password, touch {Continue} to register without entering anything in the "New password" field.

6.1.5 Data Output - [S-lecting Data Output Destination]


The following destinations can be set for data output:

- · Output of test data to a printer (printing)
- · Output of test data to external media
- · Output of time course to external media
- · Output of test data to a computer


(Note) When the user is logged with an administrator ID, or when "No" is selected for "Operator mode," settings can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.

Page 182 "6.1.1 Sample Barcode Settings (Common)"

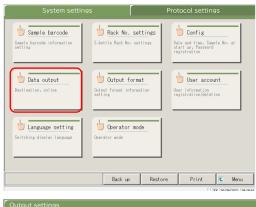


1 Touch {Data output}.

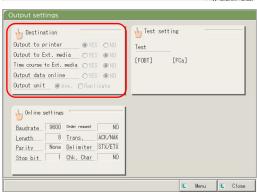

2 Touch {Destination}.

{Menu}: Returns to the [MENU] screen.

{Close}: Returns to the [System Settings] screen.

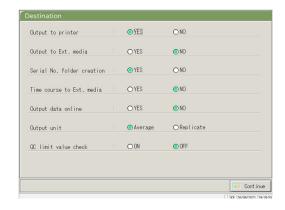


Select the destinations for data output (see the next page).


6 Settings

Setting	Value	Meaning
Output to printer		Selects whether or not test data is printed.
	YES	During a test, the test data is printed in real time.
		If "Print Saving mode" in which only the STD
		and QC process results and errors are printed is
		desired, please contact your seller.
	NO	Does not print test data.
Output to Ext. media		Selects whether or not test data is automatically
		sent to the external media upon completion of the
		test.
	YES	Output to the external media.
	NO	Not output to the external media.
Serial No. folder creation		Selects whether or not a serial No. folder is
		created on the external media when outputting
		test data.
	YSE	Creates a serial No. folder. (Ex. 00226)
	NO	Does not create a serial No. folder.
Time course to Ext. media		Selects whether or not a time course is output to
	VEC	the external media.
	YES NO	Output to the external media.
	NO	Not output to the external media.
Output data online		Selects whether or not the test data is output to a
		computer.
		Detailed settings for computer output are found
	VEC	on the [Online settings] screen.
	YES	Outputs online.
Output units	NO	Does not output online.
Output units		Selects the output format for sample or QC test data output online.
	Average	Outputs an average value of the output test
	Average	results.
	Replicate	Outputs all measured data.
QC limit value check		Selects the QC control limit value check.
	a	
	ON	If the data is normal as a result of the check, the
		test result is output as is. Otherwise the QC control
	0.55	limit value error is output.
	OFF	Outputs the measured results as-is.

6.1 System Settings

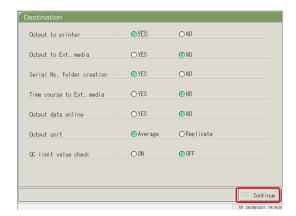

4 Touch {Continue}.

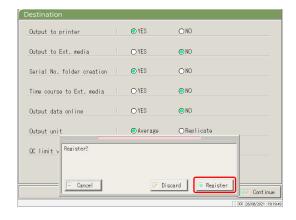

5 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts selection of data output and returns the screen to the [Data output] screen.

{Register}: Register the set contents.

6 Settings

6.1.6 Data output - [O-line Settings]

The control conditions for network communication are set.

For details, refer to "OC-SENSOR Ceres Interface Specifications."


Note When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," settings can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.

Page 182 "6.1.1 Sample Barcode Settings (Common)"



1 Touch {Data output}.

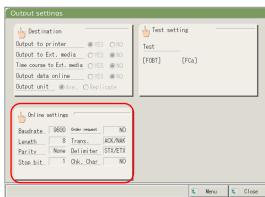

2 Touch {Online settings}.

 $\{Menu\}$: Returns to the [MENU] screen.

{Close}: Returns to the [System Settings] screen.

6 Settings



3 Select the conditions for network communication (see the next page).

Setting

IP address

Value


0.0.0.0	v - 0.	* Valid only when the communication mode is LAN or HL7.
	1 (5505	valid only when the communication mode is Erriv of TiE7.
Port No.	1 - 65535	
Outgoing/Incoming		
Order request		Selects whether or not order requests are sent to the upper level
		system.
	YES	Sends an order request.
	NO	Does not send an order request.
Com mode		Selects a communication mode.
	OC sensor IO	OC sensor IO compatible mode
	PLEDIA	OC sensor PLEDIA-compatible mode
	LAN	LAN communication mode
	ASTM	ASTM communication mode
	HL7	
D 1 .		HL7 communication mode
Baudrate	2400	Selects the communication speed.
	2400	2400 bps
	4800	4800 bps
	9600	9600 bps
	19200	19200 bps
	38400	38400 bps
Length		Selects the word length
Length	7	7 bits
	8	8 bits
	0	
		Select 8-bit when sending 2-byte characters.
Parity		Selects the parity check.
	None	Do not use parity check.
	Even	Check with even-number parity.
	Odd	Check with odd-number parity.
Stop bit		Select as the stop bit.
	1	1 bit
	2	2 bits
Trans.		Selects the transmission control.
	NONE	Does not use transmission control
	ACK/NAK	Uses ACK/NAK control.
Delimiter		Selects the delimiter.
	STX/ETX	Start of Text (0x02) / End of Text (0x03)
	CR/LF	Carriage Return (0x0D) / Line Feed (0x0A)
	CR	Carriage Return (0x0D) Carriage Return (0x0D)
Chk. Char	- CK	Selects the error detecting method.
Clik. Cliai	BCC	Block Check Code
	SUM	Sum
	NO	Does not use a check character.
Senarator	NO	
Separator		Selects whether or not commas (,) are used to separate data
	VEC	items.
	YES	Separated by commas.
	NO	Not separated by commas.

Meaning


Sets the IP and port number for the network connection.

0 - 254

- Note In the case of LAN or HL7 communication, the settings for delimiter, check character, and item separation become "none."
- Note For ASTM communication, the settings for delimiter, check character, and item separation are fixed. They cannot be changed on the setting screen.

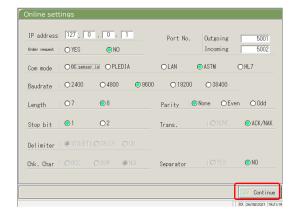

4 Touch {Continue}.

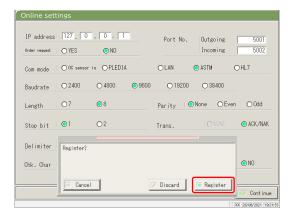

5 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts Online settings and returns to the

[Data output] screen.

 $\{Register\} \colon Registers \ the \ set \ content.$

6.1.7 Data Output - [T-st Setting]


The test items to be handled by the system are set here.

Note When the user is logged with an administrator ID, or when "No" is selected for "Operator mode," settings can be changed.


Note Settings are also available from the [Assay] screen, [Retest] screen, [QC process] screen, and the [Set reagent] screen.

The explanation starts from the condition at which the [System Settings] screen is displayed.

Page 182 "6.1.1 Sample Barcode Settings (Common)"

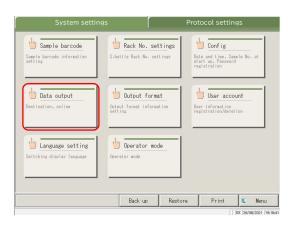


1 Touch {Data output}.

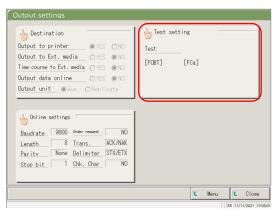

2 Touch {Test setting}.

 $\{Menu\}\colon Returns$ to the [MENU] screen.

{Close}: Returns to the [System Settings] screen.

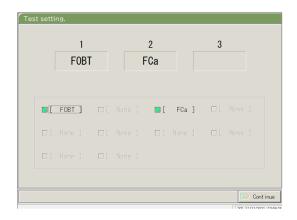


3 Selects the test items (see the next page).

Setting	Value	Meaning
Test item		selects the test item.
	[FOBT]	Fecal hemoglobin
	[FCa]	Calprotectin



4 Touch {Continue}.

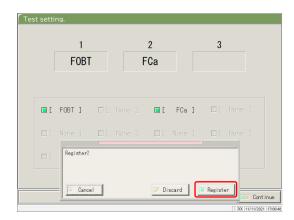

5 Touch {Register}.

 $\{Cancel\} \colon Closes \ the \ dialog \ box.$

{Discard}: Aborts the test setting and returns to the

previous screen.

 $\{Register\}\colon Registers\ the\ set\ content.$

6.1.8 Output Format Settings (Basic Format)


The output format of data such as the rack, sample No., sample ID, etc. can be set for each output destination (printer, external media, and network).

For data output to a printer, the settings for the rack, sample No., and sample ID cannot be changed.


Note When the user is logged with an administrator ID, or when "No" is selected for "Operator mode," the settings can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.


Continue

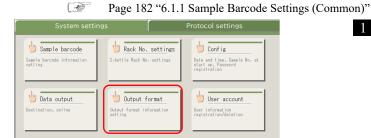


1 Touch {Output format}.

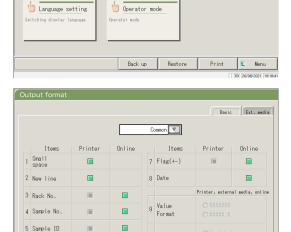

2 Select an output item (see page 202).

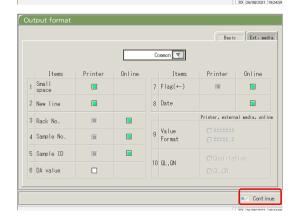

3 Touch {Continue}.

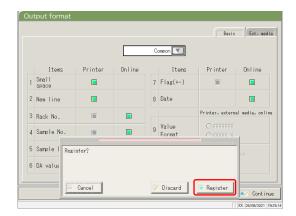

Note An item with a check mark is output. 4 Touch {Register}.

 $\{Cancel\} \colon Closes \ the \ dialog \ box.$

{Discard}: Aborts the output format and returns to the

[System Settings] screen. {Register}: Registers the set content.

6.1 System Settings

Output item	Meaning
Test item	Selects test items for output.
Common	common settings (items 1 - 8)–
90 : [FOBT]	FOBT setting (items 9 - 10–)
53 : [FCa]	FCa setting (items 9 - 10–)
1.Small space	
Printer	Line spacing becomes narrower when printing.
2. New line	
Printer	A line feed is output at the following positions during printing.
	· Between test data sets
	• Between items when testing multiple items
	(Applied for both real time and processed data printing.)
3. Rack No.	
Printer	Rack No. and Rack Position No. are output.
Online	Rack No. and Rack Position No. are output.
4. Sample No.	
Printer	Outputs sample Nos.
Online	Outputs sample Nos.
5. Sample ID	
Printer	Outputs sample IDs.
Online	Outputs sample IDs.
6. DA value	1 1
Printer	Prints DA values.
Timer	Outputs DA values.
7. Flag (+-)	1
Printer	Outputs judged results.
Online	Outputs judged results.
8. Date	
Online	Outputs test dates.
	•
9. Value Format	Selects the output format for the measured data.
###### (integer) ######.# (1st digit of decimal number)	Outputs integers.
######.# (1 digit of decilial number)	Outputs numbers with one decimal digit (all measured data values are rounded up.)
10. QL, QN	* <i>′</i>
Qualitative	Outputs the qualitative value of the measured results.
QL, QN	Outputs the qualitative and quantitative values of the measured results.

1

6 Settings

When the communication mode is set to ASTM on the [Online settings] screen, only the [Value Format] and [QL, QN] settings are applied, and other settings are not applied.

202

Note


6.1.9 Output Format Setting (External Media)

The data output to external media can be selected, and the output order can be changed.


When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," the settings can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.


Page 182 "6.1.1 Sample Barcode Settings (Common)"

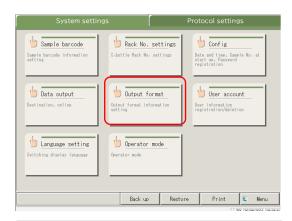

1 Touch {Output format}.

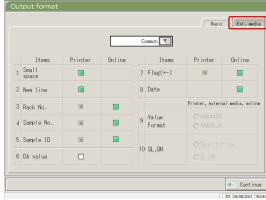


2 Touch the {Ext. Media} tab.

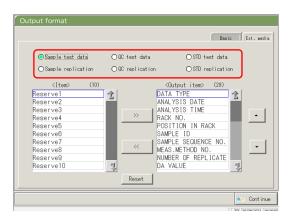

3 Select the test information to be changed using the radio buttons.

6.1 System Settings

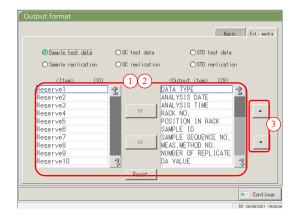

- 4 Change the order of output and select the output data.
 - Output items: List of items to be output


 Output order to: The data is output in the order shown.
- ① When an item on the left-side <Item> list is selected and the {>>} button is clicked, the corresponding item is added to the right-side <Output items> list.
- ② To delete an item from <Output items>, select the item in the list and touch the {<<} button.
- ③ Use the {▲} and {▼} buttons to change the order of a selected item.

{Reset}: Returns to the initial condition.


{ 1 }: Scrolls the list upward.

{ **!**}: Scrolls the list downward.

Note Some items are necessary for output. Such items cannot be deleted from the <Output items> list. For the items required for output, see 271 "4 Saving to External Media."

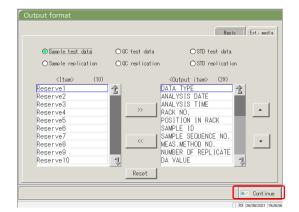

5 Touch {Continue}.

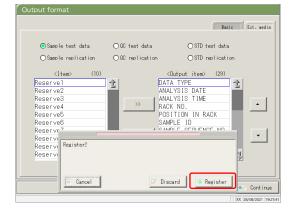

6 Touch {Register}.

{Cancel}: Closes the dialog box. {Discard}: Aborts output format and

returns to the [System Settings] screen.

{Register}: Registers the set content.


6.1.10 User Account Settings

Operator IDs can be registered, deleted, and changed.


Note If the current operator ID is "User," registration and changing the operator ID of the administrator is not possible. Setting an administrator ID is available only when the user is logged in with an administrator ID or when Operator mode is set to "NO."

The explanation starts from the condition at which the [System Settings] screen is displayed.

Page 182 "6.1.1 Sample Barcode Settings (Common)"

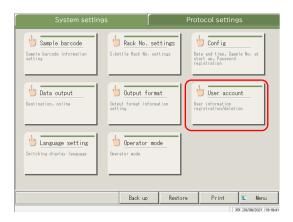

1 Touch {User account}.

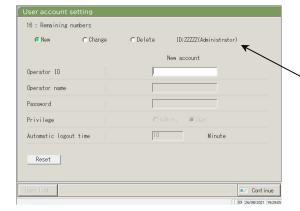

2 Select "New," "Change," or "Delete."

* At "ID:", the ID of the currently logged in operator is displayed.

3 Enter each setting.


<In case of a new registration>Register a new operator ID by entering the items in the following order.


- ① Operator ID
- ② Operator name
- ③ Password
- 4 Privilege
- (5) Automatic logout time

{Reset}: Clears the entered value.

The privilege returns to "User."

- Note) Up to 20 operator IDs can be registered.
- Note The operator ID registered first automatically becomes an administrator

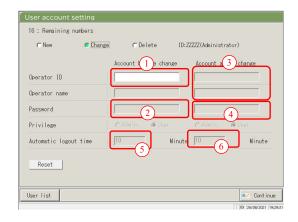

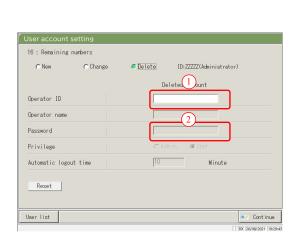

<To change an ID>

Enter the items in the following order for the operator ID to be changed.

- ① Enter the current operator ID.
 - * Selecting from the {User list} is also available.
 - * The current operator name, new operator ID and operator name are automatically filled in.
- ② Enter the current password.
- 3 Enter the new operator ID and name. The current operator ID and name can be left as they are.
- Enter a new password.To use a new password, enter a password different from the current password.

{Reset}: Clears the entered value.

The privilege returns to "User."

- ⑤ Set the new privilege.
- 6 Set the automatic logout time.
- <To delete an ID>

Enter the items in the following order for the operator ID to be deleted.

- ① Operator ID
 - * Selecting from the {User list} is also available.
 - * The Operator name will be automatically filled in.
- ② Password

{Reset}: Clears the entered value.

6.1 System Settings

Setting		Meaning
Operator ID	1 - 20-digits	Enter an operator ID.
	alphanumeric characters	<when changing="" deleting=""></when>
		Selecting from the {User list} is available.
Operator name	1 - 20-digits	Displays the name or the entered operator ID.
	alphanumeric characters	
		<when changing=""></when>
		Enter the new operator's name.
Password	5 - 10–digits	Enter a password.
	alphanumeric characters	
Privilege		Displays the privilege.
	Admin.	<when changing=""></when>
	User	The privilege can be changed.
Automatic logout	0 – 99 min.	While the [MENU] screen is displayed, if no
time		operation is conducted for this length of time,
		automatic logout occurs.



4 Touch {Continue}.



5 Touch {Register}.

{Cancel}: Closes the dialog box.

 $\{Discard\} \colon Aborts \ the \ user \ account \ setting \ and \ returns$

to the [System Settings] screen.

{Register}: Registers the set content.

 $\underset{\text{Settings}}{6}$


6.1.11 Language Settings

Application language can be selected.


(Note) When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," the settings can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.

Page 182 "6.1.1 Sample Barcode Settings (Common)"

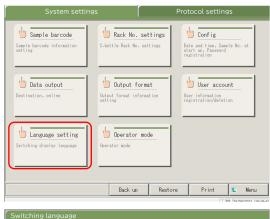

1 Touch {Language setting}.

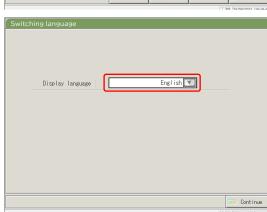

2 Touch {▼} to select a language.



3 Touch {Continue}.

6.1 System Settings



4 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts changing the language and returns to

the [System Settings] screen. {Register}: Enables the selected language.

- 5 After changing the language, restart the system.
- ① Shut down the system.

Page 89 "2.7 Shutting Down the System"


② Press the System switch to start up the system.

6.1.12 Operator Mode


The operator mode can be changed.

The explanation starts from the condition at which the [System Settings] screen is displayed.


Page 182 "6.1.1 Sample Barcode Settings (Common)"

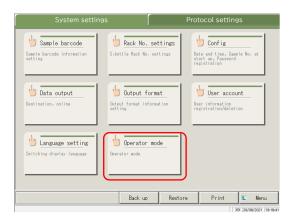

1 Touch {Operator mode}.

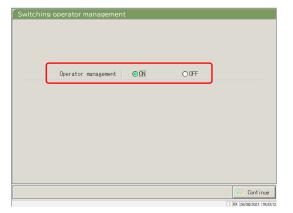


2 Switch the Operator mode.

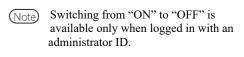



3 Touch {Continue}.

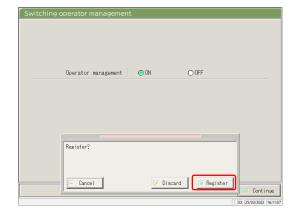

4 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts switching the Operator mode and

returns to the [System Settings] screen.

{Register}: Registers the set content.

<When changed from "ON" to "OFF">
Returns to the [MENU] screen.
Continue using the system as is.




<When changing from "OFF" to "ON"> Returns to the [MENU] screen and logs-out. Touch the LOGIN button to log in.

Page 28 "2.3 Logging In"

6.1.13 Printing the System Settings

The information registered in System Settings can be printed.

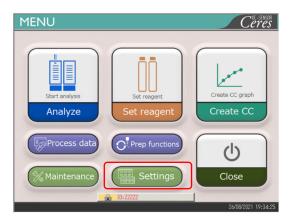

1 Touch {Settings}.



- 2 Touch {Print}.
 - * All the settings registered in [System Settings] are printed.


The system settings and protocol settings can be backed up.


(Note)

When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," backup is possible.





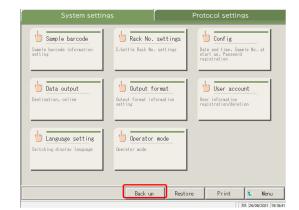

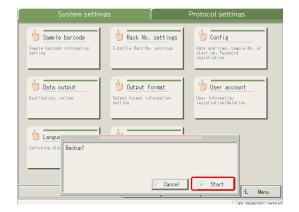

1 Connect the management USB stick to the system.

For the management USB stick, see Page 283 "Appendix 5 Management USB Stick Setting"

2 Touch {Settings}.

3 Touch {Back up}.

6 Settings

4 Touch {Start}.


* [System settings] and [Protocol settings] are backed up to the external media.

{Close}: Aborts backup. {Start}: Executes backup.

6.1.15 Restoring System Settings and Protocol Settings


System settings and protocol settings can be restored from external media.

(Note) When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," restoration becomes possible.

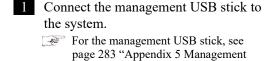

USB Stick Setting"

Touch {Settings}.

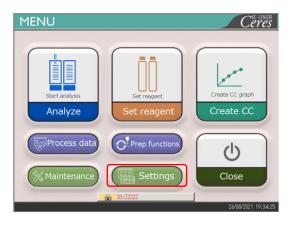

Sample barcode
Sample barcode information
Settings
Sample barcode information
Settings
Sample barcode information
Settings
Substance information
Setting Data output
Destination, online

Output format
Output format
Output format
Output formation
Setting
Operator mode
Operator mode
Operator mode

Data output

Operator mode
Operator mode
Operator mode

Destination

Operator mode

Destination


Operator mode

Operator mode

Destination

Operator mode

Operator mode

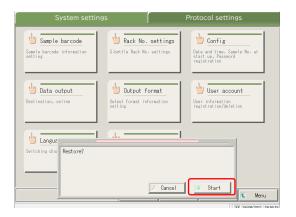

3 Touch {Restore}.

4 Touch {Start}.

* [System Settings] and [Protocol settings] are restored from the external media.

{Close}: Aborts restoring. {Start}: Executes restoring.

6.2 Protocol Settings

The protocol settings can be roughly divided into the following three types:


- · Conditions for samples and QC samples (Sample/QC protocol settings)
- CC protocol settings
- · Common conditions for testing samples, STD samples, and QC samples (maker settings)

Protocol settings do not need to be changed for normal test flows unless the contents of the flow are changed. In addition, the maker settings are set before shipping from the factory and are not available for users.

6.2.1 Sample/QC Protocol Settings

Conditions to test the samples (including interrupted samples) and QC samples can be set. The setting screen consists of two pages, and the tabs are used to switch the pages.

(Note) When the user is logged in with an administrator ID, or when "No" is selected for "Operator mode," the settings can be changed.

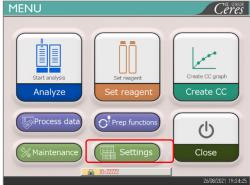


2 Touch the {Protocol settings} tab.


1 Touch {Settings}.




6.2 Protocol Settings

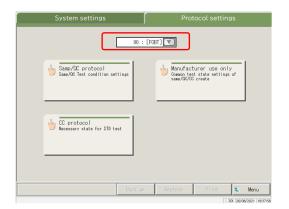


3 Select a test item from the pull-down menu.

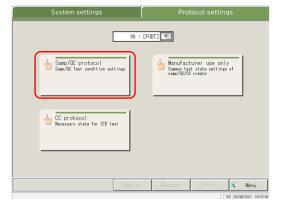

4 Touch {Samp/QC protocol}.

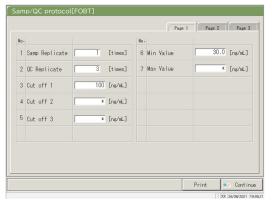

5 Set each item (see page 220).

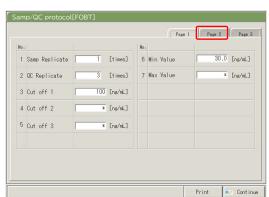


 $\{Print\}\colon Prints$ the [Sample and QC Protocol] settings.

6 Touch the {Page 2} tab.


{Print}: Prints the contents of [Samp/QC protocol].


6



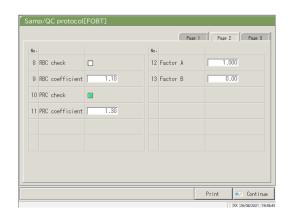



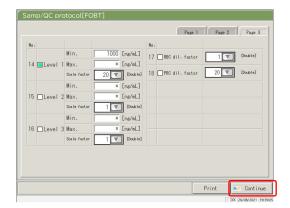

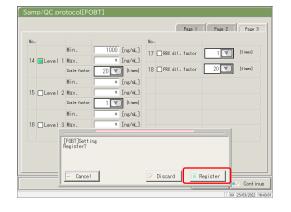

- 7 Set each item (see page 221).
- {Print}: Prints the contents of [Samp/QC protocol].
- 8 Touch the {Page 3] ta}.
- {Print}: Prints the contents of [Samp/QC protocol].
- 9 Set each item (see page 222).

{Print}: Prints the contents of [Samp/QC protocol].

10 Touch {Continue}.

11 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts and returns to the [Protocol settings]

{Register}: Registers the set content.

{Page 1} tab

	Setting	Value	Meaning		
1	Samp Replicate	1 – 10 times	Sets the nun	Sets the number of sample tests.	
2	QC Replicate	1 – 10 times	Sets the nun	nber of QC sample tests.	
3	Cut off 1		Sets the crite	eria for Cut off 1.	
		0 - 99–9999 (integer)	1+	Measured data is higher than "Cut off 1" or equal to or lower than "Cut off 2"	
			-	Measured data ≤ Cut off 1	
4	Cut off 2		Sets the crite	eria for Cut off 2.	
		, 0 - 99–9999 (integer)	2+	Cut off 2 < Measured data ≤ Cut off 3 When "" is set, judging Cut off 2 is emitted. Cut off 1 < Cut off 2	
5	Cut off 3	*, 0 - 99–999	Sets the crite	eria for Cut off 3.	
		(integer)	3+	Measured data exceeds the set value. When "*" is set, judging Cut off 3 is emitted. Cut off 2 < Cut off 3	
6	Min Value	0.0 - 99–9999.9	UR	Displayed when the measured data is "Min. value or lower" in diluent retesting.	
7	Max Value	0.0 - 99–9999.9	OR	Displayed when the measured data is "Max value" or higher.	

{Page 2} tab

	Setting	Value	Meaning
8	RBC check		Sets whether or not the RBC check is conducted. If conducting the check, set by clicking the box. Page 238 " RBC method"
9	RBC coefficient	1.00 - 20	At the detection point of the RBC method, when the sample ABS becomes larger than STD-6 ABS x "RBC coefficient," that sample is judged "prozone sample." Page 238 " RBC method"
10	PRC check		Sets whether or not the PRC check is conducted. If conducting the check, set by clicking the box. Page 239 " PRC method"
11	PRC coefficient	0.01 - 20	At No. 2 detection point (T2), when the sample DA2 value becomes larger than STD-6 DA2 value x "PRC coefficient," that sample is judged to be a "prozone sample." Page 239 " PRC method"
12	Factor A	0.001 - 99–9.999	Measured data = A x Concentration value + B
13	Factor B	-999.999 - 99–.999	

{Page 3} tab

	Setting		Value	Meaning
14	Level 1	Check		Sets whether or not the automatic retest is
				conducted for samples within the Level 1 range.
		Min.	*, 0 -	Sets the minimum value.
			9999999	
		Max.	*, 0 -	Sets the maximum value.
			9999999	Min. value < Max. value
		Scale	1	Sets the dilution factor for automatic retest.
		factor	10	
			20	
			100	
			200	
			400	
15	Level 2	Check		Sets whether or not the automatic retest is
				conducted for samples within the Level 2 range.
		Min.	*, 0 -	Sets the minimum value.
			9999999	
		Max.	*, 0 -	Sets the maximum value.
			9999999	Min. value < Max. value
		Scale	1	Sets the dilution factor for automatic retest.
		factor	10	
			20	
			100	
			200	
			400	
16	Level 3	Check		Sets whether or not the automatic retest is
) (°	* 0	conducted for samples within the Level 3 range.
		Min.	*, 0 -	Sets the minimum value.
		3.5	9999999	
		Max.	*, 0 -	Sets the maximum value.
			9999999	Min. value < Max. value
		Scale	1	Sets the dilution factor for automatic retest.
		factor		
			20	
			100	
			200	
			400	

	Setting		Value	Meaning
17	RBC dil. factor	Check		Sets whether or not an automatic retest is
				conducted for the prozone samples in the RBC
				method.
				Page 238 "■ RBC method"
		Scale	1	Sets the dilution factor for automatic
		factor	10	retesting.
			20	Page 238 "■ RBC method"
			100	
			200	
			400	
18	PRC dil. factor	Check		Sets whether or not an automatic retest is
				conducted for the prozone samples in the PRC
				method.
				Page 239 "■ PRC method"
		Scale	1	Sets the dilution factor for automatic
		factor	10	retesting.
			20	Page 239 "■ PRC method"
			100	
			200	
			400	

Note To perform automatic retest of OR, set the Level 1 Min. and Max values as follows.

Ex. In case of FOBT, because the test Max. value is 1000 ng/mL, set 1000.

Level 1 Min. value = 1000

Max value = *

6.2.2 CC Protocol Settings


The necessary conditions for STD sample measurements can be set.

The setting screen consists of two pages, and the tabs are used to switch pages.


Note When the user is logged with an administrator ID, or when "No" is selected for "Operator mode," the settings can be changed.

The explanation starts from the condition at which the [Protocol settings] screen is displayed.


Page 217 "6.2.1 Sample/QC Protocol Settings"

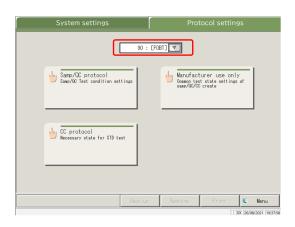

1 Select a test item.

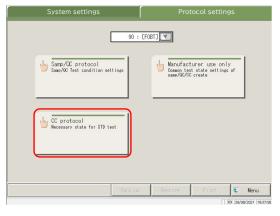


2 Touch {CC protocol}.

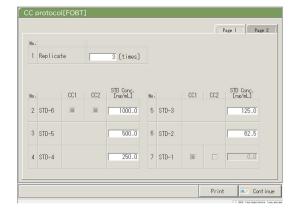


- 3 Set each item.
 - Page 226 "{Page 1} tab"

 $\{Print\} \qquad : Prints \ the \ [CC \ protocol] \ settings.$

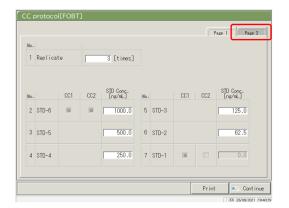


4 Touch the {Page 2} tab.

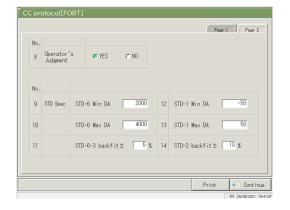

5 Set each item.


Page 226 "{Page 2} tab"

 $\{Print\} \qquad : Prints \ the \ [CC \ protocol] \ settings.$

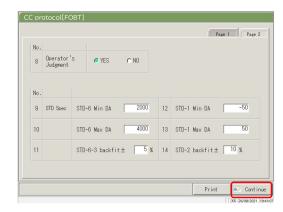

6 Touch {Continue}.

6 Settings

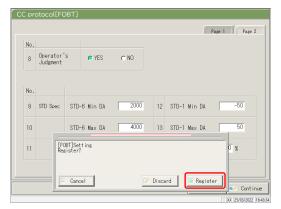

7 Touch {Register}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts and returns to the [Protocol settings] screen.

{Register}: Registers the set content.

6.2 Protocol Settings

{Page 1} tab

	Setting	Value	Meaning
1	Replicates	1 – 10	Sets the number of STD sample tests.
2	STD-6	0 - 99–99.9	Sets the concentration value which is described in the
3	STD-5		calibrator operation manual.
4	STD-4		
5	STD-3		
6	STD-2		
7	STD-1	(No input)	

{Page 2} tab

8 Operator's Judgment Sets whether or not the operator judged of CC. YES The system enters standby after the order to wait for the operator's judged	
YES The system enters standby after the order to wait for the operator's judg	ges after creation
order to wait for the operator's judg	
	creation of CC in
	ment.
The operator does not judge.	
NO When the measured data are normal	, the measurement
continues. When the measured data	are abnormal, wait
for the operator's judgment. The sys	stem enters
standby.	
STD Spec. Sets the standard value for each STI	O point.
The STD judgment is based on the s	specified values.
9 STD-6 DA Min DA -9999 - Checks against the DA1 value.	
99–99	
10 STD-6 DA Max DA -9999 - Sets as the Min. value < Max. value	•
99_99	
11 STD-6 - 3 –rigin value 0 - 10– Checks for deviation from the origin	ı value.
± □ %	
12 STD-1 DA Min DA -9999 - Checks against the DA1 value.	
99–99	
13 STD-1 DA Max DA -9999 - Sets the Min. value < Max. value.	
99_99	
14 STD-2 DA Origin value 0 - 10— Checks for deviation from the origin	ı value.
±□ %	

Note For the checking of the STD and QC measured data, see page 235 "1.1 STD/QC Sample Measured Data Check."

Chapter 7 Error Handling


- 7.1 How to Read the [FUNCTION ERROR] Screen
- 7.2 Error Handling Buttons
- 7.3 Cancelling Errors



Chapter 7 Error Handling

When an error occurs, the [FUNCTION ERROR] screen opens. If several errors have occurred, they are displayed on multiple pages. Touch {Next err} to display them.

7.1 How to Read the [FUNCTION ERROR] Screen

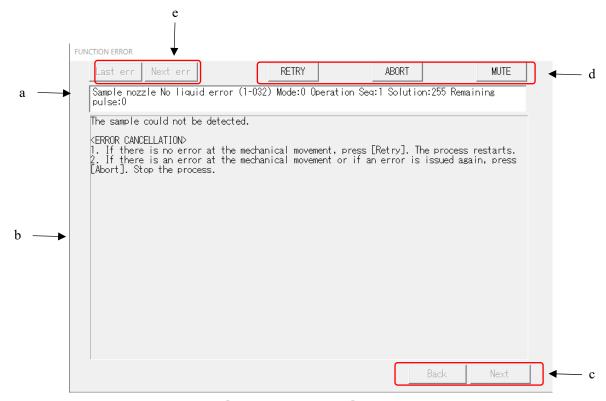

[FUNCTION ERROR] Screen

a	Error	Error No. and Error name.
b	Cause and method of cancellation	Explains errors and displays the cancellation method.
С	Page switching button	When the handling method for one error is explained over several pages, the pages can be switched.
	{Back}	Displays the previous page.
	{Next}	Displays the next page.
d	Handling buttons	Cancels an error.
		Page 229 "7.2 Error Handling Buttons"
e	Error select buttons	When several errors have occurred, error display is switched.
	{Last err}	Displays the previous error.
	{Next err}	Displays the next error.
-		

7 Error Handling

Error Handling Buttons


At the top of the [ERROR FUNCTION] screen, there are buttons to handle the errors. Operations after touching the cancellation button are as follows.

Button name	Function	
{RETRY}	Retries the sample process for which the error occurred.	
{ABORT}	Ends the testing. When selected during testing, finishes the current sample process.	
{Continue}	Resumes the operation of the mechanism stopped by the error.	
{Close}	Closes the screen.	
{MUTE}	Mutes the alarm sound.	
{Shut down}	Shuts down the system. It may require a few minutes until the system shuts down.	

(Note) The cancellation button names change depending on the error.


7.3 Cancelling Errors

When an error occurs, cancel the error as follows:

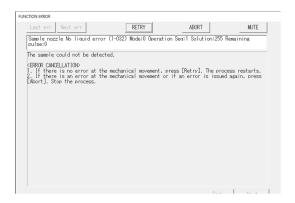


- 1 Check the error.
 - ① Check the section with the error, and the error details.
 - ② Touch one of the error handling buttons.

 Page 229 "7.2 Error Handling Buttons"

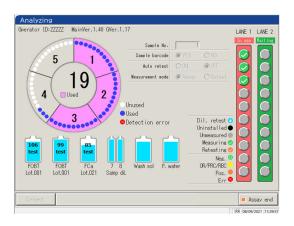


When a button other than {Abort} is selected, the error is cancelled, and the operation continues.

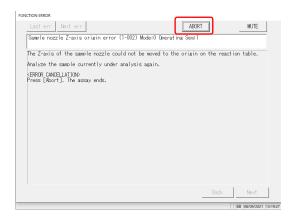

2 (Continued)

<When {ABORT} is selected>

① Touch { ABORT}.

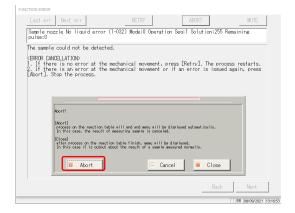

- ② Touch { Abort}.
 - * Forcibly ends the processes on the reaction table and returns to the [Assay] screen.

 The results of samples testing are discarded.

{ Abort}: Forcibly ends the process and returns to the [Assay] screen.

{Cancel}: Aborts the forcible end and returns to the [Error] screen.

{Close}: Returns to the [Assay] screen upon completion of the process on the reaction table.

Note

For the following types of errors, touching {ABORT}

ends the process forcibly without opening the dialog box for forcible end execution.

- Errors that occur during maintenance or prep functions (not during testing)
- Errors that occur on the reaction table (Errors which cannot be closed)

Appendix

- 1 Calculation Processes
- 2 Printing Example
- 3 Error List
- 4 Saving to External Media
- 5 Management USB Stick Setting



1 Calculation Processing

The following calculation processes are performed during an analysis.

- 1.1 STD/QC Sample Measured Data Check
- 1.2 DA Value Calculation
- 1.3 Measured Data (Concentration) Calculation and Qualitative Assessment
- 1.4 Prozone Check
- 1.5 Reagent Blank Check
- 1.6 Calculate Cell Blank

1.1 STD/QC Sample Measured Data Check

Check whether the measurement data of STD samples or QC samples are proper values.

1. STD sample measured data check

- ① By looking at the large and small DA1 values at each point, check that the CC is showing an ascending curve.
- ② Check that the DA1 values of adjacent points are not the same. If the values are the same, the condition becomes "Abnormal."
- ③ Based on the STD standard set on the [CC Protocol] screen, check the deviation (%) between the theoretical value and the back fit value.
 - STD-1 "Abn"rmal" if DA is out of min/max range.
 - STD-2 If the deviation is deviated from the set value, the
 - STD-5 condition becomes "Abnormal."
 - STD-6 "Abn"rmal" if "DA is out of min/max range.

The data is checked based on the limit value set on the {Set} tab on the [QC process] screen.

The condition becomes abnormal under the following conditions:

- · When the QC setting is lower than the min value
- 2. QC sample measured data check
 - · When the QC setting is the max value or above

1.2 DA Value Calculation

The DA value is the final measurement result. Measured data (concentration value: X) are calculated from DA values and calibration curves. When a reagent is dispensed to a cell that has a sample dispensed, latex agglutination occurs and the absorbance (ABS) changes.

The time course shows changes in absorbance.

The time course is monitored once a sample is dispensed into a cell (see Figure 5).

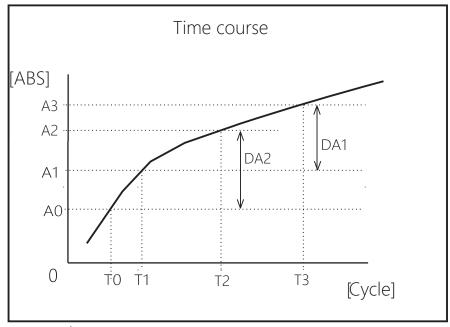


Figure 5

T0: 0th detection point

A0: T0 absorbance

T1: 1st detection point

A1: T1 absorbance

T2: 2nd detection point

A2: T2 absorbance

T3: 3rd detection point

A3: T3 absorbance

A0,A1, A2, and A3 are the mean values of absorbances of the former and current cycles.

(Ex.)
$$T0 = 3,T1 = 4, T2 = 5, T3 = 12$$
 [cycle]

 $A0 = (Absorbance of the 3rd c^{yc}le + Absorbance of the 2rd c^{yc}le)/2$

 $A1 = (Absorbance of the 4th c^{yc}le + Absorbance of the 3rd c^{yc}le)/2$

 $A2 = (Absorbance of the 5th c^{yc}le + Absorbance of the 4rd c^{yc}le)/2$

A3 = (Absorbance of the 12th c^{yc} le + Absorbance of the 11th c^{yc} le)/2

However, if you set the 1st or 2nd c^{yc}le, it will be the absorbance of the set cycle.

DA values are the amount of change for A0,A1, A2, and A3.

DA1: A3 - A1

DA2: A2 - A0

1.3 Measured Data (Concentration) Calculation and Qualitative Assessment

Measured data (concentrations) are calculated using DA values and calibration curves. The qualitative assessment is performed by comparing the corrected measured data and the cut-off values.

1. Calculate measured data (concentrations) using DA values and calibration curves.

Page 236 "1.2 DA Value Calculation"

2. Range check by minimum/maximum measurement value*

OR : Displayed when the measured data has exceeded the maximum

measurement value.

Concentration : Displayed when the measured data is at or below the maximum

measurement value.

UR : Displayed when the measured data is lower than the minimum

measurement value.

Page 217 "6.2.1 Sample/QC Protocol Settings"

* Measured data is the value prior to correction such as correction by factor A/B or by applying a dilution ratio.

1.4 Prozone Check

Prozone checking is conducted using two methods: RBC and PRC.

RBC method: Compares the absorbance of the RBC points in STD-6 (maximum concentration point) and the sample.

PRC method: Compares the DA2 values in STD-6 (maximum concentration point) and the sample.

■ RBC method

When comparing the RBC point sample absorbance (a) and STD-6 absorbance (b), if the following relationship is formed, the sample is designated as a "prozone sample."

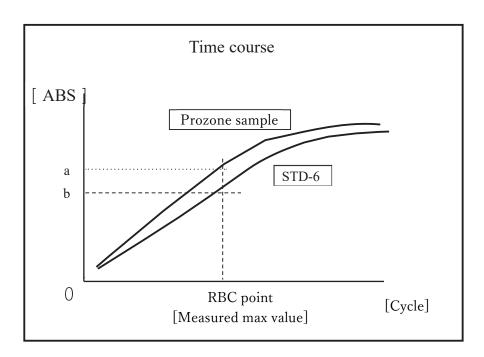


Figure 6

(Note)

RBC method coefficient is set by "9 RBC method coefficient" on {Page 2}a of [Protocol setting] - [S-mple/QC Protocol setting] screen.

Note

Sample absorbance (a) and STD-6 absorbance (b) are the mean values of the RBC point and the former point respectively.

(Ex.) When the RBC point is 5

(a) and (b) are

(Absorbance of 5th cycle + absorbance of 4th cycle)/2.

PRC method

When comparing the DA2 values of the sample and STD-6, if the following relationship is formed, the samples are designated as "prozone samples."

 $DA2_{STD-6} \times PRC$ method coefficient < DA2 sample

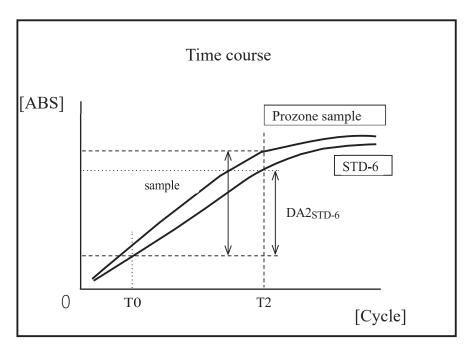


Figure 7

DA2 sample: DA2 value of prozone sample

DA2_{STD-6}: DA2 value of STD-6

1.5 Reagent Blank Check

A1 Check

Normal dispensing of the reagent is checked by using the T1 cycle absorbance.

The absorbance is obtained from the mean value of T1 cycle ADC value and T1-1 cycle ADC value. The value is compared with the max value and the min value of the A1 check.

The "reagent blank error" is output if the condition is as follows.

Mean absorbance < A1 check min value or

Mean absorbance > A1 check max value

Note Absorbance = log_{10} (ADC space/ADC mean value) × 10000 – Cell blank value

Note A1 check max and min values are set by the manufacturer. Users cannot change the settings.

●DA1 Check

Normal dispensing of the reagent is checked by using the T1 cycle absorbance and the DA1 value.

The absorbance is obtained from the mean value of T1 cycle ADC value and T1-1 cycle ADC value. The value is compared with the max value of the A1 check. Also, the DA1 value is compared with the min value of the DA1 check.

The "reagent blank error" is output if the condition is as follows.

Mean absorbance ≧ A1 check max value and

DA1 value ≦ DA1 check min value

(Note) [(A1 Check) A1 check max value] and [(DA1 Check) A1 check max value] are different.

Note A1 check max value and DA1 check min values are set within the device. Users cannot change the settings.

*If an error occurs during the reagent blank check, please check the condition of the sample and reagent bottle and retest. If the problem occurs again, please contact the manufacturer or a legal representative. If an error occurs during the DA1 check, the cell may need to be investigated, so please save the cell where the reagent blank error occurred.

1.6 Calculate Cell Blank

After the analysis starts, detection is performed for all cells (55 cells) for each cycle.

The cell blank value of each cell is calculated from the following formula using ADC value (ADC space, ADC blank cell) which was measured while the cell had no sample dispensed.

Cell blank value = log_{10} (ADC space/ADC blank cell) x 10000


ADC space: ADC value of the space between cells (no cell condition)

ADC blank cell: ADC value when a blank cell is measured

2 Printing Examples

This section describes various printing examples.

- 2.1 Positive Sample printing
- 2.2 Final Result Printing at Retesting
- 2.3 Printing when STD Sample and QC Sample are Measured
- 2.4 Printing Error Messages

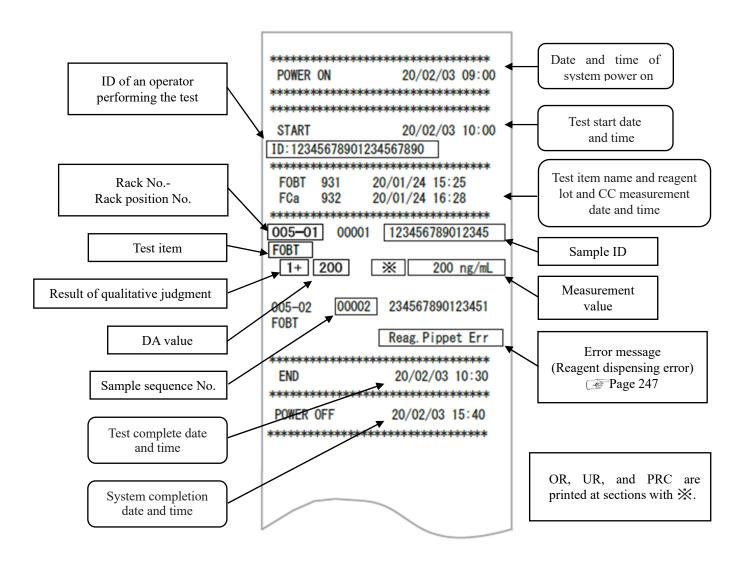

2.1 Positive Sample Printing


<When the number of replications is 1>

"Sample information" and the positive "test result" are printed in bold letters.

005-01	00001	123456789012345
FOBT -	50	50 ng/mL
005-02	00002	234567890123451
FOBT 1+	200	200 ng/mL

<When the number of replications>1 and the mean value is a positive sample>Only the line containing the mean value is printed in bold.

(Example: Replications = 3)

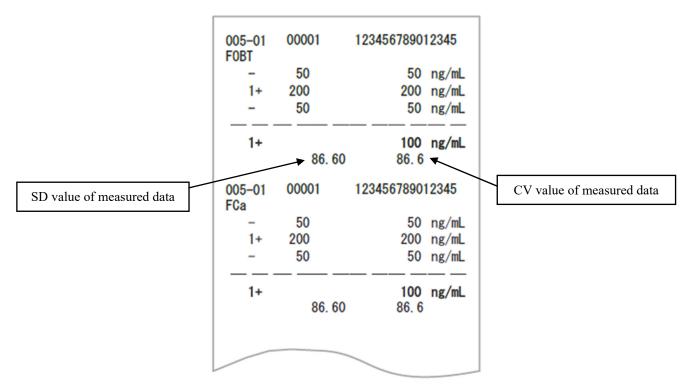

Note The calculation of average depends on digit setting (integer or tenths place).

Printed value is rounded to the set digit.

9. Value Format under "6.1.8 Output Format Settings (Basic Format)" on page 200.

2.2 Final Result Printing at Retesting


The Dilution factor is printed after the item name. Other sections are the same as the first test.

<When the number of replications is 1>


094-03 F0BT 1+ FCa - 040-04 F0BT - FCa	00001 (A100) 200 (A) 50 00002 (A200) 40 (A100) 34	200 ng/mL 50 ng/mL 123456789012345 40 ng/mL 34 ng/mL	Dilution factor A: Retest without dilution A10: Dilute by a factor of 10 and retest A20: Dilute by a factor of 20 and retest A100: Dilute by a factor of 100 and retest A200: Dilute by a factor of 200 and retest A400: Dilute by a factor of 400 and retest

2 Printing Examples


<When the number of replications is larger than 1>

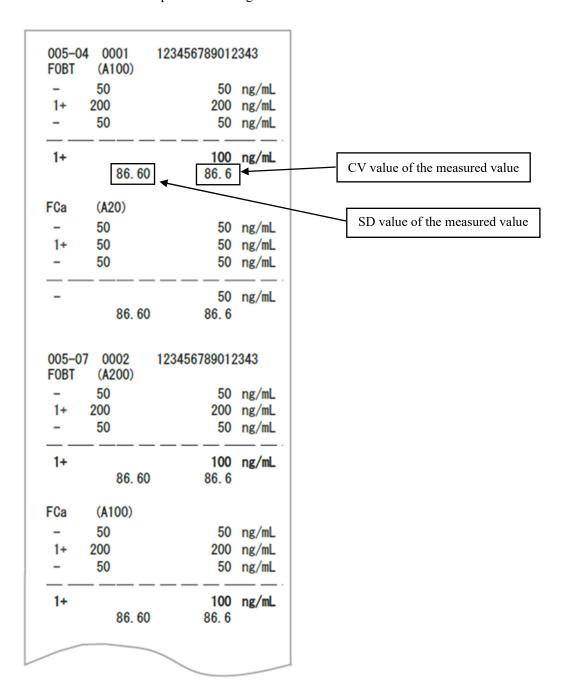

2.3 Printing when STD Sample and QC Sample are Measured

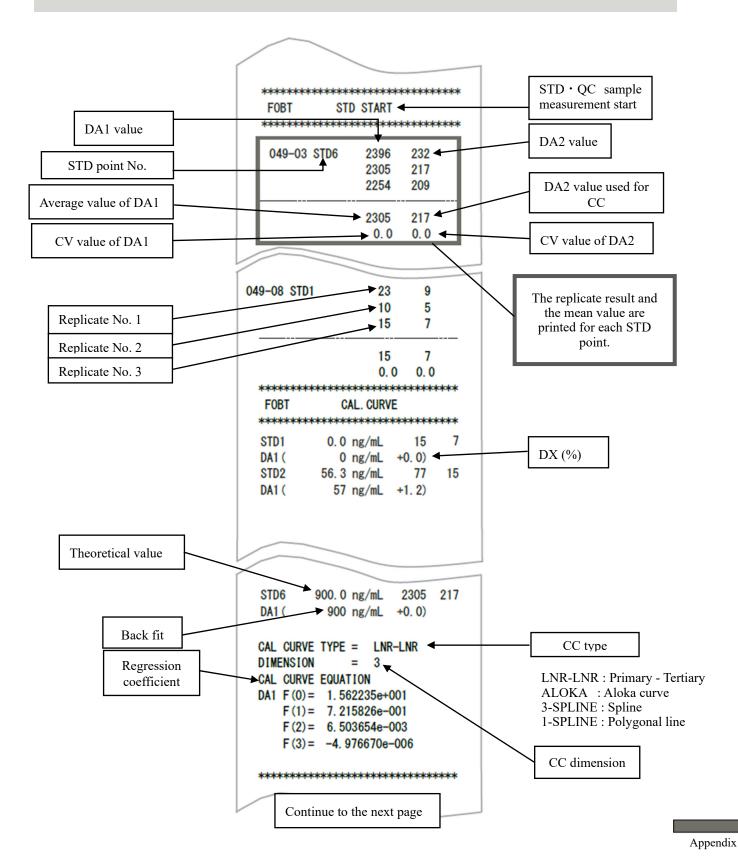


2 Printing Examples

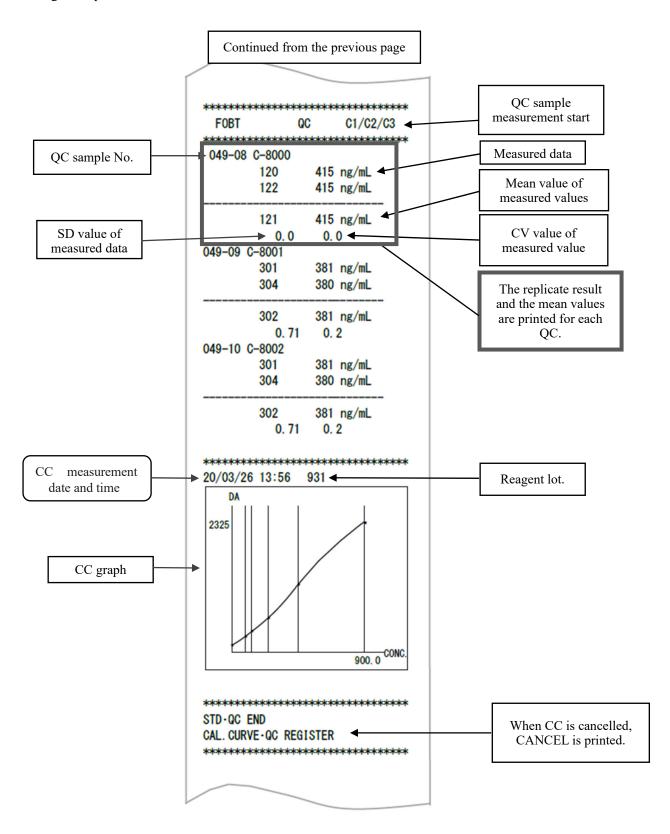

2.4 Printing Error Messages

Meanings of error messages are as follows.

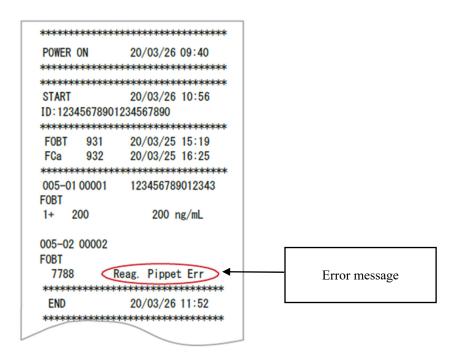

Printed Error Message	Meaning
Cell Blank Error	Cell blank error
No Sample	No sample
Sample Fusoku	Sample short
No Reagent1	No R1 reagent
No Reagent2	No R2 reagent
Mixing Error	Mixing error
Reag.Blank Error	Reagent blank error
Cal.Curve1 Error	STD error (DA1)
Cal.Curve2 Error	STD error (DA2)
Samp.Pippete Err	Sample not dispensed
R1. Pippete Err	R1 reagent no dispensing
R2. Pippete Err	R2 reagent no dispensing
Sample Blank Err	Sample amount error
Control Error	Control error

This section describes error messages that are displayed on the screen, as well as error messages printed by the printer.

ERR# 0-1001 to 0-3005 (Main) 3.1

Error No.	Screen Message (Upper)	
ERR#0-	Printed Message (Lower)	Error Description and Cancelling Procedure
ERR#	0-1001 -	
	G communication error	(Not displayed on screen)
1001	G communication error	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[SHUT DOWN] Turn off the system.
	Unexpected command	(Not displayed on screen)
1003	received	Operation continues automatically.
	Unexpected command received	
1004	TELEGRAM DATA ERROR	(Not displayed on screen)
1004	TELEGRAM DATA ERROR	Operation continues automatically.
	No reply from GLIFE	(Not displayed on screen)
1005	command	Touch [SHUT DOWN] to turn off the power.
	No reply from GLIFE command	Finish the sample process currently being tested.
	No reply from G command	(Not displayed on screen)
1006	No reply from G command	Touch [SHUT DOWN] to turn off the power.
		Finish the sample process currently being tested.
	Abort failed	(Not displayed on screen)
1007	Abort failed	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
1007		[SHUT DOWN] Turn off the system.
		[CLOSE] Close error display
	Multiple absorbance data	INF1: Ready counter
	received in the same cycle.	* No effect on the measurement data.
1008	Multiple absorbance data	However, if it occurs repeatedly, contact the seller.
1000	received in the same cycle.	<error cancellation=""></error>
	Online ACK Timeout	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[CLOSE] Close error display

Error No.	Screen Message (Upper)		
ERR#0-	Printed Message (Lower)	Error Description and Cancelling Procedure	
ERR#	ERR# 0-1100 -		
	Online ACK timeout error	(Not displayed on screen)	
1101	Online ACK Timeout	An error occurred in communication with an external computer.	
		Operation continues automatically.	
	Online NAK count error	(Not displayed on screen)	
1102	Online NAK count	An error occurred in communication with an external computer.	
		Operation continues automatically.	
	Online data reception error	(Not displayed on screen)	
1103	Online data reception error	An error occurred in communication with an external computer.	
		Operation continues automatically.	
	Online connection error	(Not displayed on screen)	
1111	Online connection error	An error occurred in communication with an external computer.	
		Operation continues automatically.	
	Online err [NG RECEIVE]	(Not displayed on screen)	
1112	Online err [NG RECEIVE]	An error occurred in communication with an external computer.	
		Operation continues automatically.	
	Online err [ABNORMAL	(Not displayed on screen)	
1113	DATA]	An error occurred in communication with an external computer.	
1113	Online err [ABNORMAL	Operation continues automatically.	
	DATA]		
	Online err [TIME OUT]	(Not displayed on screen)	
1114	Online err [TIME OUT]	An error occurred in communication with an external computer.	
		Operation continues automatically.	
ERR#	0-1400 -		
	Insufficient test reagents	The reagent ran out.	
	Insufficient test reagents	Set the reagent after the assay completes.	
1409		<error cancellation=""></error>	
1109		<error cancellation=""></error>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
		[CLOSE]: Close error display.	
	Insufficient buffer volume	Buffer ran out.	
1410	Insufficient buffer volume	Set the buffer after the assay completes.	
		<error cancellation=""></error>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
		[CLOSE]: Close error display.	

Error	Screen Message (Upper)		
No.	Printed Message (Lower)	Error Description and Cancelling Procedure	
ERR#0-	Timica Message (Lower)	Error Beseription and Cantering Procedure	
	Double sample barcode	(Not displayed on screen)	
1416	Double sample barcode	A sample barcode is duplicated. This sample will not be tested.	
	1	Operation continues automatically.	
	Sample barcode reading error	(Not displayed on screen)	
1417	Sample barcode reading error	A sample barcode couldn't be read. This sample will be tested.	
		Operation continues automatically.	
	Sample barcode digit error	(Not displayed on screen)	
		While reading sample barcodes, for one sample, the number of	
1418	Sample barcode digit error	barcode digits exceeded the limit was found. This sample will be	
		tested.	
		Operation continues automatically.	
1430	Cell check error	(Not displayed on screen)	
1430	Cell check error	A measurement cell has been set. Touch [ABORT] and set the cells.	
	Reagent blank error	(Not displayed on screen)	
1435	Reagent blank error	The reagent blank is abnormal. Operation continues automatically.	
1433		A1 check error : Additional information INF3:0	
		DA1 check error : Additional information INF3 : 1	
1.10.5	Cell blank error	(Not displayed on screen)	
1436	Cell blank error	Cell blank status is abnormal. Operation continues automatically. To	
		replace the cells, touch [ABORT].	
1.40=	Transportation Result Data Error	(Not displayed on screen)	
1437	Transportation Result Data Error	An error occurred in rack transfer.	
		Touch [ABORT].	
1.420	Container check results error	(Not displayed on screen)	
1438	Container check results error	CONTAINER CHECK FAIL	
		This sample will not be tested. Operation continues automatically.	
	1	Rack loading stopped due to reaching maximum number of sample	
		analysis after a system start up.	
	1	Close error display. Close the instrument under main analysis screen	
1439	reached Warning	of [Completed]	
		After system restart, new analysis can be started.	
		<error cancellation=""> PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR</error>	
		[CLOSE]: Close error display	
	Program error	PROGRAM ERROR	
	Program error	Close error display. Close the instrument under main analysis screen	
1 4 4 0	1 Togram Citor	of [Completed]	
1440		<pre><error cancellation=""></error></pre>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR	
		[CLOSE]: Close error display	

Error No.	Screen Message (Upper)	Error Description and Cancelling Procedure	
ERR#0-	Printed Message (Lower)		
	Detection start position error	Some cells output a detection start position error during cell	
	Detection start position error	replacement.	
	•	Cells with an error will not be used for the assay.	
1441		The cell may be tilted.	
1441		Check the cell placement and replace the cell again.	
		<error cancellation=""></error>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
		[CLOSE]: Close error display	
	Usable cells ran out	Usable cells ran out.	
	Usable cells ran out	After closing this error message, replace the cell on the Cell	
1442		replacement screen.	
11.12		<error cancellation=""></error>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
		[CLOSE]: Close error display	
1443	Purified water volume 0	(Not displayed on screen)	
1443	Purified water volume 0	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
	West in a start of a section of	[CLOSE]: Close error display	
1444	Washing solution volume 0	(Not displayed on screen) PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
1777	Washing solution volume 0	[CLOSE]: Close error display	
	Reagent double barcode	There are multiple reagents with the same barcode.	
1445	Reagent double bareoue	*P1 to P8: Presence of double barcode at installation positions 1 to 8	
1443		(0: Normal, 1: Double barcode)	
	Reagent double barcode		
	LED deterioration detected	LED has deteriorated. Check that there are no foreign objects on the	
	LED deterioration detected	table. If the issue persists, please contact the seller.	
		LED wavelength (nm): Status	
		Wavelength nm	
1446		660, 800, 600, and 340 Status	
1440		0: Normal, 1: Caution about deterioration, 2: Deterioration	
		detected	
		<pre><error cancellation=""></error></pre>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
		[CLOSE] Close error display	
	Caution about LED	LED may be deteriorated. Be careful.	
	deterioration	LED wavelength (nm): Status	
	Caution about LED	Wavelength nm	
	deterioration	660, 800, 600, and 340	
1447		Status	
		0: Normal, 1: Careful about deterioration	
		<error cancellation=""></error>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
		[CLOSE] Close error display	

Error No.	Screen Message (Upper)			
ERR#0-	Printed Message (Lower)	Error Description and C	Cancelling Procedure	
	Cell blank range value	Cell blank range value error occu	urred for some cells during cell replacement.	
	error	Cells with an error will not be used for the assay.		
1448	Cell blank range value	Check the cell condition on the C	Cell replacement screen.	
1770	error	<error cancellation=""></error>		
		PRESS THE FOLLOWING BU	TTON TO SOLVE THE ERROR.	
		[CLOSE] Close error display		
	Measurement	The measurement was interrupted		
	interruption occurred	INF1: Causes of interruptions No	o. 1 to 21	
		No.01: All cells were used	No.11: Container check results error	
		No.02: Cell shortage (for one te	st) No.12: Order error	
	Measurement	No.03: Cell shortage	No.13: Puncture error	
	interruption occurred	(for one consecutive samp	·	
		No.04: Empty cell	No.14: Sample dispense error	
		No.05: Double BC error	No.15: Reagent dispense error	
		No.06: Reagent shortage	No.16: Mixing error	
1449		No.07: Buffer shortage	No.17: Temperature error	
1117			(reagent refrigerator)	
		No.08: Purified water shortage	No.18: Temperature error (reaction table)	
		No.09: Washing solution shortage	-	
		No.10: LED deterioration error	No.20:Temperature communication error	
			(reagent refrigerator)	
			No.21: Temperature communication error	
			(reaction table)	
		<error cancellation=""></error>		
			TTON TO SOLVE THE ERROR.	
		[CLOSE] Close error display		
	Temperature error at	(Not displayed on screen)		
	reagent refrigerator		TTON TO SOLVE THE ERROR.	
1450	during analysis	[CLOSE] Close error display		
	Temperature error at			
	reagent refrigerator			
	during analysis			

Error No.	Screen Message (Upper)	
ERR#0-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Cell presence check error	In priming before the assay, some cells output the cell presence
	Cell presence check error	error.
		Blocks containing cells with an error will not be used for the assay.
1452		Check the cell condition on the Cell replacement screen.
		<error cancellation=""></error>
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[CLOSE] Close error display
	Rack error	Set rack error.
	Rack transfer error	INF1: Reason for the stop
		No.05: Rack barcode reading error
1453		No.06: Rack setting outside the range
		<error cancellation=""></error>
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[CLOSE] Close error display
1454	Sampling volume check error	(Not displayed on screen)
1.0.	Sampling check error	Operation continues automatically.
	Temperature error of reaction	(Not displayed on screen)
1455	table during analysis	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
1.00	Temperature error of reaction	[CLOSE] Close error display
	table during analysis	
	Usable reagent is not	(Not displayed on screen)
1456	installed.	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
	Usable reagent is not	[CLOSE] Close error display
	installed.	
	Temperature communication	(Not displayed on screen)
	error at reagent refrigerator	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
1457	during analysis	[CLOSE] Close error display
	Temperature communication	
	error at reagent refrigerator	
	during analysis	
	Temperature communication	(Not displayed on screen)
	error of reaction table during	PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
1458	analysis	[CLOSE] Close error display
1100	Temperature communication	
	error of reaction table during	
	analysis	

Error No.	Screen Message (Upper)	
ERR#0-	Printed Message (Lower)	Error Description and Cancelling Procedure
ERR	# 0-1500 -	
	Startup failed	(Not displayed on screen)
1501		Unable to start the system.
	Startup failed	Touch [SHUT DOWN] to turn off the power.
	Analysis end processing failed	(Not displayed on screen)
1502		Unable to complete testing.
	Analysis end processing failed	Touch [SHUT DOWN] to turn off the power.
	Program end processing failed	(Not displayed on screen)
1503	Program end processing failed	Unable to end the system.
		Touch [SHUT DOWN] to turn off the power.
	A required file cannot be found.	FILE MISSING
	No file	INF1: MISSING FILE No.
		1 = Config.ini
1504		2 = Support.ini
		<error cancellation=""></error>
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[SHUT DOWN] Shut down the system.
	func.ini is abnormal or	(Not displayed on screen)
1505	cannot be found	The file is missing.
	func.ini is abnormal	Touch [SHUT DOWN] to turn off the power.
	kinou.dat is abnormal or	(Not displayed on screen)
1506	cannot be found	The file is missing.
	kinou.dat is abnormal	Touch [SHUT DOWN] to turn off the power.
	Necessary common memory	(Not displayed on screen)
1509	cannot be found.	Memory cannot be found.
1307	Necessary common memory	Operation continues automatically.
	cannot be found.	

Error No.	Screen Message (Upper)	
ERR#0-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Common memory	PROGRAM ERROR (MEMORY) due to file setting failure
	initialization failed	INF1: MEMORY No.1 - 53–01: Sample barcode (SysTBcr.txt) 11: Sample cup
		(SysTube.txt)
		02: Rack information (SysRack.txt) 12: Order test (SysOdrAna.txt)
		03: Environment setting (SysEnv.txt) 13: Sample/QC protocol (PrtSmpQc.txt)
		04: Data output (SysDtOut.txt) 14: CC1-CC6 protocol (PrtCC.txt)
		05: Output format (SysForm.txt) 15: Common protocol 1 (PrtCom1.txt)
		06: RS232C setting (SysRS.txt) 16: Common protocol 2 (PrtCom2.txt)
		07: Analysis method (SysSoku.txt) 17: Analysis (AnaInf.dat)
		08: Alarm setting (SysAlm.txt) 18: Test mode (SokuMode.dat) 09: Screen saver (SysScrn.txt) 19: Reagent CC (SiyakuCC.dat) 10:
		STD/QC run (SysStdQc.txt) 20: Reagent volume (SiyakuZan.dat)
		<error cancellation=""></error>
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[SHUT DOWN] Shutdown the system.
		PROGRAM ERROR (MEMORY) due to file setting failure
		INF1: MEMORY No.1 to 53
		21: Buffer volume (KBufZan.dat) 31: System status ()
		22: Shift register () 32: Error information ()
		23: Supply units (KUnitinf.dat) 33: Subsystem ()
		24: Unload units (HUnitinf.dat) 34: Additional analysis ()
		25: Rack · Sample (RackInf.dat) 35: Test result ()
1511		26: Analysis data (AnaRslt.dat) 36: STD·QC result ()
		27: STD result data (StdRslt.dat) 37: Instrument information ()
		28: Cell blank analysis (CellBlnk.dat) 38: CC information ()
		29: Temperature information () 39: Printer ()
		30: Protocol DWLD () 40: Spare parts (U_Parts.dat)
		<error cancellation=""></error>
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[SHUT DOWN] Shutdown the system.
	(Messages will not be	PROGRAM ERROR (MEMORY) due to file setting failure INF1: MEMORY No.1 to 53
	printed.)	41: Spare parts (S Parts.dat) 51: Washing solution information (WashSol.dat)
	printed.)	42: Additional analysis recovery () 52: Retest order information ()
		43: Reagent · CC () 53: Switch display language (SysLang.txt)
		44: Auto Startup (AutoStart.txt)
		45: Log in information ()
		46: Cell management information (CellMng.dat)
		47: Rack lane information (RackLane.dat)
		48: Reagent history information (SRireki.dat) 49: Buffer information (KRireki.dat)
		50: Purified water information (Water.dat)
		<error cancellation=""></error>
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
		[Shutdown] Shutdown the system.

Error No.	Screen Message (Upper)	
ERR#0-	Printed Message (Lower)	Error Description and Cancelling Procedure
ERR# 0-1600 -		
	COM port is not properly	PRINT ERROR
	connected	COM PORT CONNECTION FAIL.
		CHECK CONNECTION.
1601		<error cancellation=""></error>
1001		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
	COM port error	
		[CLOSE] Close error display
ERR# 0-2000 -		
	Replace the purified water	Replace from purified water to washing solution.
	with the washing solution	After the replacement, press [Close] button.
2002		<error cancellation=""></error>
2003		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
	(The message is not printed.)	
		[CLOSE] Close error display
	Shutdown error Warning	The system was shut down abnormally last time.
		The hard disk may be damaged.
		If there is something wrong with the hard disk after system
2004		startup, please contact our seller.
		<error cancellation=""></error>
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.
	Shutdown error Warning	[CLOSE] Start up the system.

Error No.	Screen Message (Upper)		
ERR#0-	Printed Message (Lower)	Error Description and Cancelling Procedure	
ERR	ERR# 0-3000 -		
	Reagent shortage	Reagent shortage.	
		Finish analysis.	
2001		<error cancellation=""></error>	
3001		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
	Reagent shortage		
		[CLOSE] Close error display	
	Empty cell shortage	Analyzing cell shortage.	
		Finish analysis.	
3002		<error cancellation=""></error>	
3002		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
	Empty cell shortage		
		[CLOSE] Close error display	
	STD or QC installation error	STD or QC installation error	
		Finish analysis.	
3003		<error cancellation=""></error>	
3003		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
	Installation error		
		[CLOSE] Close error display	
	QC expiration date expired	QC expiry date expired.	
		Finish analysis.	
3004		<error cancellation=""></error>	
3004		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
	QC expired		
		[CLOSE] Close error display	
3005	Test item is undecided	Test item setting error	
		Finish analysis.	
		<error cancellation=""></error>	
		PRESS THE FOLLOWING BUTTON TO SOLVE THE ERROR.	
	Test item is undecided		
		[CLOSE] Close error display	

3.2 ERR# 1-001 - 1-200(SS1)

Error No.	Screen Message (Upper)		
ERR#1-	Printed Message (Lower)	Error Description and Cancelling Procedure	
ERR	ERR# 1-001 -		
	Sample nozzle Z-axis origin	The Z-axis of the sample nozzle could not be moved to the origin on	
	error	the reaction table.	
002	(Test)	Analyze the sample currently under analysis again.	
	SAMP Z ORG error	<error cancellation=""></error>	
		Press [ABORT]. The assay ends.	
	Sample nozzle Z-axis origin	The Z-axis of the sample nozzle could not be moved to the origin on	
	error	the rack or on the overflow cell.	
003	(Test)	Wait for the results for samples currently under analysis.	
	SAMP Z ORG error	Re-analyze the errored samples and unmeasured samples.	
		<error cancellation=""></error>	
		Press [Close]. The Error screen closes.	
	Sample nozzle Z-axis origin	The Z-axis of the sample nozzle could not be moved to the origin.	
	error	<error cancellation=""></error>	
004	(Out of test)	1. If there is no error at the mechanical movement, press [Retry]. The	
	SAMP Z ORG error	process restarts.	
		2. If there is an error at the mechanical movement or if an error is	
		issued again, press [ABORT]. Stop the process.	
	Sample nozzle Theta-axis origin	The Theta-axis of the sample nozzle could not be moved to the	
005	error (shaft origin error	origin. Wait for the results for samples currently under analysis.	
005	(Test)	Re-analyze the errored samples and unmeasured samples.	
	SAMP θ ORG error	<error cancellation=""></error>	
		Press [Close]. The Error display closes.	
	Sample nozzle Theta-axis origin	The Theta-axis of the sample nozzle could not be moved to the origin.	
	error	<pre><error cancellation=""></error></pre>	
006	(Out of test)	1. If there is no error at the mechanical movement, press [Retry]. The	
	SAMP θ ORG error	process restarts.	
		2. If there is an error at the mechanical movement or if an error is	
		issued again, press [ABORT]. Stop the process.	
	Sample dispensing pump origin	The syringe of the sample dispensing pump could not be moved to	
	error	the origin.	
007	(Test)	Wait for the results for samples currently under analysis.	
	SAMP P ORG error	Re-analyze the errored samples and unmeasured samples.	
		<pre><error cancellation=""></error></pre>	
		Press [Close]. The Error display closes.	

Error No.	Screen Message (Upper)	
ERR#1-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Sample dispensing pump	The syringe of the sample dispensing pump could not be moved to the
	origin error	origin.
	(Out of test)	<error cancellation=""></error>
008	SAMP P ORG error	1. If there is no error at the mechanical movement, press [Retry]. The
		process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Rack installation lane origin	The rack lane could not be moved to the origin.
	error	<error cancellation=""></error>
009	(Out of test)	1. Press [ABORT]. Stop the process.
	Rack installation lane ORG	
	error	
	Rack conveyance lane origin	The rack lane could not be moved to the origin.
	error	<error cancellation=""></error>
010	(Out of test)	1. Press [ABORT]. Stop the process.
	Rack conveyance lane ORG	
	error	
	Sample nozzle Z-axis jam	The Z-axis of the sample nozzle jammed while descending.
011	error	Wait for the results for samples currently under analysis.
011	(Test)	Re-analyze the errored samples and unmeasured samples.
	SAMPZ JAM error	<error cancellation=""></error>
		Press [Close]. Close Error display.
	Sample nozzle Z-axis jam error	The Z-axis of the sample nozzle jammed during descending.
	(Out of test)	<pre><error cancellation=""></error></pre>
012	SAMPZ JAM error	1. If there is no error at the mechanical movement, press [Retry]. The
		process restarts.
		2. If there is an error at the mechanical movement or if an error is
	Comple noggle ions conserver	issued again, press [ABORT]. Stop the process.
	Sample nozzle jam sensor error	The jam sensor did not work correctly at the sample nozzle. Wait for the results for samples currently under analysis.
013	(Test)	Re-analyze the errored samples and unmeasured samples.
013	SAMP jam sensor error	Ke-analyze the errored samples and unineasured samples. <error cancellation=""></error>
		Press [Close]. Close Error display.
	Sample nozzle jam sensor error	The jam sensor did not work correctly at the sample nozzle.
	(Out of test)	The juin sensor are not work correctly at the sample nozzle.
	SAMP jam sensor error	<error cancellation=""></error>
014		1. If there is no error at the mechanical movement, press [Retry]. The
		process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.

Error No.	Screen Message (Upper)	
ERR#1-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Rack conveyance error	The rack could not be pulled into the end in the equipment during the
	Rack conveyance error	rack transfer.
022		A rack or other obstacle may exist on the transfer line.
022		<error cancellation=""></error>
		1. Press [ABORT].
		2. Remove the rack on the transfer line.
	Rack detection error	While the rack transfer, the rack could not be detected.
		The result of this rack is not output.
		After the analysis is completed, remove any racks in the rack set
023		position
025		and initialize with the Prep functions.
		Check the rack orientation and analyze again.
		<error cancellation=""></error>
	Rack detection error	1. Press [Close]. Close Error display.
	Rack discharge error	While unloading the rack, the rack could not be transferred to
	Rack discharge error	the set position.
		Or the rack could not be provided correctly.
024		An obstacle may exist on the transfer line.
024		The result for this rack is not output. Analyze again.
		If the analysis result of the previous rack is output, close the Error
		screen and abort.
		<cancellation></cancellation>
		Press [Close]. Close Error display.
	Sample nozzle No liquid error	The liquid level could not be detected.
021	(Test)	Wait for the results for samples currently under analysis.
031		Re-analyze the errored samples and unmeasured samples.
	CAMPAL 11 11	<pre><error cancellation=""></error></pre>
	SAMP No liquid	Press [Close]. Close Error display.
	Sample nozzle No liquid error	The liquid level could not be detected.
	(Out of test)	<pre><error cancellation=""></error></pre>
032		1. If there is no error at the mechanical movement, press [Retry]. The
		process restarts.
	CAMD No liquid	2. If there is an error at the mechanical movement or if an error is
033	SAMP No liquid	issued again, press [ABORT]. Stop the process.
	Sample nozzle Erroneous	The position which the liquid level was detected was not the expected position.
	liquid level detection error	Wait for the results for samples currently under analysis.
	(Test)	Re-analyze the errored samples and unmeasured samples.
	SAMP nozzle Erroneous liquid	Ke-anaryze the errored samples and unmeasured samples. <error cancellation=""></error>
	level detection error	
		Press [Close]. Close Error display.

Error No.	Screen Message (Upper)	
ERR#1-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Sample nozzle Erroneous	The position which the liquid level was detected was not the expected
	liquid level detection error	position.
	(Out of test)	<error cancellation=""></error>
034	SAMP nozzle Erroneous liquid	1. If there is no error at the mechanical movement, press [Retry]. The
	level detection error	process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Overflow cell (for Sample	When cleaning the sample nozzle, the waste liquid in the overflow cell
	nozzle) Poor discharge	could not be discharged.
035	(Test)	Wait for the results for samples currently under analysis.
033	OF Poor discharge	Re-analyze the errored samples.
		<error cancellation=""></error>
		Press [Close]. The Error screen closes.
	Overflow cell (for Sample	When cleaning the sample nozzle, the waste liquid in the overflow cell
036	nozzle) Poor discharge	could not be discharged.
030	(Out of test)	<error cancellation=""></error>
	OF Poor discharge	Press [ABORT]. The process is aborted.
	Squeeze mechanism	The squeeze DC motor did not work correctly.
	Operation error	Wait for the results for samples currently under analysis.
041	(Test)	Re-analyze the errored samples and unmeasured samples.
	Squeeze error	<error cancellation=""></error>
		Press [Close]. Close Error display.
	Squeeze mechanism	The squeeze DC motor did not work correctly.
	Operation error	<error cancellation=""></error>
042	(Out of test)	1. If there is no error at the mechanical movement, press [Retry].
072	Squeeze error	Restart the process.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Puncturing	The puncture DC motor did not work correctly.
	mechanism Operation	Wait for the results for samples currently under analysis.
043	error	Re-analyze the errored samples and unmeasured samples.
	(Test)	<error cancellation=""></error>
	Puncture error	Press [Close]. Close Error display.
	Puncturing	The puncture DC motor did not work correctly.
	mechanism Operation	<error cancellation=""></error>
044	error	1. If there is no error at the mechanical movement, press [Retry]. The
	(Out of test)	process restarts.
	Puncture error	2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.

Error No.	Screen Message (Upper)	
ERR#1-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Sample and rack barcode	The communication with the barcode reader did not work correctly.
071	reader error	Analyze the sample currently under analysis again.
0/1	(Test)	<error cancellation=""></error>
	SAMP barcode reader error	Press [ABORT]. The assay ends.
	Sample and rack barcode reader	The communication with the barcode reader did not work correctly.
	error	Wait for the results for samples currently the analysis.
	(Test)	Re-analyze the errored samples and unmeasured samples.
072	SAMP barcode reader error	<error cancellation=""></error>
072		① Press [ABORT].
		② The assay ends when [ABORT] is pressed on the displayed dialog.
		When [Close] is pressed, the current sample is analyzed through the end,
		but no additional samples will be analyzed.
	Sample and rack barcode reader	The communication with the barcode reader did not work correctly.
	error	<error cancellation=""></error>
073	(Out of test)	1. If there is no error at the mechanical movement, press [Retry]. The
	SAMP barcode reader error	process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Sample and rack barcode reader	Reading the barcode did not complete within the specified time.
074	Communication timeout error	Analyze the sample currently under analysis again.
	(Test)	<error cancellation=""></error>
	Barcode communication timeout	Press [ABORT]. The assay ends.
	Sample and rack barcode reader	The reagent could not be detected.
	Communication timeout error	Wait for the results for samples currently under analysis.
	(Test)	Re-analyze the errored samples and unmeasured samples. <error cancellation=""></error>
075	Barcode communication timeout	① Press [ABORT].
		② The assay ends when [ABORT] is pressed on the displayed dialog.
		When [Close] is pressed, the current sample is analyzed through the end,
		but no additional samples will be analyzed.
	Sample and rack barcode reader	Reading the barcode did not complete within the specified time.
	Communication timeout error	<error cancellation=""></error>
076	(Out of test)	1. If there is no error at the mechanical movement, press [Retry]. The
076	Barcode communication timeout	
		2. If there is an error at the mechanical movement or if an error is issued
		again, press [ABORT]. Stop the process.

Error No.	Screen Message (Upper)	
ERR#1-	Printed Message (Lower)	Error Description and Cancelling Procedure
ERR	#1-101 -	
	Set the rack.	<cancellation></cancellation>
101	(The message is not printed.)	1. Set the rack on the lane and press [Continue]. The process
101		continues.
		2. When aborting the process, press [ABORT].
	Close the rack setting cover.	The process temporarily stops.
105	(The message is not printed.)	<cancellation></cancellation>
103		1. Close the rack lane cover and press [Retry]. The process restarts.
		2. If the message is issued again, press [ABORT]. Stop the process.
	Remove the rack from the	The rack could not be ejected.
	displayed lane No.	<cancellation></cancellation>
108	(Test)	1. Remove the racks on the displayed lane No. and press [Retry].
	(The message is not printed.)	Restart the process.
		2. When aborting the process, press [ABORT].
	Remove the rack from the	The rack transfer motor was returned to the origin.
109	displayed lane No.	<cancellation></cancellation>
	(Out of test)	1. Remove the rack of lane 1 or lane 2 and press [Continue]. The
	(The message is not printed.)	process continues.
	,	2. When aborting the process, press [ABORT].

3.3 ERR# 2-001 - 2-200(SS2)

Error No.	Screen Message (Upper)		
ERR#2-	Printed Message (Lower)	Error Description and Cancelling Procedure	
ERR	ERR#2-001 -		
002	Reagent nozzle Z-axis origin error (Test) REAGZ ORG error	The Z-axis of the reagent nozzle could not be moved to the origin on the reaction table. Analyze the sample currently under analysis again. <error cancellation=""> Press [ABORT]. The assay ends.</error>	
003	Reagent nozzle Z-axis origin error (Test) REAGZ ORG error	The Z-axis of the Reagent nozzle could not be moved to the origin on the reagent refrigerator or on the overflow cell. Wait for the results for samples currently under analysis. Re-analyze the errored samples and unmeasured samples. <error cancellation=""> Press [Close]. The Error display closes.</error>	
004	Reagent nozzle Z-axis origin error (Out of test) REAGZ ORG error	The Z-axis of the reagent nozzle could not be moved to the origin. <error cancellation=""> 1. If there is no error at the mechanical movement, press [Retry]. The process restarts. 2. If there is an error at the mechanical movement or if an error is issued again, press [ABORT]. Stop the process.</error>	
005	Reagent nozzle Theta-axis origin error (Test) REAG θ ORG error	The Theta-axis of the reagent nozzle could not be moved to the origin. Wait for the results for samples currently under the analysis. Re-analyze the errored samples and unmeasured samples. <error cancellation=""> Press [Close]. The Error display closes.</error>	
006	Reagent nozzle Theta-axis origin error (Out of test) REAG θ ORG error	The Theta-axis of the reagent nozzle could not be moved to the origin. <error cancellation=""> 1. If there is no error at the mechanical movement, press [Retry]. The process restarts. 2. If there is an error at the mechanical movement or if an error is issued again, press [ABORT]. Stop the process.</error>	
007	Reagent dispensing pump Origin error (Test) REAGP ORG error	The syringe of the reagent dispensing pump could not be moved to the origin. Wait for the results for samples currently under analysis. Re-analyze the errored samples and unmeasured samples. <error cancellation=""> Press [Close]. The Error display closes.</error>	

Error No.	Screen Message (Upper)	
ERR#2-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Reagent dispensing pump	The syringe of the reagent dispensing pump could not be moved to the
	Origin error	origin.
	(Out of test)	<error cancellation=""></error>
008	REAGP ORG error	1. If there is no error at the mechanical movement, press [Retry]. The
		process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Reaction table Origin error	The motor on the reaction table could not move to the origin.
	Reaction table ORG error	<error cancellation=""></error>
010		1. If there is no error at the mechanical movement, press [Retry]. The
010		process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Reagent refrigerator Origin	The motor of the reagent refrigerator could not be moved to the origin.
	error	<error cancellation=""></error>
012	REAG rotation ORG error	1. If there is no error at the mechanical movement, press [Retry]. The
012		process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Reagent nozzle Z-axis jam error	The Z-axis of the reagent nozzle jammed during descending.
0.1.2	(Test)	Wait for the results for samples currently under analysis.
013	REAGZ JAM error	Re-analyze the errored samples and unmeasured samples.
		<error cancellation=""></error>
		Press [Close]. Close Error display.
	Reagent nozzle Z-axis jam error	The Z-axis of the reagent nozzle jammed during descending.
	(Out of test)	<error cancellation=""></error>
014	REAGZ JAM error	1. If there is no error at the mechanical movement, press [Retry]. The
		process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Reagent nozzle Jam sensor error	
015	(Test)	Wait for the results for samples currently under analysis.
015	REAG jamming sensor error	Re-analyze the errored samples and unmeasured samples.
		<error cancellation=""></error>
	D (1.7	Press [Close].
	Reagent nozzle Jam sensor error	A jam sensor error was detected at the reagent nozzle.
016	(Out of test)	<pre><error cancellation=""> 1. If the region of the green beginning to the green of t</error></pre>
	REAG jamming sensor error	1. If there is no error at the mechanical movement, press [Retry]. The
		process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.

Error No.	Screen Message (Upper)	
ERR#2-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Reaction table Step-out error	The pulse motor on the reaction table stepped out.
017	Table motor step-out	<error cancellation=""></error>
		1. Press [ABORT]. End the assay.
	Reagent refrigerator Step-out	The pulse motor of the reagent refrigerator stepped out.
	error	Wait for the results for samples currently under analysis.
018	Refrigerator motor step-out	Re-analyze the errored samples and unmeasured samples.
		<error cancellation=""></error>
		Press [Close].
	Pulse motor Step-out error	The pulse motor stepped out.
	Motor step-out	<motor></motor>
		1: Reaction table
010		6: Reagent refrigerator
019		<error cancellation=""></error>
		1. If there is no error at the mechanical movement, press [Retry]. It is
		initialized, and the process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Reagent nozzle No	The liquid level could not be detected.
021	liquid error	Wait for the results for samples currently under analysis.
031	(Test)	Re-analyze the errored samples and unmeasured samples.
	REAG No liquid	<pre><error cancellation=""></error></pre>
	D	Press [Close]. Close Error display.
	Reagent nozzle No	The liquid level could not be detected. <error cancellation=""></error>
	liquid error (Out of	
032	test)	1. If there is no error at the mechanical movement, press [Retry]. The process restarts.
	REAG No liquid	2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. The process stops.
	Reagent nozzle Erroneous	The position which the liquid level was detected was not the
	liquid level detection error	expected position.
	(Test)	Wait for the results for samples currently under analysis.
033	REAG Liquid level error	Re-analyze the errored samples and unmeasured samples.
	TEMO Elquid level ellor	<pre><error cancellation=""></error></pre>
		Press [Close]. Close Error display.
	Reagent nozzle Erroneous	The position which the liquid level was detected was not the expected
	liquid level detection error	position.
	(Out of test)	<pre><error cancellation=""></error></pre>
034	REAG Liquid level error	1. If there is no error at the mechanical movement, press [Retry]. The
	-	process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. The process is aborted.

Error No.	Screen Message (Upper)	
ERR#2-	Printed Message (Lower)	Error Description and Cancelling Procedure
037	Overflow cell (for Reagent nozzle) Poor discharge (Test) OF Poor discharge	When cleaning the regent nozzle, the waste liquid in the overflow cell could not be discharged. Wait for the results for samples currently under analysis. Re-analyze the errored samples. <error cancellation=""> Press [Close]. The Error screen closes.</error>
038	Overflow cell (for Reagent nozzle) Poor discharge (Out of test) OF Poor discharge	When cleaning the reagent nozzle, the waste liquid in the overflow cell could not be discharged. <error cancellation=""> Press [ABORT]. The process is aborted.</error>
041	Reagent refrigerator Shutter operation error (Test) Reagent refrigerator Shutter operation error	The shutter of the reagent refrigerator did not work correctly. Wait for the results for samples currently under analysis. Re-analyze the errored samples and unmeasured samples. <error cancellation=""> Press [Close]. Close Error display.</error>
042	Reagent refrigerator shutter operation error (Out of test) Reagent refrigerator shutter error	The shutter of the reagent refrigerator did not work correctly. <error cancellation=""> 1. If there is no error at the mechanical movement, press [Retry]. The process restarts. 2. If there is an error at the mechanical movement or if an error is issued again, press [ABORT]. Stop the process.</error>
043	Reagent refrigerator barcode reader error REAG barcode reader error	The communication with the barcode reader did not work correctly <error cancellation=""> 1. If there is no error at the mechanical movement, press [Retry]. The process restarts. 2. If there is an error at the mechanical movement or if an error is issued again, press [ABORT]. Stop the process.</error>
044	Mixer operation error (Test) Mixer operation error	The mixer arm DC motor did not work correctly above the reaction table. Wait for the result for the currently testing sample. Retest error samples and untested samples. <error cancellation=""> Press [ABORT]. The test ends.</error>

Error No.	Screen Message (Upper)	
ERR#2-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Mixer operation	The mixer arm DC motor did not work correctly above the reaction
	error	table.
	(Out of test)	<error cancellation=""></error>
046	Mixer operation error	1. If there is no error at the mechanical movement, press [Retry].
		The process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Mixing operation	The mixer DC motor did not work correctly.
	error	Wait for the results for samples currently under analysis.
047	(Test)	Re-analyze the errored samples and unmeasured samples.
	Mixing error	<error cancellation=""></error>
		Press [Close]. The Error screen closes.
	Mixing operation	The mixer DC motor did not work correctly.
	error	<error cancellation=""></error>
048	(Out of test)	1. If there is no error at the mechanical movement, press [Retry].
040	Mixing error	The process restarts.
		2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
	Temperature control error	An error was detected in the temperature control.
	Temperature control error	Analyze the sample currently under analysis again.
		<error cancellation=""></error>
051		1. Press [ABORT].
		If an analysis is ongoing, such analysis will be ended.
		If any process other than analysis is ongoing, such process will be
		aborted.
	Detection control error	An error was detected in the detection control.
	Detection control error	Analyze the sample currently under analysis again.
0.50		<error cancellation=""></error>
052		1. Press [ABORT].
		If an analysis is ongoing, such analysis will be ended.
		If any process other than analysis is ongoing, such process will be
		aborted.

3

Error No.	Screen Message (Upper)	
ERR#2-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Liquid level detection control	An error was detected in the liquid level detection control.
	error	Analyze the sample currently under analysis again.
	Liquid level detection control	<error cancellation=""></error>
053	error	1. Press [ABORT].
		If an analysis is ongoing, such analysis will be ended.
		If any process other than analysis is ongoing, such process will be
		aborted.
	DC pump control error	An error was detected in the DC pump control.
	DC pump control error	Analyze the sample currently under analysis again.
		<error cancellation=""></error>
054		1. Press [ABORT].
		If an analysis is ongoing, such analysis will be ended.
		If any process other than analysis is ongoing, such process will be
		aborted.
	Purified water set position	Lack of solution in the purified water bottle was detected.
0.61	Purified water shortage	Wait for the results for samples currently under analysis.
061	(Test)	Re-analyze the errored samples and unmeasured samples.
001	Purified water shortage	<pre><error cancellation=""></error></pre>
		Press [Close].
	Purified water set position	Lack of solution in the purified water bottle was detected.
062	Purified water shortage	<pre><error cancellation=""></error></pre>
002	(Out of test)	1. When the bottle is replaced, press [Retry]. The process restarts.
	Purified water shortage	2. If there is an error at the mechanical movement or if an error is issued
	Washing solution set	again, press [ABORT]. Stop the process. Lack of solution in the washing solution bottle was detected.
	position Washing solution	Wait for the results for samples currently under analysis.
063	shortage	Re-analyze the errored samples and unmeasured samples.
003	(Test)	Ke-anaryze the criored samples and difficastred samples. <error cancellation=""></error>
	Washing solution shortage	Press [Close].
	Washing solution set	Lack of solution in the washing solution bottle was detected.
	position Washing solution	<pre><error cancellation=""></error></pre>
064	shortage	1. When the bottle is replaced, press [Retry]. Restart the process.
	(Out of test)	2. If there is an error at the mechanical movement or if an error is
	Washing solution shortage	issued again, press [ABORT]. Stop the process.
	Safety guard error	The analysis was ended because the safety guard was opened during an
	Safety guard open error	analysis.
071	Same open sitor	Analyze the sample currently under analysis again.
		<pre><error cancellation=""></error></pre>
		Press [ABORT]. The assay ends.

3 Error List

Error No.	Screen Message (Upper)	
ERR#2-	Printed Message (Lower)	Error Description and Cancelling Procedure
	Mixer nozzle Presence sensor	The mixer nozzle could not be detected in the overflow cell.
	error	Wait for the results for samples currently under analysis.
072	(Test)	Re-analyze the errored samples and unmeasured samples.
	Mixer nozzle Presence sensor	<error cancellation=""></error>
	error	Press [Close].
	Mixer nozzle Presence sensor	The mixer nozzle could not be detected in the overflow cell.
	error	<error cancellation=""></error>
073	(Out of test)	1. If there is no error at the mechanical movement, press [Retry]. The
073	Mixer nozzle Presence sensor	process restarts.
	error	2. If there is an error at the mechanical movement or if an error is
		issued again, press [ABORT]. Stop the process.
ERR	#2-101 -	
	Close the safety guard.	The process temporarily stops.
101	(The message is not printed.)	<cancellation></cancellation>
101		1. Close the safety guard and press [Retry]. Restart the process.
		2. If the error is issued again, press [ABORT]. Stop the process.
	Close the reagent refrigerator	The process temporarily stops.
102	cover.	<cancellation></cancellation>
102	(The message is not printed.)	1. Close the reagent refrigerator and press [Retry]. Restart the process.
		2. If the error is issued again, press [ABORT]. Stop the process.
103	Check the reagent again.	<cancellation></cancellation>
105	(The message is not printed.)	Close the reagent refrigerator cover and press [Continue].
	Remove the pipes of purified	<cancellation></cancellation>
107	water and washing solution.	1. Remove pipes and press [Continue]. The process continues.
	(The message is not printed.)	2. When aborting the process, press [ABORT].
	Replace the washing solution	<cancellation></cancellation>
	line to the purified water.	1. Replace the washing solution line to the purified water. Disconnect
111	(The message is not printed.)	the pipe connected to the washing solution bottle and connect the
		bottle to the purified water container.
		2. Press [Continue]. When aborting the process, press [ABORT].

4 Saving to External Media

The following data can be saved to external media: Sample measured data information, Sample replication information, QC measured data information, QC replication information, STD measured data information, STD replication information, and Time course data information.

Output data are variable-length.

4.1 Sample Measured Data Information

Sample Measured Data Information is saved.

■ File name: ocsamp.csv

No.	Item name	Note (△ is a blank [20H]))	Output
			setting
1	Data type	Analysis date, re-measurement data: 'N △'	Yes
		Retest data (buffer retest included): 'A \triangle '	
		If the data is edited, set "E" to \triangle .	
		Example) Analysis data edited: 'NE'	
2	Date of analysis	Date of analysis	_
		Example) September 23, 2020 -> 2020/09/23	
3	Time of analysis	Time of analysis (24-hour notation)	_
		Example: 2:05 P.M> 14:05	
4	Rack No.	Barcode information put on the rack	_
5	Position in rack	Positions in the rack: 1 - 10	_
6	Sample ID	Barcode information put on the sampling bottle (*1)	Yes
7	Sample sequence No.	Measurement sequence number: 1 to 99999	_
8	Measurement method counter	Unused. ",," is output.	Yes
9	Number of replications	Number of replications in the replication measurements:	_
		1 - 10	
10	DA value	Value obtained by No. 3 detection – No. 1 detection	Yes
		See "Appendix 1.2 Calculation of DA Value."	

4 Saving to External Media

No.	Item name	Note (△ is a blank [20H])	Output setting
11	Measured data	Calculated measured data: Up to 7 digits in the integer part and 1 digit in the decimal part (the number of digits in the decimal part can be changed in the "Output Format Settings.")	Yes
12	Judgment result	Qualitatively converted result of measured data by the cut off value \triangle -, \triangle +, 1+, 2+, and 3+	Yes
13	SD value	SD value of measured data (statistic)	Yes
14	CV value	CV value of measured data (statistic)	Yes
15	Error code	See "Appendix 4.8 List of Error Codes for External Media Output."	Yes
16	Test item code	Measured item code Not output in order error (CSV format. Thus, it is displayed as , ,).	_
17	Test item name	Name of measured item Not output in order error (CSV format. Thus, it is displayed as , ,).	Yes
18	Units of inspection result	Units of measure inspection result	Yes
19	Normal value range 1	Set cut off 1 value	Yes
20	Normal value range 2	Set cut off 2 value ("*" when not set)	Yes
21	Normal value range 3	Set cut off 3 value ("*" when not set)	Yes
22	Operator ID	ID of logged-in operator	Yes
23	R1 reagent lot	Lot of used R1 reagent	Yes
24	R1 reagent expiration date	Expiration date of used R1 reagent lot	Yes
25	R2 reagent lot	Lot of used R2 reagent	Yes
26	R2 reagent expiration date	Expiration date of used R2 reagent lot	Yes
27	Buffer expiration date	Lot of used buffer (, , is output when brand new)	Yes
28	Buffer expiration date	Expiration date of used buffer (, , is output when the expiration date is unknown)	Yes
29	Dilution factor	Sample dilution factor in a retest (0 is output in regular tests) 0, 1, 10, 20, 100, 200, 400	Yes



- The item name is placed at the head of a file name as a header followed by data.
- Each item is separated by a comma (,).
- Output order and content depend on the setting of output item selection of the output format setting.
- Zero suppression is performed for items other than date of analysis, time of analysis, rack number, test item code, and test item name.
- * 1 A maximum of 50 digits are output for sample IDs when 2D code is used (option).

4.2 Sample Replication Information

Sample replication information is saved.

■ File name: ocrsamp.csv

No.	Item name	Note	Output
			setting
1	Data type	See "Appendix 4.1 Sample Measured Data Information."	Yes
2	Date of analysis	See "Appendix 4.1 Sample Measured Data Information."	_
3	Time of analysis	See "Appendix 4.1 Sample Measured Data Information."	_
4	Relative cell No.	Used cells No.: 1 - 55	Yes
5	Rack No.	See "Appendix 4.1 Sample Measured Data Information."	_
6	Position in rack	See "Appendix 4.1 Sample Measured Data Information."	_
7	Sample ID	See "Appendix 4.1 Sample Measured Data Information."(* 1)	Yes
8	Sample sequence No.	See "Appendix 4.1 Sample Measured Data Information."	_
9	Replication counter	Number of replications in replication measurement: 1 - 10	_
10	A1 value	See "Appendix 1.2 Calculation of DA Value."	Yes
11	A2 value	See "Appendix 1.2 Calculation of DA Value."	Yes
12	A3 value	See "Appendix 1.2 Calculation of DA Value."	Yes
13	DA1 value	See "Appendix 1.2 Calculation of DA Value."	Yes
14	DA2 value	See "Appendix 1.2 Calculation of DA Value."	Yes
		It is output when there is no measured data.	
15	Measured data	See "Appendix 4.1 Sample Measured Data Information."	Yes
16	Judgment result	See "Appendix 4.1 Sample Measured Data Information."	Yes
17	Error code	See "Appendix 4.1 Sample Measured Data Information."	Yes
18	Test item code	Measured item code	_
19	Test item name	Measured item name	Yes
20	Units of inspection result	Measured test result units	Yes

4 Saving to External Media

No.	Item name	Note	Output setting
21	Normal value range 1	Set cut off 1 value	Yes
22	Normal value range 2	Set cut off 2 value ("*" when not set)	Yes
23	Normal value range 3	Set cut off 3 value ("*" when not set)	Yes
24	Operator ID	See "Appendix 4.1 Sample Measured Data Information."	Yes
25	R1 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
26	R1 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
27	R2 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
28	R2 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
29	Buffer expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
30	Sample dilution factor	See "Appendix 4.1 Sample Measured Data Information."	Yes
31	Dilution factor	See "Appendix 4.1 Sample Measured Data Information."	Yes
32	A0 value	See "Appendix 1.2 Calculation of DA Value."	Yes



- The item name is placed at the head of a file name as a header followed by data.
- Each item is separated by a comma (,).
- Output order and content depend on the setting of output item selection of the output format setting.
- Zero suppression is performed for items other than date of analysis, time of analysis, rack number, test item code, and test item name.
- · Sample uninstallation" is not output for Sample Replication Information.
- · Sample Replication Information is not output for no order and order error.
- * 1 The maximum of 50 digits are output for sample IDs when 2D code is used (option).

4.3 QC Measured Data Information

QC measured data information is saved.

■ File name: ocqc.csv

No.	Item name	Note	Output setting
1	Data type	QC data: 'Cx' x: 1 – 4 QC numbers	Yes
2	Date of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
3	Time of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
4	Rack No.	See "Appendix 4.1 Sample Measured Data Information."	Yes
5	Position in rack	See "Appendix 4.1 Sample Measured Data Information."	Yes
6	Sample sequence No.	See "Appendix 4.1 Sample Measured Data Information."	Yes
7	QC lot	QC lot	Yes
8	Number of replications	See "Appendix 4.1 Sample Measured Data Information."	Yes
9	DA value	See "Appendix 4.1 Sample Measured Data Information."	Yes
10	Measured data	See "Appendix 4.1 Sample Measured Data Information."	Yes
11	Judgment result	Not output	Yes
12	SD value	See "Appendix 4.1 Sample Measured Data Information."	Yes
13	Error code	See "Appendix 4.8 List of Error Codes for External Media Output."	Yes
14	Test item code	Measured item code	Yes
15	Test item name	Measured item name	Yes
16	QC ID	Barcode information put on the QC container (*1)	Yes
17	Units of inspection result	Units of measured inspection result	Yes
18	Operator ID	See "Appendix 4.1 Sample Measured Data Information."	Yes
19	Control expiration date	QC expiration date	Yes
20	R1 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
21	R1 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
22	R2 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
23	R2 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes

4 Saving to External Media



- The item name is placed at the head of a file name as a header followed by data.
- Each item is separated by a comma (,).
- · Output order and content depend on the setting of output item selection of the output format setting.
- Zero suppression is performed for items other than date of analysis, time of analysis, rack number, test item code, and test item name.
- * 1 A maximum of 50 digits are output for sample IDs when 2D code is used (option).

4.4 QC Replication Information

Information for each QC replication is saved.

■ File name: ocrqc.csv

No.	Item name	Note	Output
			setting
1	Data type	See "Appendix 4.3 QC Measured Data Information."	Yes
2	Date of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
3	Time of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
4	Relative cell No.	See "Appendix 4.2 Sample Replication Information."	Yes
5	Rack No.	See "Appendix 4.1 Sample Measured Data Information."	Yes
6	Position in rack	See "Appendix 4.1 Sample Measured Data Information."	Yes
7	Sample sequence No.	See "Appendix 4.1 Sample Measured Data Information."	Yes
8	QC lot	See "Appendix 4.3 QC Measured Data Information."	Yes
9	Replication counter	See "Appendix 4.2 Sample Replication Information."	Yes
10	A1 value	See "Appendix 1.2 DA Value Calculation."	Yes
11	A2 value	See "Appendix 1.2 DA Value Calculation."	Yes
12	A3 value	See "Appendix 1.2 DA Value Calculation."	Yes
13	DA1 value	See "Appendix 1.2 DA Value Calculation."	Yes
14	DA2 value	See "Appendix 1.2 DA Value Calculation."	Yes
15	Measured data	See "Appendix 4.1 Sample Measured Data Information."	Yes
16	Judgment result	Not output.	Yes
17	Error code	See "Appendix 4.8 List of Error Codes for External Media Output."	Yes
18	Test item code	Measured item code	Yes
19	Test item name	Measured item name	Yes
20	QC ID	Barcode information put on the QC container (*1)	Yes
21	Units of inspection result	Units of measured sample result	Yes

4 Saving to External Media

No.	Item name	Note	Output setting
22	Operator ID	See "Appendix 4.1 Sample Measured Data Information."	Yes
23	Control expiration date	QC expiration date	Yes
24	R1 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
25	R1 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
26	R2 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
27	R2 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
28	A0 value	See "Appendix 1.2 DA Value Calculation."	Yes



- The item name is placed at the head of a file name as a header followed by data.
- Each item is separated by a comma (,).
- · Output order and content depend on the setting of output item selection of the output format setting.
- Zero suppression is performed for items other than date of analysis, time of analysis, rack number, test item code, and test item name.
- * 1 A maximum of 50 digits are output for sample IDs when 2D code is used (option).

4.5 STD Measured Data Information

STD Measured Data Information is saved.

■ File name: ocstd.csv

No.	Item name	Note	Output setting
1	Date of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
2	Time of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
3	Rack No.	See "Appendix 4.1 Sample Measured Data Information."	Yes
4	Position in rack	See "Appendix 4.1 Sample Measured Data Information."	Yes
5	STD point No.	Measurement point No. (STD1 - STD6): 1 - 6	Yes
6	DA1 value	See "Appendix 1.2 DA Value Calculation."	Yes
7	DA2 value	See "Appendix 1.2 DA Value Calculation."	Yes
8	DA1 applying value	Concentration acquired by applying DA1 to CC	Yes
9	DA1CV value	DA1 CV value (statistic)	Yes
10	DA2CV value	DA2 CV value (statistic)	Yes
11	DA1SD value	DA1 SD value (statistic)	Yes
12	DA2SD value	DA2 SD value (statistic)	Yes
13	Theoretical value	STD concentration	Yes
14	Number of replications	STD Number of replications	Yes
15	Error code	See "Appendix 4.8 List of Error Codes for External Media Output."	Yes
16	Test item code	Measured item code	Yes
17	Test item name	Measured item name	Yes
18	R1 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
19	R1 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
20	R2 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
21	R2 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
22	Cal. lot	Calibrator lot	Yes
23	Cal. expiration date	Calibrator expiration date	Yes



- The item name is placed at the head of a file name as a header followed by data.
- Each item is separated by a comma (,).
- Output order and content depend on the setting of output item selection of the output format setting.
- Zero suppression is performed for items other than date of analysis, time of analysis, rack number, test item code, and test item name.

4.6 STD Replication Information

Information for each STD replication is saved.

■ File name: ocrstd.csv

No.	Item name	Note	Output setting
1	Date of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
2	Time of analysis	See "Appendix 4.1 Sample Measured Data Information."	Yes
3	Relative cell No.	See "Appendix 4.2 Sample Replication Information."	Yes
4	Rack No.	See "Appendix 4.1 Sample Measured Data Information."	Yes
5	Position in rack	See "Appendix 4.1 Sample Measured Data Information."	Yes
6	STD point No.	See "Appendix 4.5 STD Measured Data Information."	Yes
7	Replication counter	See "Appendix 4.2 Sample Replication Information."	Yes
8	A1 value	See "Appendix 1.2 Calculation of DA Value."	Yes
9	A2 value	See "Appendix 1.2 Calculation of DA Value."	Yes
10	A3 value	See "Appendix 1.2 DA Value Calculation."	Yes
11	DA1 value	See "Appendix 1.2 DA Value Calculation."	Yes
12	DA2 value	See "Appendix 1.2 DA Value Calculation."	Yes
13	Error code	See "Appendix 4.8 List of Error Codes for External Media Output."	Yes
14	Test item code	Measured item code	Yes
15	Test item name	Measured item name	Yes
16	R1 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
17	R1 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
18	R2 reagent lot	See "Appendix 4.1 Sample Measured Data Information."	Yes
19	R2 reagent expiration date	See "Appendix 4.1 Sample Measured Data Information."	Yes
20	Cal. lot	Calibrator lot	Yes
21	Cal. expiration date	Calibrator expiration date	Yes
22	A0 value	See "Appendix 1.2 Calculation of DA Value."	Yes

Note

- The item name is placed at the head of a file name as a header followed by data.
- Each item is separated by a comma (,).
- · Output order and content depend on the setting of output item selection of the output format setting.
- Zero suppression is performed for items other than date of analysis, time of analysis, rack number, test item code, and test item name.

4.7 Time Course Data Information

Information for the time course data is saved.

■ File name: tcourse.csv

No.	Item name	Note	Output setting
1	Cell information 1	Time course information of relative cell 1	_
			•
		•	-
n	Cell information n	Time course information of relative cell n	_

■ Cell information

No.	Item name	Note	Output setting
1	Date of analysis	See "Appendix 4.1 Sample Measured Data Information."	_
2	Time of analysis	See "Appendix 4.1 Sample Measured Data Information."	_
3	Relative cell No.	See "Appendix 4.2 Sample Replication Information."	_
4	Test item code	Measured item code	_
5	Test item name	Measured item name	_
6	Cell blank ABS value		
7	ABS value of 1st cycle	See "Appendix 1.2 DA Value Calculation."	_
8	ABS value of 2 nd cycle	See "Appendix 1.2 DA Value Calculation."	_
	•	•	
		•	
	•	·	•
26	ABS value of 20th cycle	See "Appendix 1.2 DA Value Calculation."	_



- The file contains only data without a header.
- Each item is separated by a comma (,).
- · CR/LF is added as a delimiter, and EOF is added at the end of the file.
- Zero suppression is performed for items other than date of analysis, time of analysis, rack number, test item code, and test item name.

4.8 List of Error Codes for External Media Output

Code	Error content	Measured data (*3)	Note
10	Sample barcode read error	Yes	
01	Sample shortage, No sample	_	Dispensing is not conducted.
02	No reagent error	_	*2
03	RBC (Prozone)	Yes	
04	PRC (Prozone)	Positive/Negative	
		judgment only	
05	OR (Over Range)	Positive/Negative	
		judgment only	
06	UR (Under Range)	_	
07	Sample dispense error	_	Dispensing is not conducted.
	(Sample nozzle jamming, etc.)		
08	Reagent dispense error	_	Dispensing of R1 reagent or R2 reagent
	(Reagent nozzle jamming, etc.)		is not conducted. Dispensing of buffer
			can also cause this error.
09	Mixer error	_	No mixing (agitation) is conducted.
	(Mixer jamming, etc.)		
0A	Reagent blank error	_	The reagent blank is abnormal.
0.0	(A1 check, DA1 check)		1 1 1 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2
0B	No CC	_	Includes mismatch of CC lot.
0D	QC control limit check error	_	v. 1
11	Combination error of "10" + "01" errors	_	*1
12	Combination error of "10" + "02" errors	_	*1
13	Combination error of "10" + "03" errors	Yes	*1
14	Combination error of "10" + "04" errors	_	*1
15	Combination error of "10" + "05" errors	_	*1
16	Combination error of"10" + "06" errors	_	*1
17	Combination error of"10" + "07" errors	_	*1
18	Combination error of"10" + "08" errors	_	*1
19	Combination error of"10" + "09" errors	_	*1
1A	Combination error of"10" + "0A" errors	_	*1
1B	Combination error of "10" + "0B" errors	_	*1

^{* 1 11 - 1}B is a combined error with two errors overlapped. No other error is overlapped.

^{* 2 &}quot;No reagent error" is output if the reagent volume could not be detected when dispensing the reagent is attempted.

^{*3} For errors with "Yes" in the "Measured data" field outputs the measured data and the judgment result.


For errors with "-" in the "Measured data" field outputs blank (20H) to the measured data and the judgment result.


5 Management USB Stick Setting

When saving measurement data or other data to a USB stick, the USB stick needs to be set as a management USB stick first.

This section describes the procedure to set a USB stick as a management USB stick.

5.1 How to Set a Management USB Stick

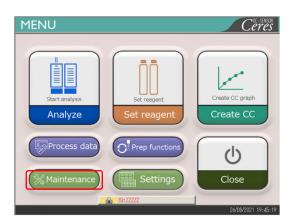


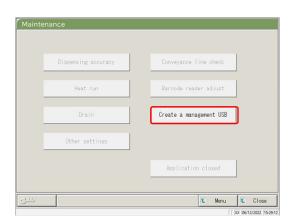


- 1 Set an administrator ID.
 - For setting an administrator ID, see "6.1.10 User Account Settings" on page 206.
- 2 Select "ON" for [Operator management].
 - For operator management setting, see "6.1.12 Operator Mode" on page 212.
- 3 Log in using the administrator ID.
 - See "2.3 Logging In" on page 28.
- 4 Touch [Maintenance] on the [MENU] screen.
 - * [Maintenance] screen opens.


- 5 {Create a management USB} button.
 - * [Create a management USB] screen opens.

5 Management USB Stick Setting

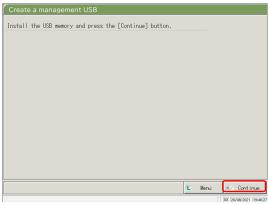

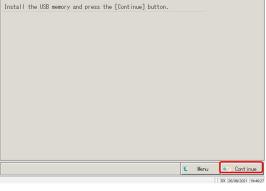

Create a management USB Install the USB memory and press the [Continue] button. - Cancel

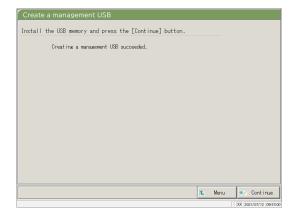

- 6 Insert the USB stick.
- 7 Touch {Continue}.

8 Touch {Start}.

{Cancel}: Closes the dialog box.

{Discard}: Aborts setting and returns to [Maintenance]

{Start}: Set the inserted USB stick as a management USB stick.

- 9 The inserted USB stick has been successfully set as a management USB stick.
- 10 Remove the USB stick from the system.

Index/Glossary

Index

A		
Abort		76
Accuracy control		137
Additional setting		70
Analysis		
	— completion of analysis	75
	— item setting	198
	— main screen during analysis	23
В		
Backup		32, 215
C		
Calibrator		47
CC protocol setting		34, 224
Cell blank		83
Cleaning		84, 170
Close		75
Config.		32, 190
Control panel		10

D			
DA value calculation			236
Daily operation flow			35
Data output			32, 192
Data output destination			
selection			192
Drain tank			
	— check		46
	— process		91
Е			
Error			
	— cancellation		230
		— button	229
	— list		248
	— log		88
	— screen		228
External media			44, 203, 271, 283
I			
Initial test			60
Initialization			162
Intra-day/Inter-day data			
	— deletion		147
	— editing		145
	— range specification		149
	— screen		142
Item selection			116, 125

L Language setting 32, 210 List of users 30 Login 28 — changing password 159 Logout 158 M Measurement cell 78 replacement MENU screen 16 O Online settings 195 Operator mode 32, 212 Output format setting 32, 200

Index

P 86, 178 Parts checklist PRC method 239 Primary power switch 10 Prime 164 Printer 10, 44 Printing — Example of real-time printing 241 — (CC) 56 — (Sample time course) 131 — (STD time course) 132 33,217 Protocol settings 238 Prozone judgment Purified water bottle 41, 43 QC — lot — deletion 141 — list 137 - selection 140 65 - process

- sample

Index/Glossary

Qualitative judgment

47, 65

237

R

Rack		
	— information	73
	— setting	32, 188
Range specification		
	—(intra-day/inter-day data)	149
	—(replicate data)	123, 125
	—(test data)	102
RBC method		238
Reagent		
	— barcode	40, 50, 52
	— blank check	240
	— setting	25, 37
	— volume	72
Replication		
	— data item selection	116, 125
	— data range specification	123, 125
	— data search	126
	— list display	114
Restore		32, 216
Retest		62

S

8		
Safety guard		10
Sample		
	— additional setting	70
	— barcode setting	32, 182, 184
	— ID editing	112
	— setting	58
Sample/QC protocol settings		34, 217
	— check	56
	— creation	47
	— editing/re-calculating	128
	— list	57
STD		
	— replicate	118, 120
	— sample	47
System setting		31, 182
System switch		10
T		
Test data		
	measurement datespecification	100
	— output	109
	— range specification	102
	— screen indication	96
	— search	106
Time course		
Time course	—(sample)	131, 133
Time course	—(STD)	132, 135
		<i>'</i>

U		
User account setting		32, 206
W		
Washing		166
Washing solution bottle		42, 43
X		
X-R control		
	— display	151
	— range change	155

Glossary

Α

ABS

A0, A1, A2, and A3

Logarithm of the ratio of the strength of incident light striking a sample (Io) and the strength of the transmitted light (I) (i.e. Io/I).

Analysis mode

Used when the sample is analyzed for the first time. One of the measurement modes.

В

Back fit value

The value obtained by fitting absorbance to a calibration curve.

D

DA value

Absorbance difference. Calculated as the amount of change in absorbance. For example,

DA1 = A3-A1 [ABS]

DA2 = A2-A0 [ABS]

Data

The concentration value of a target, such as hemoglobin.

Measured data (concentration values) calculated using DA values and calibration curves.

J

Jam error

An error indicating that something is touching the sample nozzle or mixer.

P

PRC method

Compares the DA2 values of STD-6 (the highest concentration of an STD sample series) and a patient sample.

Prozone

Phenomenon where there are excessive antibodies or antigens, and observable reactions in a mixture of specific antigens and antibodies no longer occur.

This is seen in high value samples. Samples where the amount of change decreases in the late stage of a reaction are referred to as prozone samples.

Q

QC (sample)

Control sample. Also referred to as reference material or a reference sample.

OC value check

Check based on the control limit values set on the [QC process] screen.

Specification:

When less than Min Value +1: Abnormal When Max Value +1 or more: Abnormal

R

RBC method

Compares the absorbance at the RBC point (in the initial stage of a reaction) of the highest concentration of an STD sample series (STD 6) and a patient sample.

Remeasure mode

Measures the sample again. One of the measure modes.

Remeasured samples are handled in the same manner as first test samples.

Retest mode

Retests samples measured in the analysis mode. However, they are not punctured again. One of the measurement modes.

Glossarv

S

STD (sample)

The calibrator sample. Also referred to as standard material or standard sample.

T

Time course

Results (graph) of measuring/recording changes in absorbance as time passes.



Manufacturer

EIKEN CHEMICAL CO., LTD.

4-19-9, TAITO, TAITO-KU, TOKYO 110-8408, JAPAN

TEL: +81-280-56-2822

FAX: +81-280-56-2707

URL:http://www.eiken.co.jp/en/ifu

